For who you are & what you need...

EN Steel Transport Chair

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2

EN CHECK ALL PARTS

Carefully examine all parts for shipping damage. DO NOT use the Transport Chair if there is apparent damage or if any parts are missing. If there are missing or damaged parts refer to the warranty on the other side of this guide.

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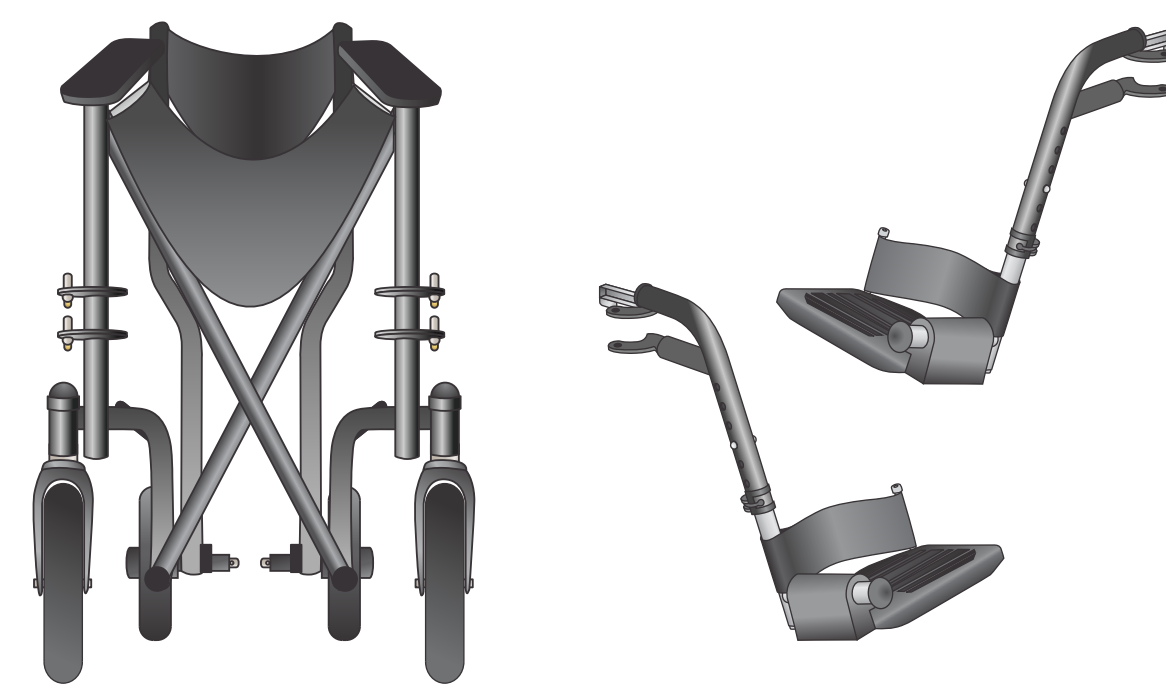
EN

READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR TRANSPORT CHAIR: You and your attendant should understand all instructions in the manual before operating.

- Consult your physician, nurse or physical therapist before using to determine how this Airgo® Transport Chair can suit your particular needs.
- IMPORTANT** - Before using, make sure that all parts are secure and that the frame of the chair is fully opened.
- This Airgo® Transport Chair is NOT intended to maneuver independently, an attendant is required.
- IMPORTANT** - Before entering or leaving the chair, engage the Wheel Locks against the tires on the Rear Wheels.
- Before entering or exiting the chair always ensure that 1) the chair is on a stable, level surface, 2) that the wheel locks are fully engaged, and 3) that the Footrest Plates are folded up.
- Always fasten the safety belt.
- Do not step on the Footrest Plates while entering or leaving the chair. Do not adjust the Footrest Plates any closer than 2 1/2 inches from the ground.
- Leaning out of the Airgo® Transport Chair without proper assistance could cause tipping.
- Always use caution when going down hills. Do not use the Wheel Locks as a brake while rolling the chair. Avoid changing direction when going down a hill.
- CAUTION** - DO NOT exceed the following weight limit, or serious injury could result:
Seat: 250 lbs.
- Your Airgo® Transport Chair should be checked periodically to ensure the brakes are functioning properly, all nuts and bolts are secure and the upholstery is undamaged.
- Proper care is required to keep your transport chair in good working condition. Refer to the maintenance instructions on the other side of this quickstart.

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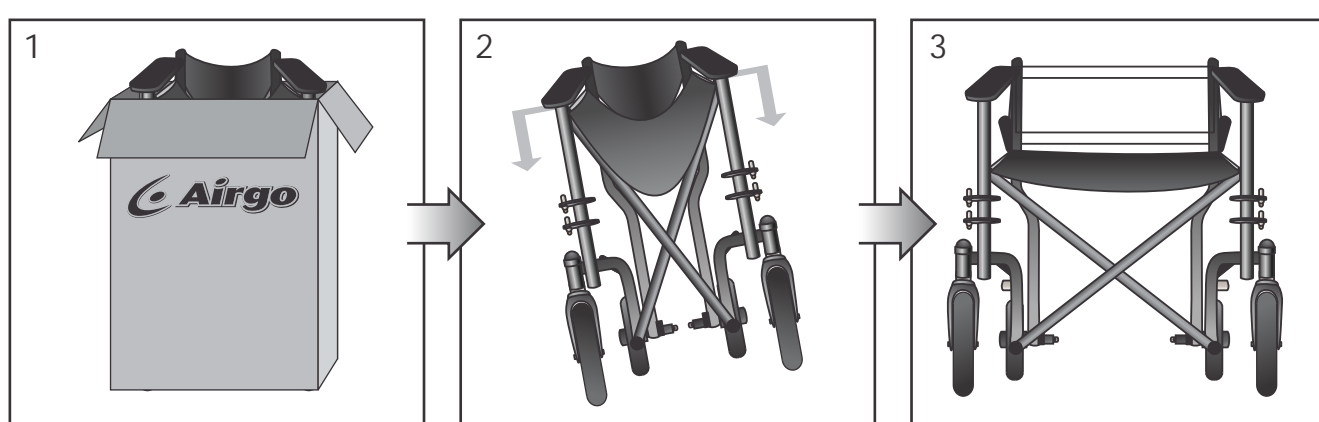
3

EN EXPAND THE FRAME

Remove the transport chair from its box. Stand the chair upright on all four wheels. Tilt to one side and push down on the side seat frame bars (2). Firmly push down on the seat frame underneath the armrests to ensure the seat is fully deployed (3).

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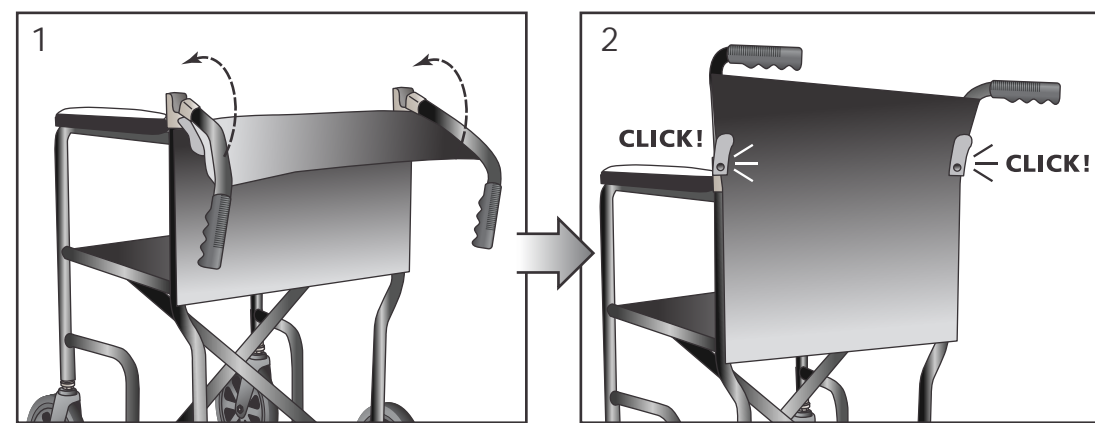
4

EN ENGAGE THE BACKREST

Grab both handlebars tucked next to frame and pull up lifting the backrest into the upright position (1) until the handlebars lock securely into place (2).

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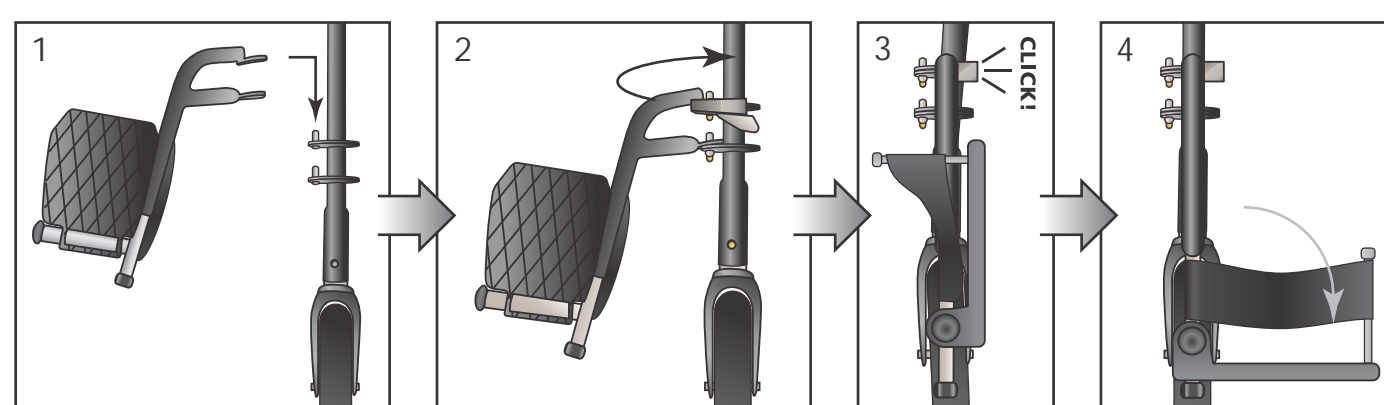
5

EN INSTALL FOOTRESTS

The chair has a right and left footrest. These must be installed correctly to ensure safety. To install footrests, face the front of the chair looking at the seat. Align the footrest holes with Alignment Pins on the front leg of the frame (1). Ensure that both pins are engaged, then swing Footrest (2) toward the center of chair until it locks into place (3). Repeat on opposite side. Do not lower footrest plates until safely in the chair (4). To enter or exit the chair, footrest plates must be folded up.

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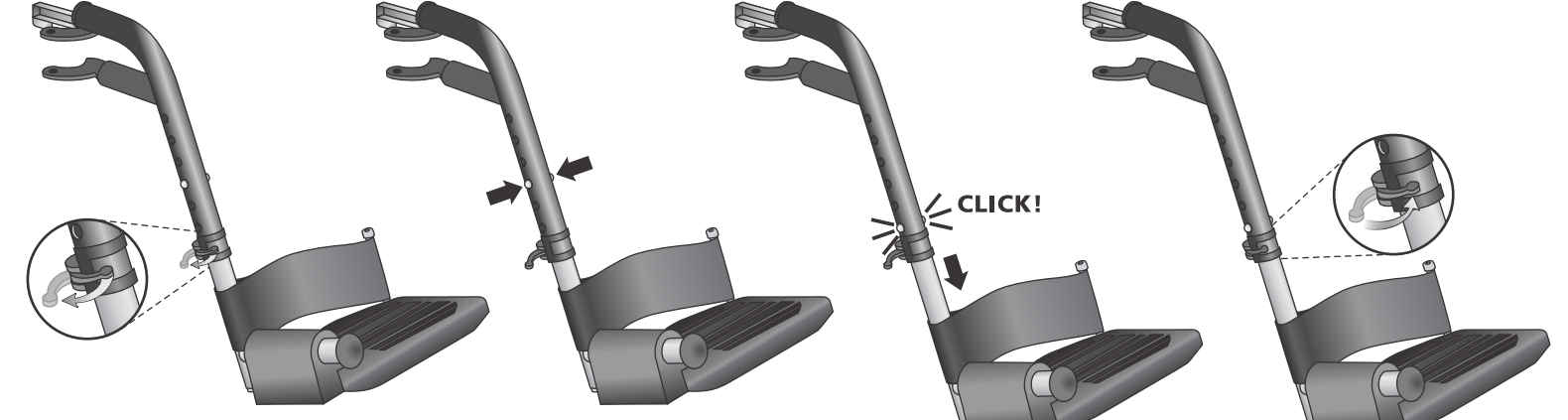
6

EN ADJUST THE FOOTRESTS

To adjust the length of the footrests, release the security clip then press the dual push buttons and select your desired height. The lowest part of the footrest should not be less than 2.5" from the ground. Ensure the push buttons have completely popped through the desired holes then snap the security clip back into place.

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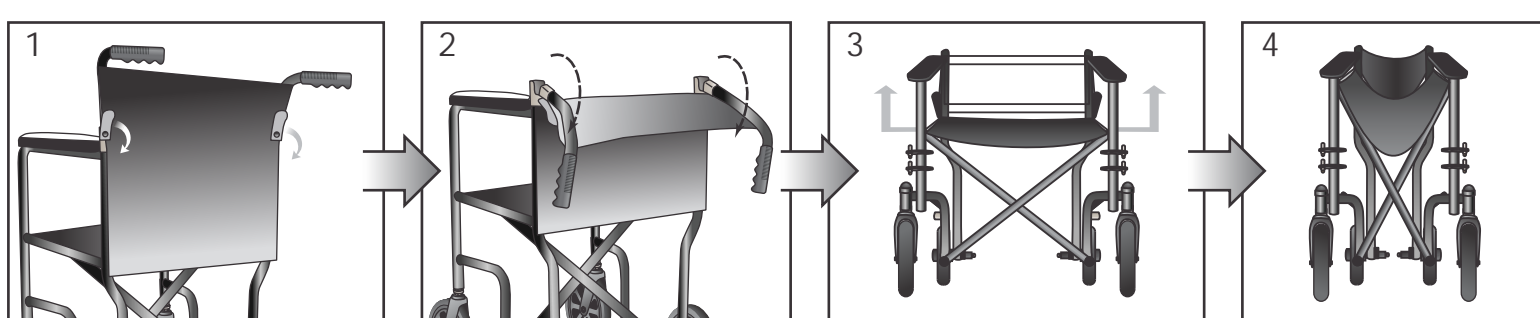
7

EN TO FOLD / STORE THE CHAIR

To close/fold or store chair, fold up the foot plates and ensure the swing away footrests are in locked position in front of the transport chair. Release the locks on both sides of back of chair (1) and fold down the backrest (2). Stand to one side and pull up on the seat upholstery handles, or tilt chair to one side and pull on front of seat while closing in with other hand on armrest (3-4).

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8

EN MAINTENANCE

- At least once a week, clean frame using a soft cloth. If exposed to moisture, dry the chair immediately.
- Clean upholstery with a mild soap solution, rinse and wipe dry. Worn upholstery should be replaced as it may not support your body weight.
- Inspect the wheels at least once a month. Correct any side play by loosening the lock nut on the axle, then tighten the axle just enough to remove side play.
- Solid rubber tires need to be cleaned periodically with a damp cloth and replaced if they become severely worn.

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EN LIMITED LIFETIME WARRANTY

This Argo® product is guaranteed to be free from defects in materials and workmanship for the life of the original user. This warranty does not extend to non-durable components which are subject to normal wear and replacement. Any alterations, abuse, misuse, or accidental damage voids this warranty.

For warranty service, you have two options:

- Return your transport chair to the Vendor where the unit was bought.
- In Canada & the U.S. call AMG's Consumer Info Line at 1-800-363-2381 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of Purchase and Serial Number (on bar underneath seat).

Our customer service team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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