# **Airgo**<sup>®</sup> *Adventure*<sup>™</sup> 8

## **EN** **Adventure 8 Rollator**

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**EN CHECK ALL PARTS**

Carefully examine all parts for shipping damage. DO NOT use the rollator if there is apparent damage or if the any parts are missing. If there are missing or damaged parts refer to the warranty on the other side of this guide.

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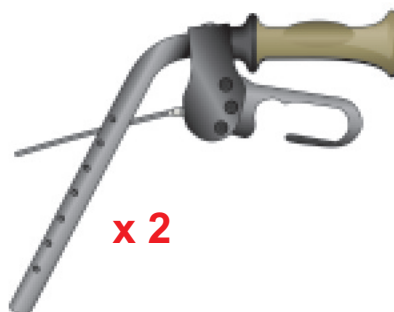
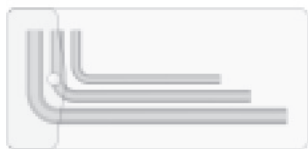
**EN** User's that are 5' 3" or shorter will need to remove the spacer tubes before inserting the wheels.

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





**3**
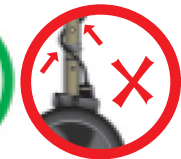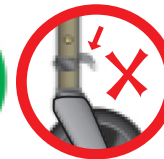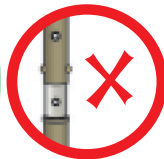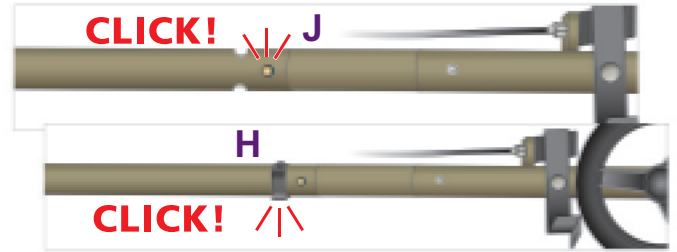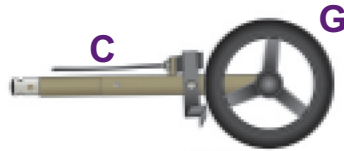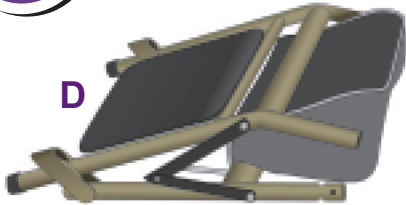
### **EN** INSTALL REAR WHEELS

- Lay the Rollator down so that the top of Seat (D) is facing up.
- Insert Rear Wheel (G) with the Brake Cable (C) facing up, into the frame until Spring Pins (J) lock into place. Brake Cable should NOT wrap around frame.
- Insert E-Clip (H) into hole in frame above the Spring Pins. Close the E-Clip arm around the frame until it snaps closed.

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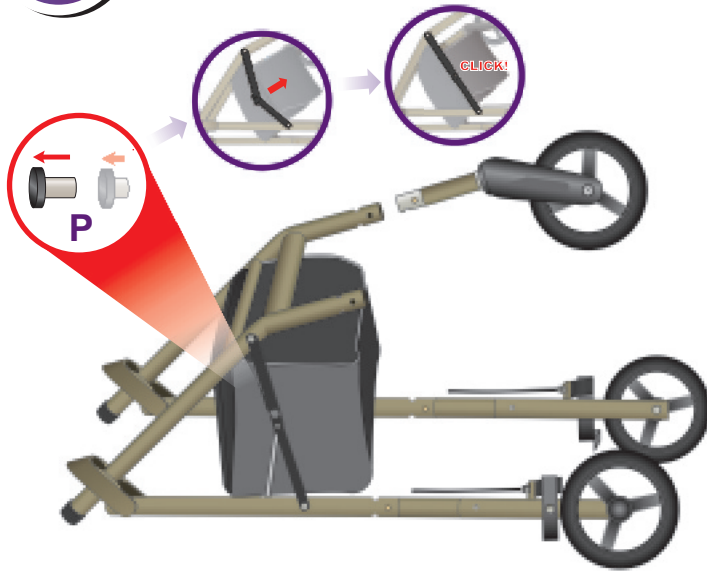
# 4

## EN INSTALL FRONT WHEELS

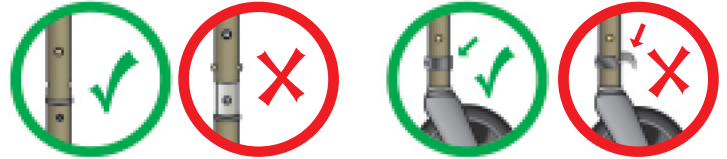
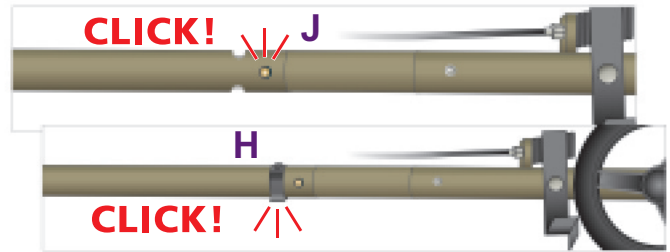
- Open the frame by pulling out the Safety Latch (P) at the center of the black Folding Bracket (L), located at the right side of the Rollator. You will hear a click when the Folding Brackets are fully extended.
- Insert Front Wheel into frame until Spring Pins lock into place.
- Insert E-Clip into hole in frame above the Spring Pins. Close the E-Clip arm around the frame until it snaps closed.

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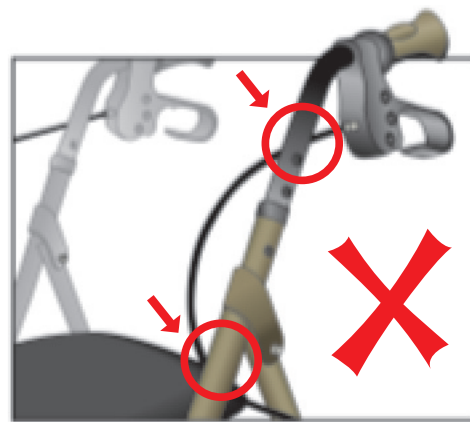
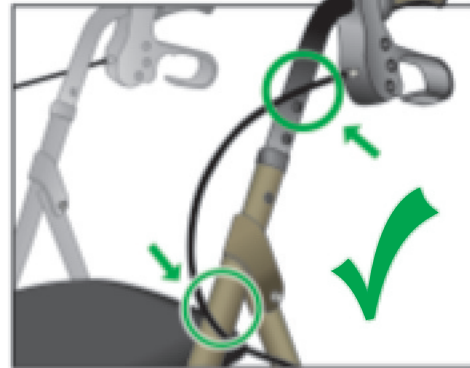
#### EN INSTALL THE HANDLEBARS

Stand the Rollator upright. Push the Seat down into place. Insert the Handlebars (A) so that the Handgrips align with the Rear Wheels. Brake Cables should be on the outside of the Handlebars and Frame.

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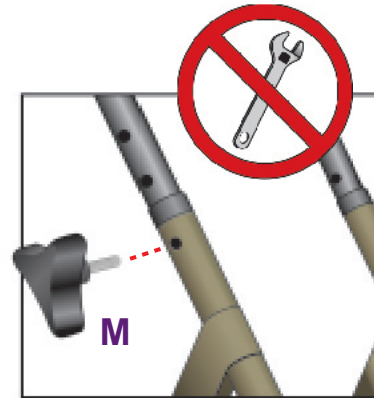
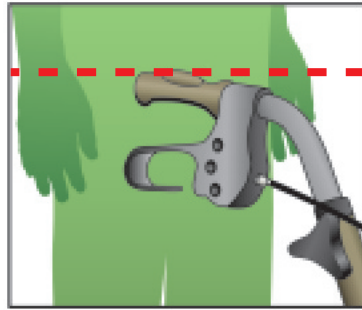
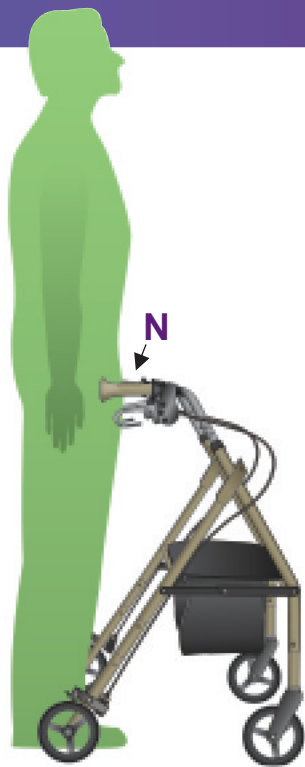
#### EN ADJUST THE HANDLEBARS

Stand upright behind the Rollator, shoulders relaxed, arms hanging at your side, and eyes looking straight ahead. In this position, adjust the Handgrips (N) of the Rollator to a height even with the wrists. Secure the Handlebars at the closest hole to the desired height. Insert and tighten the triangular Hand Screws (M) until secure. **HAND TIGHTEN ONLY.**

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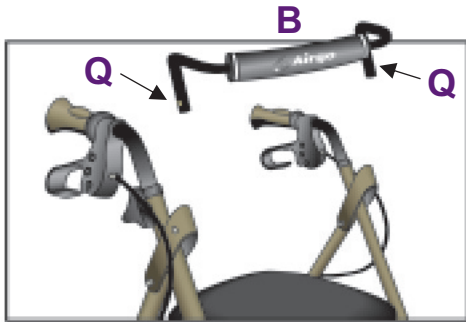
# 7

## EN INSTALL BACKREST

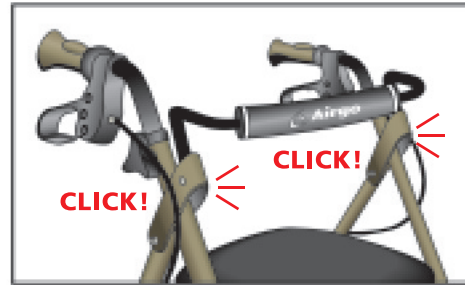
Push in both Push-Buttons (Q) and insert the Backrest (B) into both sides of the Rollator simultaneously (as shown) until it locks in place.

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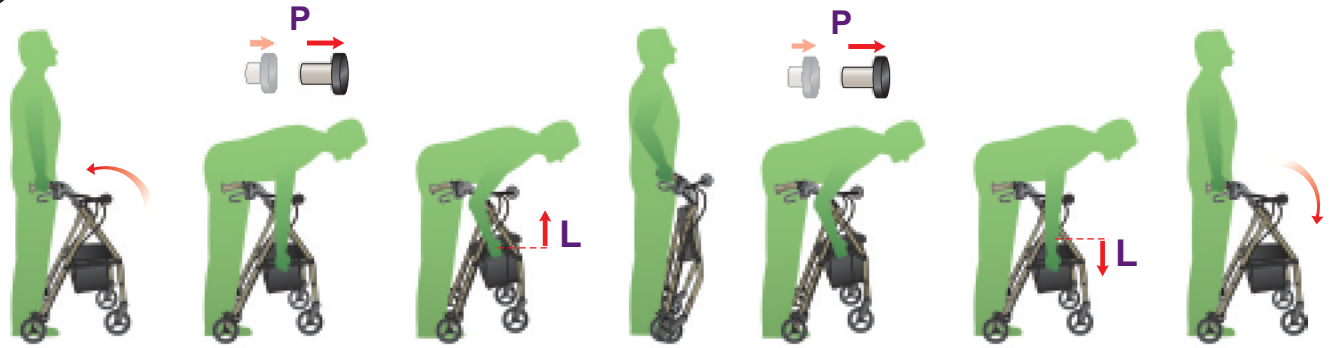
## EN FOLDING / UNFOLDING

Before folding or unfolding the Airgo® Comfort-Plus Rollator, lock the Brakes by pushing down on the Brake Handles until you feel or hear the Brakes click into locked position.

Folding the Airgo® Comfort-Plus™ rollator

1. Lean the rollator backwards. Pull the Safety Latch (P) located at the centre of the black folding bracket (L) on the right side of the rollator, towards you to initiate folding.
2. The position of the casters must be aligned properly to allow the rollator to fold completely and lock.

Unfolding the Airgo® Comfort-Plus™ rollator

1. Pull the Safety Latch (P), located at the center of the black folding bracket (L) on the right side of the rollator, outwards to disengage the folding mechanism.
2. Unfold the walker and press down on the seat with one hand to ensure it is fully opened.

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## EN LIMITED LIFETIME WARRANTY

AMG Medical Inc. covers this Airgo® rollator by a Lifetime Warranty on the Frame and a Five (5) Year Warranty on Brake Handles against defects in materials and workmanship. All other parts and components are warranted for a period of 90 days. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.

For warranty service, you have two options:

1. Return your rollator to the Vendor where the unit was bought.
2. In Canada & the U.S. call AMG's Consumer Info Line at 1-800-363-2381 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of Purchase, Serial Number (on bar underneath seat)

Once you have a return authorization number, the unit should be shipped prepaid to: AMG Medical Inc., 8505 Dalton, Montreal, QC, H4T 1V5.

Include a copy of the original bill of sale and a money order of \$10.00 to cover cost of insurance and return transportation.

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EN

## READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR ROLLATOR:

1. Consult your doctor, therapist or other qualified professional to determine if the Airgo® rollator will suit your particular needs.
2. **IMPORTANT** - Before using, make sure that all parts are secure and the Airgo® rollator frame is fully opened and in locked position.
3. For proper positioning and use of your Rollator, imagine a line drawn across the back of the Rear Wheels. Part of one foot should always be slightly ahead of that line as you walk.
4. To slow down, squeeze the Brake Handles up. To lock the Brakes, push down on the Brake Handles until the Brakes click into locked position. If the Rear Wheels of the Rollator move while the Brakes are in locked position, DO NOT use the Rollator. Please call our toll-free number (1-800-363-2381) for return or repair. To release the Brakes from locked position, pull up on the Brake Handles.
5. **IMPORTANT** - Before using the Seat portion of your Rollator, ALWAYS lock the brakes FIRST. DO NOT navigate the Rollator while sitting on the seat. The brakes should ALWAYS be locked when the seat is being used. Risk of fall and serious injury may result if Brakes are not locked while the user is sitting on the Seat. Falls are possible if the user leans or slides backward on the Seat without first installing the backrest. NEVER attempt to sit on the Seat when the Rollator is on an incline. Serious risk of falls and injury may result.  
**DO NOT push the Rollator while someone is sitting in the Seat. Damage to the Rollator frame or wheels, and serious injury to the user, could result from improper use. CAUTION - DO NOT exceed the following weight limits, or serious injury could result:**  
**Seat: 300 lbs.**  
**The Seat is not intended to transport people or objects. DO NOT use the Seat to carry or move anything.**
6. Get assistance if you are uncomfortable folding or unfolding your Airgo® rollator.
7. Your Airgo® rollator should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.

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