# Airgo<sup>®</sup>

## duo<sup>™</sup> Rollator+Transport

Patents Pending

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**EN CHECK ALL PARTS**

Carefully examine all parts for shipping damage. DO NOT use the rollator if there is apparent damage or if any parts are missing. If there are missing or damaged parts refer to the warranty on the other side of this guide.

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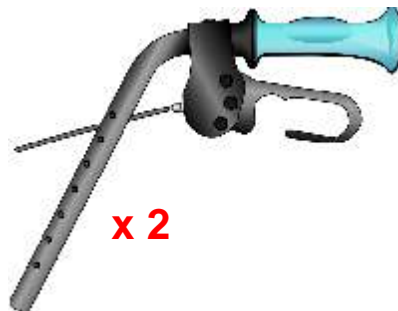
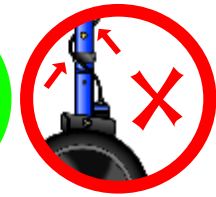
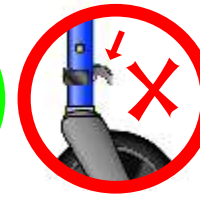
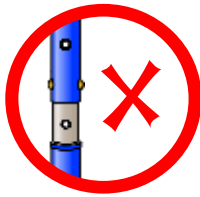
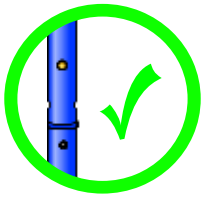
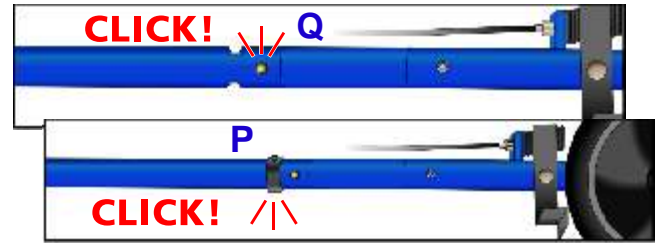
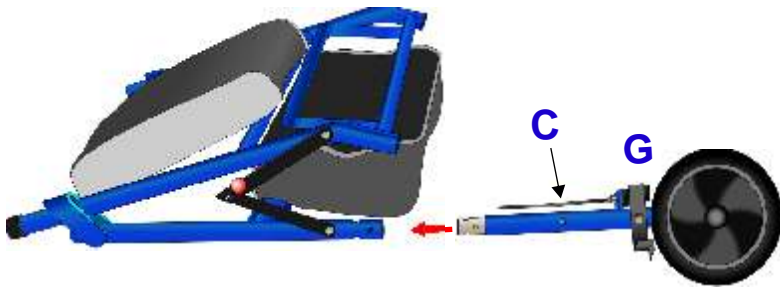
### EN INSTALL REAR WHEELS

- Lay the Rollator down so that the top of Seat (D) is facing up.
- Insert Rear Wheel (G) with the Brake Cable (C) facing up, into the frame until Spring Pins (Q) lock into place. Brake Cable should NOT wrap around frame.
- Insert E-Clip (P) into hole in frame above the Spring Pins. Close the E-Clip arm around the frame until it snaps closed.

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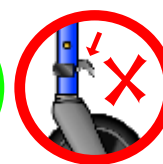
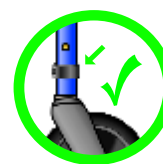
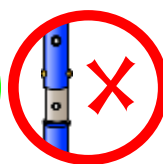
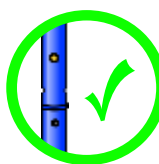
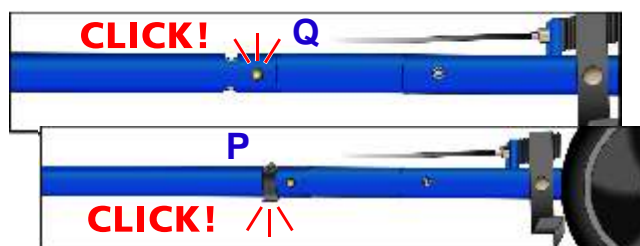
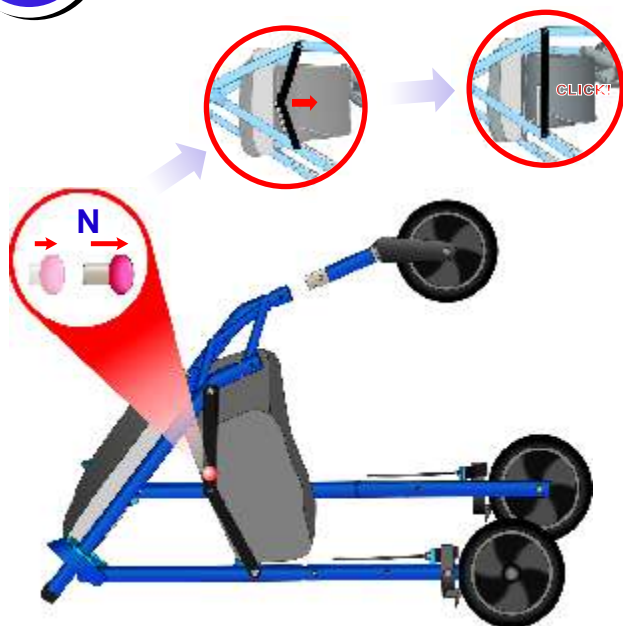
#### EN INSTALL FRONT WHEELS

- Open the frame by pulling out the Safety Latch (N) at the center of the black Folding Bracket (K), located at the right side of the Rollator. You will hear a click when the Folding Brackets are fully extended.
- Insert Front Wheel into frame until Spring Pins lock into place.
- Insert E-Clip into hole in frame above the Spring Pins. Close the E-Clip arm around the frame until it snaps closed.

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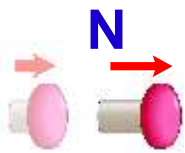
### EN OPEN ROLLATOR FRAME

- Ensure the folding mechanism has been disengaged, by pulling the Safety Latch (N) outwards, and all 4 wheels are installed securely before attempting to completely unfold the rollator.
- Unfold the walker and press down on the folding brackets (K) to ensure it is fully opened. The safety latch (N) will click into place when the folding brackets are completely unfolded.

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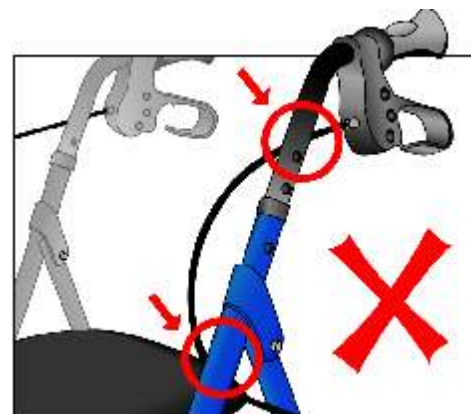
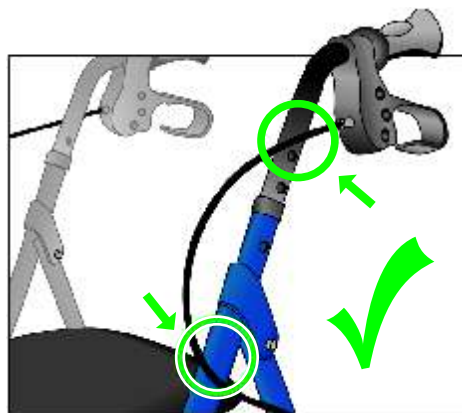
**6****EN INSTALL THE HANDLEBARS**

Insert the Handlebars (A) so that the Handgrips align with the Rear Wheels. Brake Cables should be on the outside of the Handlebars and Frame.

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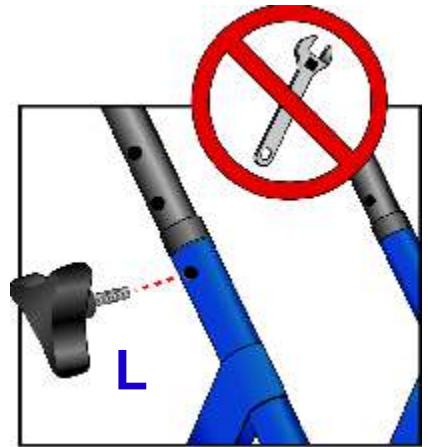
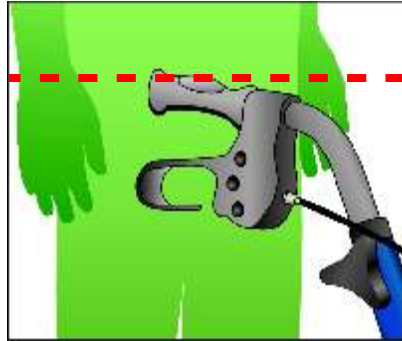
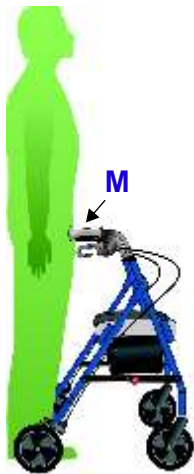
### EN ADJUST THE HANDLEBARS

Stand upright behind the Rollator, shoulders relaxed, arms hanging at your side, and eyes looking straight ahead. In this position, adjust the Handgrips (M) of the Rollator to a height even with the wrists. Secure the Handlebars at the closest hole to the desired height. Insert and tighten the triangular Hand Screws (L) until secure. **HAND TIGHTEN ONLY.**

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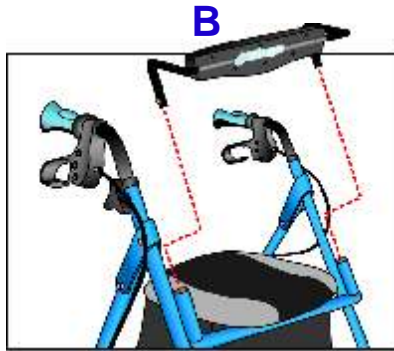
### EN INSTALL BACKREST

Insert the Backrest (B) so that the padded portion faces toward the handles. Depress the Push-Buttons (O) and insert the Backrest into both of the front backrest brackets simultaneously. Lock the backrest in the desired height setting.

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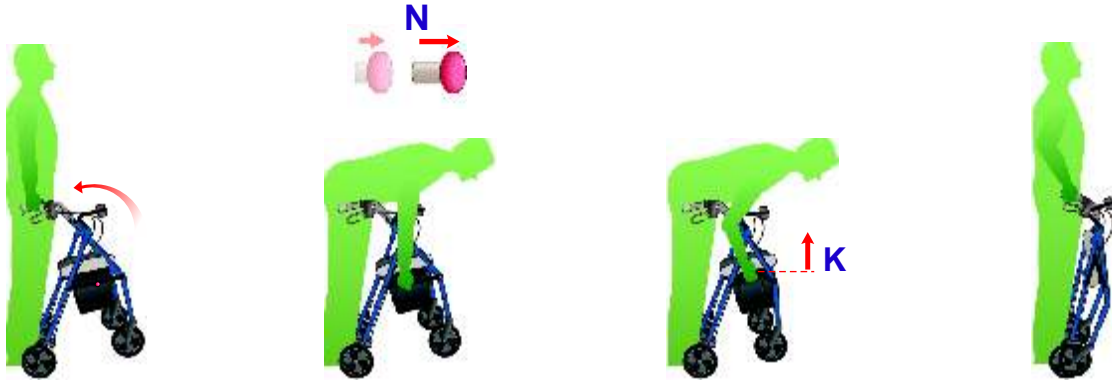
### EN FOLDING / UNFOLDING

Before folding or unfolding the Airgo® Comfort-Plus Rollator, lock the Brakes by pushing down on the Brake Handles until you feel or hear the Brakes click into locked position.

Folding the Airgo® Comfort-Plus™ rollator

1. Lean the rollator backwards. Pull the Safety Latch (N) located at the centre of the black folding bracket (K) on the right side of the rollator, towards you to initiate folding.
2. The position of the casters must be aligned properly to allow the rollator to fold completely and lock.

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### EN OCCASIONAL USE TRANSPORT

Remove the backrest from its front (rollator) position, reverse it and place it into brackets on the rear sides of the rollator, ensuring the buttons click into brackets holes. Undo the Velcro strap from the footrest folding bar, sit the user and then flip-down the footrest folding bar. Ensure that the footrest lower side is fully open. The footrest bar can be adjusted to accommodate taller users. Do not step on the footrest bar when getting on or off the seat.

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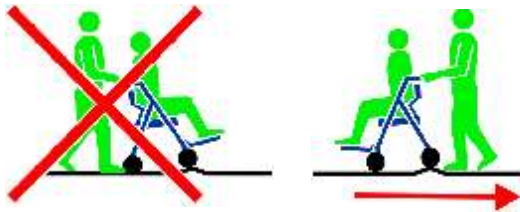
**EN CAUTION**

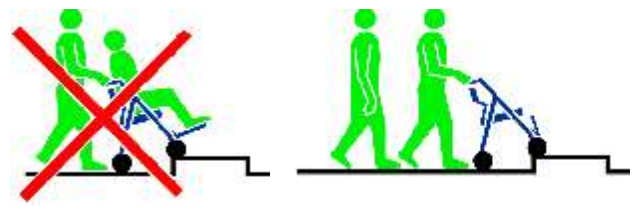
When pushing someone in the rollator, do not roll across small uneven surfaces or use on inclines, doing so can cause the rollator to suddenly tip forward! To proceed safely, turn the rollator backward. To safely cross curbs, uneven surfaces or use on large inclines, the person sitting on the rollator must get out of the rollator.

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## EN LIMITED LIFETIME WARRANTY

AMG Medical Inc. covers this Airgo® rollator by a Lifetime Warranty on the Frame and a Five (5) Year Warranty on Brake Handles against defects in materials and workmanship. All other parts and components are warranted for a period of 90 days. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.

For warranty service, you have two options:

1. Return your rollator to the Vendor where the unit was bought.
2. In Canada & the U.S. call AMG's Consumer Info Line at 1-800-363-2381 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of Purchase, Serial Number (on bar underneath seat)

Once you have a return authorization number, the unit should be shipped prepaid to: AMG Medical Inc., 8505 Dalton, Montreal, QC, H4T 1V5.

Include a copy of the original bill of sale and a money order of \$10.00 to cover cost of insurance and return transportation.

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EN

## READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR ROLLATOR:

1. Consult your doctor, therapist or other qualified professional to determine if this Airgo® rollator will suit your particular needs.
2. **IMPORTANT** - Before using, make sure that all parts are secure and the Airgo® rollator frame is fully opened and in locked position.
3. For proper positioning and use of your Rollator, imagine a line drawn across the back of the Rear Wheels. Part of one foot should always be slightly ahead of that line as you walk.
4. To slow down, squeeze the Brake Handles up. To lock the Brakes, push down on the Brake Handles until the Brakes click into locked position. If the Rear Wheels of the Rollator move while the Brakes are in locked position, **DO NOT** use the Rollator. Please call our toll-free number (1-800-363-2381) for return or repair. To release the Brakes from locked position, pull up on the Brake Handles.
5. **IMPORTANT: WHEN USING THE DUO INDIVIDUALLY AS A ROLLATOR** - Before using the Seat portion of your Rollator, **ALWAYS** lock the brakes **FIRST**. **DO NOT** navigate the Rollator while sitting on the seat. The brakes should **ALWAYS** be locked when the seat is being used. Risk of fall and serious injury may result if Brakes are not locked while the user is sitting on the Seat. Falls are possible if the user leans or slides backward on the Seat without first installing the backrest. **NEVER** attempt to sit on the Seat when the Rollator is on an incline. Serious risk of falls and injury may result.  
CAUTION - **DO NOT** exceed the following weight limits, or serious injury could result:  
Rollator: 250 lbs.  
Transport Seat: 195 lbs.
6. Get assistance if you are uncomfortable folding or unfolding your Airgo® rollator.
7. Your Airgo® rollator should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.

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