Walker

Easy-Fold Lightweight Walker
Marchette légère facile à plier
Andador ligero fácil de plegar

**PLEASE DO NOT
RETURN THIS
PRODUCT TO THE STORE.**

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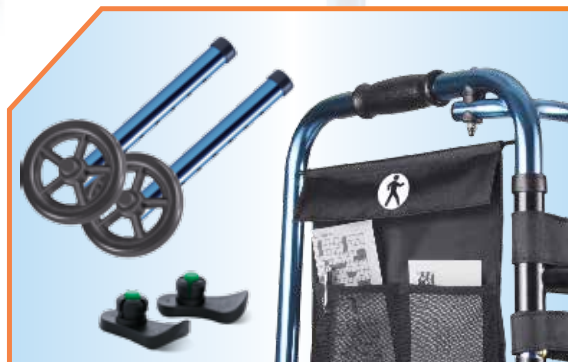
If you're experiencing problems or have a question, contact our customer service team at 1-800-363-2381. We're pleased to help you!

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BONUS: includes 5" wheels, glide tips and deluxe accessory bags!

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EN

Please register your Hugo at hugoanywhere.com to activate your warranty and stay up to date with the latest notices.

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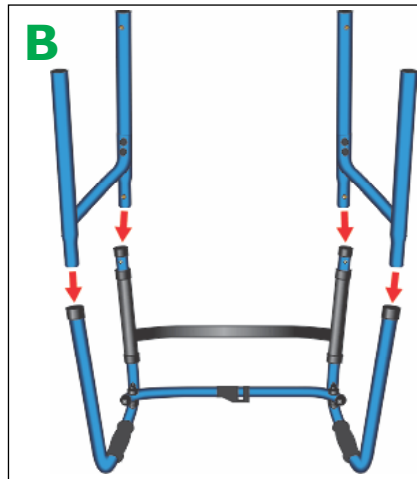
ASSEMBLE FRAME ASSEMBLER LE CHÂSSIS MONTAJE DEL MARCO


EN

Open the Upper Frame, ensuring that the push-button (A) clicks into place. Turn the walker upside down. Insert the Side Frames (B) into the leg openings until the push-buttons click into place to indicate that the assembly is secure.

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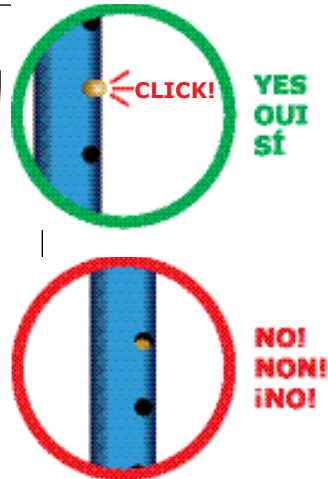More stable
Plus stable
Más estable

EN

If you cannot support your weight through your legs, then you need a more stable option with four rubber tips.

If you need a walker due to fatigue and decreased endurance, use the more maneuverable option of wheels with skis.

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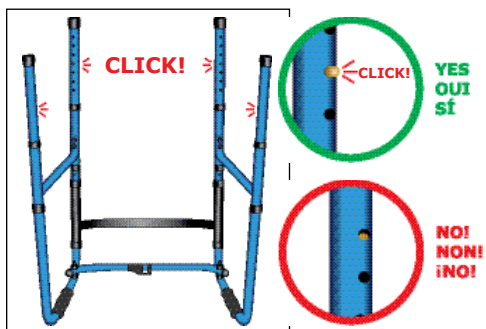
EN

Insert all four bottom legs into the side frames until the push-buttons click into place. Ensure that each leg is adjusted to the same height.

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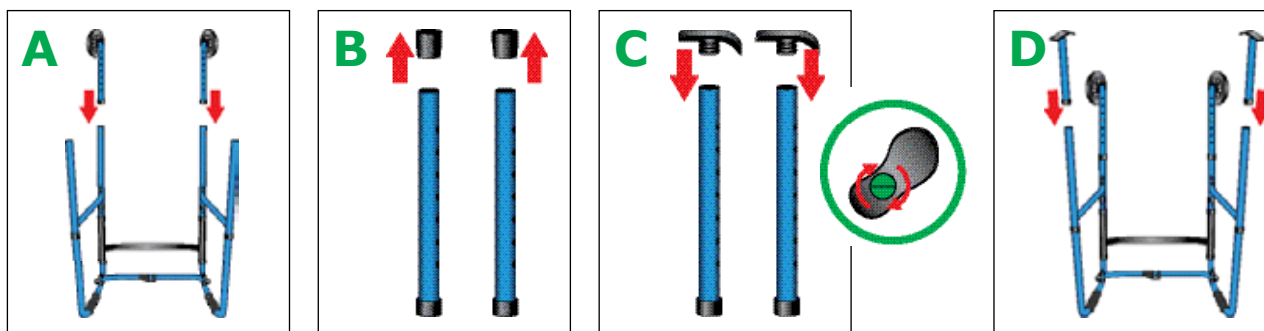
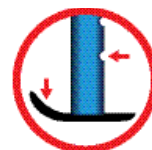
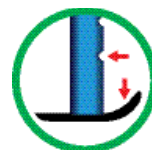
EN

Insert the wheels onto the FRONT of the frame until they lock into place (A). The wheels must be installed on the outside of the frame.

Install the 2 REAR legs and remove the rubber tips (B). Insert the glides in the rear leg tubings (C), with the front of the glides in line with the adjustment holes (D). Hand tighten the green screw.

Ensure that all glides and wheels contact the floor evenly.

DO NOT install glides on all four legs or wheels on all four legs at the same time.

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**EN**

Place the walker in front of you and let your arms hang down at your sides. When properly adjusted, the walker's handgrips should align with your wrists.

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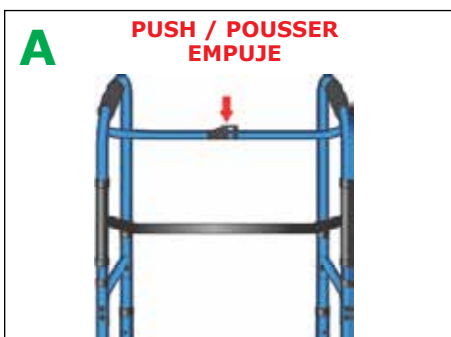
**EN**

Do not rely on the walker for stability while folding or unfolding.

TO FOLD: Press and hold the one-touch push-button while using your opposite hand to fold each side of the walker toward the center of the frame.

TO UNFOLD: Simply grab both hand grips and open the frame until the one-touch push-button clicks into place.

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READ THESE PRECAUTIONS PRIOR TO USING

EN

1. Consult your doctor, therapist or other qualified professional to determine if this mobility aid will suit your particular needs.
2. Examine all parts for shipping damage. In case of shipping damage, or if anything is missing, do not use. Call our toll-free number: 1-866-518-6099 (Canada & USA only).
3. IMPORTANT - Before each use, make sure that all parts are secure and that the frame is fully opened. Do not use the walker while folded.
4. Maximum weight capacity: 300 lb (136 kg).
5. The Hugo® Folding Walker provides support and stability while walking. It is not intended to support the full weight of the user.
6. Always use caution when walking on wet or icy surfaces.
7. Replace any broken, damaged or worn items immediately.
8. Consult a qualified dealer or repair center for periodic inspections and repairs.
9. To clean, wipe frame using a clean soft cloth. Do not use abrasive materials. Clean wheels with warm water and mild cleanser. Dry with a clean cloth.

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LIMITED WARRANTY GARANTIE LIMITÉE GARANTÍA LIMITADA

EN

AMG Medical Inc. warrants the frame of this Hugo® mobility aid to be free from defects in materials and workmanship for three (3) years. All other parts and components are warranted for a period of one (1) year. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.

For warranty service, you have two options:

1. Return your walker to the Vendor where the unit was bought.
2. In Canada & the U.S. call AMG's Consumer Info Line at 1-866-518-6099 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of Purchase and Serial Number (on front crossbar underneath Hugo® logo).

Our Customer Service Team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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Sidekick™ Side-Folding Rolling Walker with a Seat

- Ultra-lightweight & compact.
- Stands when folded.

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- Blueberry
Bleuet
Arándano
- Tangerine
Mandarina

Cane with QuadPod™ Ultra-Stable Cane Tip

- Stands on its own!
- Ergonomic handle for superior comfort and increased safety.

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Snow Paws™

- For safer walking on ice and snow! Fits any type of footwear, including heels.
- Carry pouch included.

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Universal Cup Holder

- Helps you stay hydrated wherever you go!
- Fits your favorite mug or a sports bottle.

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EN

At Hugo, our passion is to create mobility aids that improve your life. We strive to understand your needs, to develop safe, high quality products that exceed your expectations, and deliver the best possible value.

Each Hugo is designed using the Mom Test™ where we ask ourselves "Would I feel safe letting *my* mom use this product?". The answer has to be "Yes"!

This means carefully crafting every Hugo to be easy to assemble, and testing them to exacting safety standards for stability, durability, maneuverability, and weight capacity.

...and the environment too!

We use only lead-free paint and compact, recyclable packaging to reduce waste and minimize fuel used in transport.

Hugo, the freedom, mobility and confidence to stay active and to go anywhere, anytime.

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M. Devices Group / E. C. REP Ltd,