eurowalker

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PLEASE DO NOT
RETURN THIS
PRODUCT TO THE STORE.

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If you're experiencing problems or have a question, contact our customer service team at 1-800-363-2381. We're pleased to help you!

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EN

Please register your Hugo online at hugoanywhere.com to activate your warranty and stay up to date with the latest notices.

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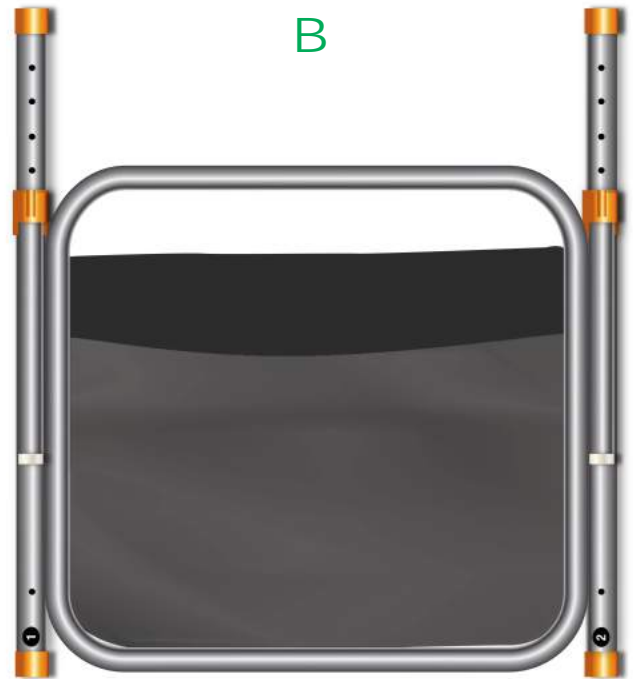
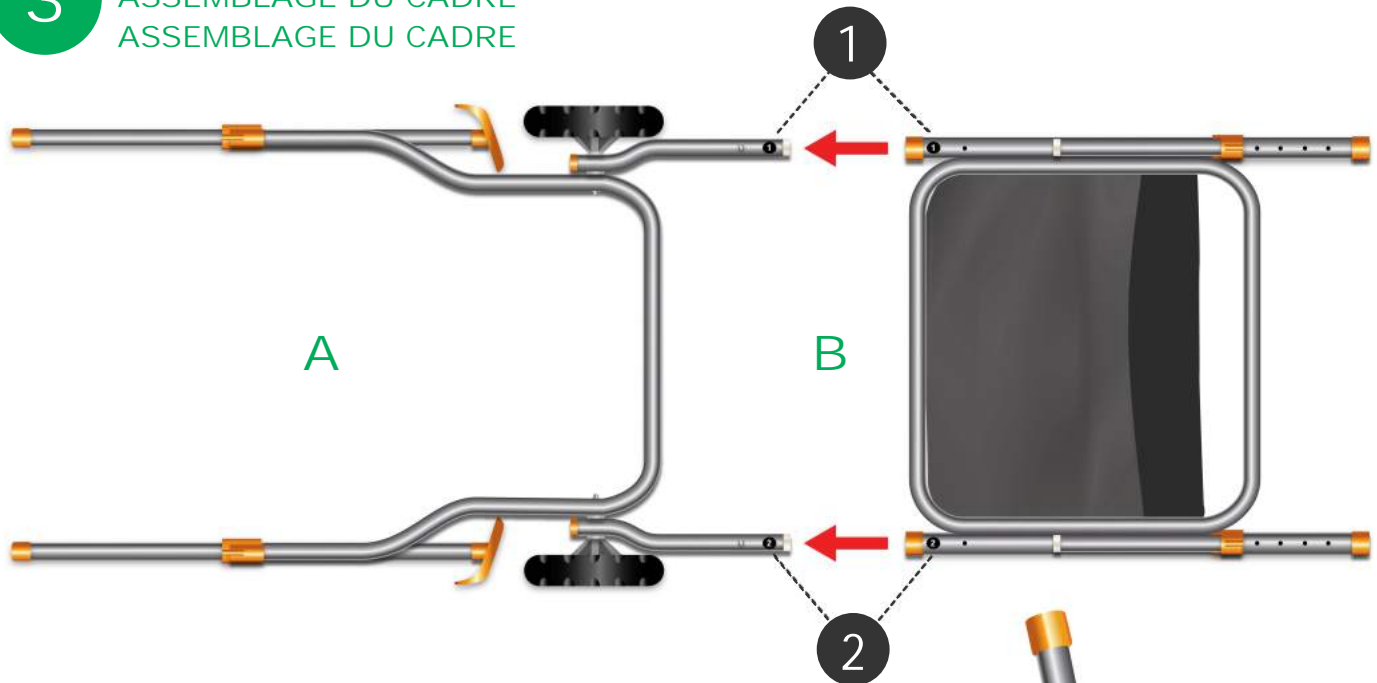
EN

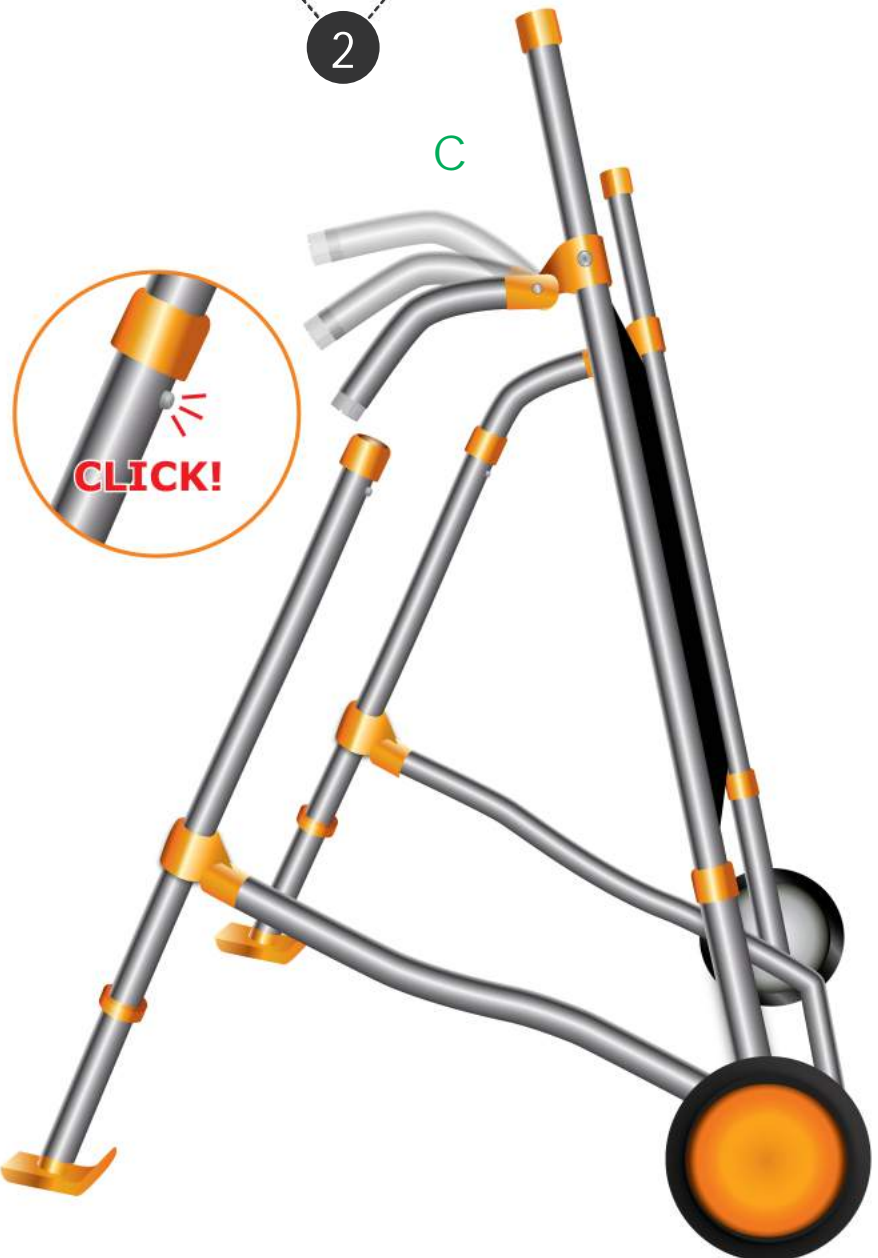
- A. Place lower frame on floor.
- B. Slide upper frame into lower frame, matching number labels **1 2** on tubing. Ensure push buttons click into place.
- C. Insert legs of lower frame into upper frame, noting that the skis must be facing forwards. Ensure push buttons click into place.

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EN

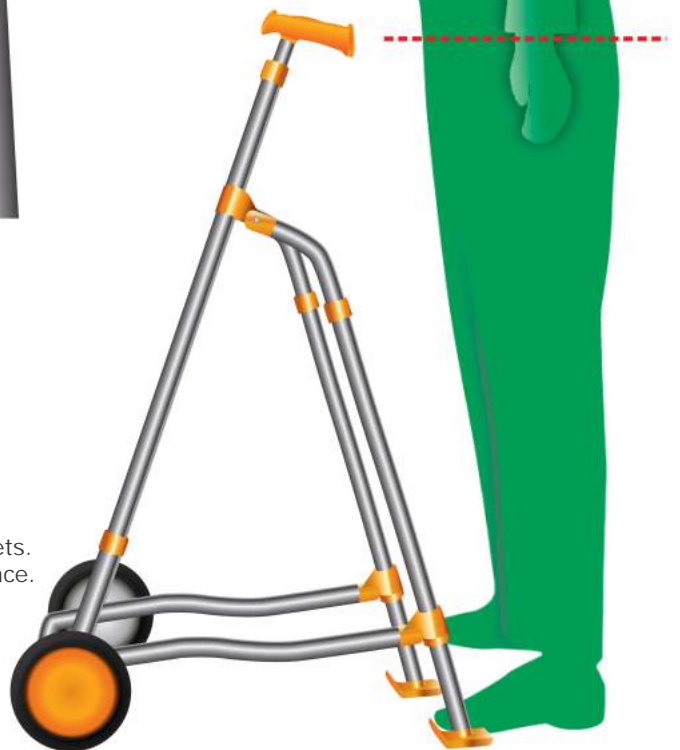
Insert handles by matching the letter labels **L** **R** on the tubing. With your arms hanging at your sides, adjust the handlegrips to align with your wrists. Ensure push buttons click into place.

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EN

TO FOLD: Simply pull up on the lower frame.

TO UNFOLD: Push back down on the lower frame, ensuring it extends all the way to the plastic stopper. Failure to fully extend will compromise stability of the walker, and might lead to a fall.

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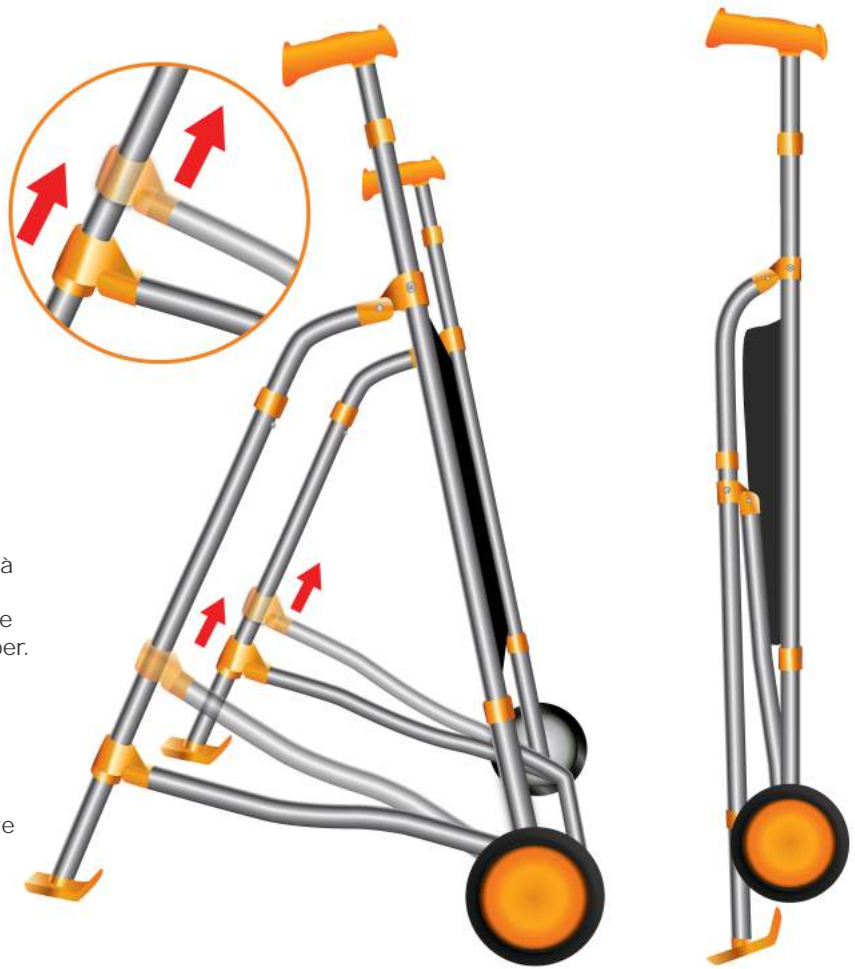
READ THESE PRECAUTIONS PRIOR TO USING

EN

1. Consult your doctor, therapist or other qualified professional to determine if this mobility aid will suit your particular needs.
2. Examine all parts for shipping damage. In case of shipping damage, or if anything is missing, do not use. Call our toll-free number: 1-866-518-6099 (Canada & USA only).
3. Before each use, ensure that all parts are secure and that the frame is fully opened. Do not use the walker while folded. Doing so might lead to a fall causing injury.
4. To clean, wipe with a damp cloth. Do not use abrasive materials. Dry immediately.
5. Maximum weight capacity is 300 lb (136 kg).

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WALKING TIPS

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EN

1. Perform simple stretches before walking. This will help you walk further and avoid injury.
2. Bring a bottle of water to stay hydrated.
3. Wear walking shoes that are well cushioned and snug in the heel, yet allow room to wiggle your toes.

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LIMITED WARRANTY GARANTIE LIMITÉE GARANTÍA LIMITADA

EN

AMG Medical Inc. covers this Hugo mobility aid by a three year warranty on the frame and a one year warranty on parts, against defects in materials and workmanship. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part.

For warranty service and general inquiries in Canada and the U.S. call our customer service team at 1-800-363-2381. Provide the following information when you call: date of purchase, receipt and serial number. Our team will provide you with all necessary instructions.

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Sidekick™ Side-Folding Rolling Walker with a Seat

- Ultra-lightweight & compact.
- Stands when folded.

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- Blueberry
Bleuet
Arándano
- Tangerine
Mandarina

Cane with QuadPod™ Ultra-Stable Cane Tip

- Stands on its own!
- Ergonomic handle for superior comfort and increased safety.

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Snow Paws™

- For safer walking on ice and snow! Fits any type of footwear, including heels.
- Carry pouch included.

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Universal Cup Holder

- Helps you stay hydrated wherever you go!
- Fits your favorite mug or a sports bottle.

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WE CARE ABOUT YOU
NOUS VOUS AVONS À COEUR
NOS PREOCUPAMOS POR USTED

EN

At Hugo, our passion is to create mobility aids that improve your life. We strive to understand your needs, to develop safe, high quality products that exceed your expectations, and deliver the best possible value.

Each Hugo is designed using the Mom Test™ where we ask ourselves "Would I feel safe letting my mom use this product?". The answer has to be "Yes"!

This means carefully crafting every Hugo to be easy to assemble, and testing them to exacting safety standards for stability, durability, maneuverability, and weight capacity.

...and the environment too!

We use only lead-free paint and compact, recyclable packaging to reduce waste and minimize fuel used in transport.

Hugo, the freedom, mobility and confidence to stay active and to go anywhere, anytime.

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M. Devices Group / E. C. REP Ltd,