NAVIGATOR

Rolling Walker & Transport Chair
Ambulateur et fauteuil de transport
Silla de Transporte y Andador de Ruedas

EN For proper assembly and to ensure your safety, follow these instructions.

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EN

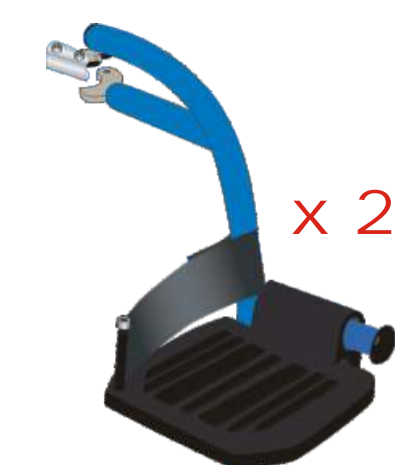
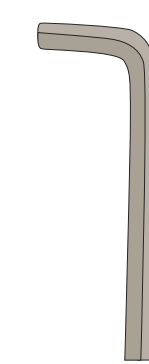
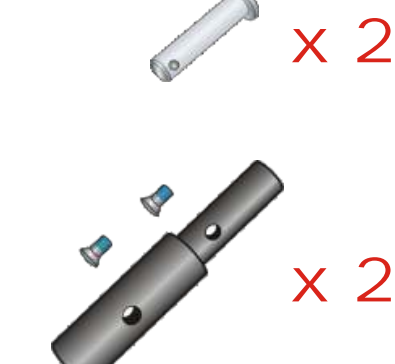
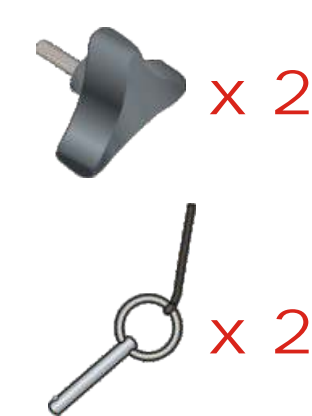
Carefully examine all parts for shipping damage. DO NOT use the Navigator™ if there is apparent damage or if any parts are missing. If there are missing or damaged parts refer to the warranty on the other side of this guide.

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SAFETY PRECAUTIONS • CONSIGNES DE SÉCURITÉ • INSTRUCCIONES DE SEGURIDAD

EN

READ THESE IMPORTANT SAFETY INSTRUCTIONS PRIOR TO OPERATING YOUR HUGO® NAVIGATOR™

- Consult your doctor, therapist or other qualified professional to determine if this Hugo® mobility aid will suit your particular needs.
- IMPORTANT** - Before using, make sure that all parts are secure and the frame is fully opened and in locked position.
- For proper positioning and use as a Rollator, imagine a line drawn between the back of the two Rear Wheels. Part of one foot should always be slightly ahead of that line as you walk.
- To slow down, squeeze the Brake Handles up. To lock the Brakes, push down on the Brake Handles until the Brakes click into locked position. If the Rear Wheels of the Navigator™ move while the Brakes are in locked position, DO NOT use the Navigator™. Please see adjustment instructions in this manual or call our toll-free number: 1-866-518-6099 (Canada & USA only) or see your distributor. To release the Brakes from locked position, pull up on the Brake Handles. Your Hugo® Navigator™ should be checked periodically to ensure the brakes are functioning properly and that all nuts and bolts are secure.
- IMPORTANT: WHEN USING THE NAVIGATOR™ INDIVIDUALLY AS A ROLLATOR** - Before using the Seat portion of your Rollator, ALWAYS lock the brakes FIRST. DO NOT navigate the Rollator while sitting on the seat. The brakes should ALWAYS be locked when the seat is being used. Risk of fall and serious injury may result if Brakes are not locked while the user is sitting on the Seat. Falls are possible if the user leans or slides backward on the Seat without first installing the backrest. NEVER attempt to sit on the Seat when the Rollator is on an incline. Serious risk of falls and injury may result. **CAUTION** - DO NOT exceed the following weight limits, or serious injury could result: Rollator: 250 lbs. Transport Seat: 250 lbs.
- Get assistance if you are uncomfortable folding or unfolding your Hugo® Navigator™.
- Store seat belt in underseat storage bag when not in use.
- Never let anyone sit in the navigator while in a moving vehicle.
- This product contains dry natural rubber.

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EN

- Allow time before walking to perform some simple stretches as this will help you walk further and avoid injury.
- Bring a bottle of water with you to stay hydrated.
- Wear walking shoes that are well cushioned and snug in the heel, yet allow room to wiggle your toes.
- When possible, walk on soft ground that is more joint-friendly such as grass or dirt paths and when walking near a road wear bright clothing that is easily visible.

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ATTENTION • ATENCIÓN

EN

PLEASE DO NOT RETURN THIS PRODUCT TO THE STORE.

If you are experiencing problems with this product, or have any question regarding its operation, contact us first at this number : 1-866-518-6099 (Canada & USA only). We will be glad to provide you with the support you need.

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LIMITED WARRANTY • GARANTIE LIMITEE • GARANTÍA LIMITADA

EN

AMG Medical Inc. warrants the frame of this Hugo® mobility aid to be free from defects in materials and workmanship for three (3) years. All other parts and components are warranted for a period of one (1) year. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.

- For warranty service, you have two options:
 - Return your Navigator™ to the Vendor where the unit was bought.
 - In Canada & the U.S. call AMG's Consumer Info Line at 1-866-518-6099 for a return authorization number.

Please ensure you provide us with the following information when you call: Date of Purchase and Serial Number (on bar underneath the seat). Our Customer Service Team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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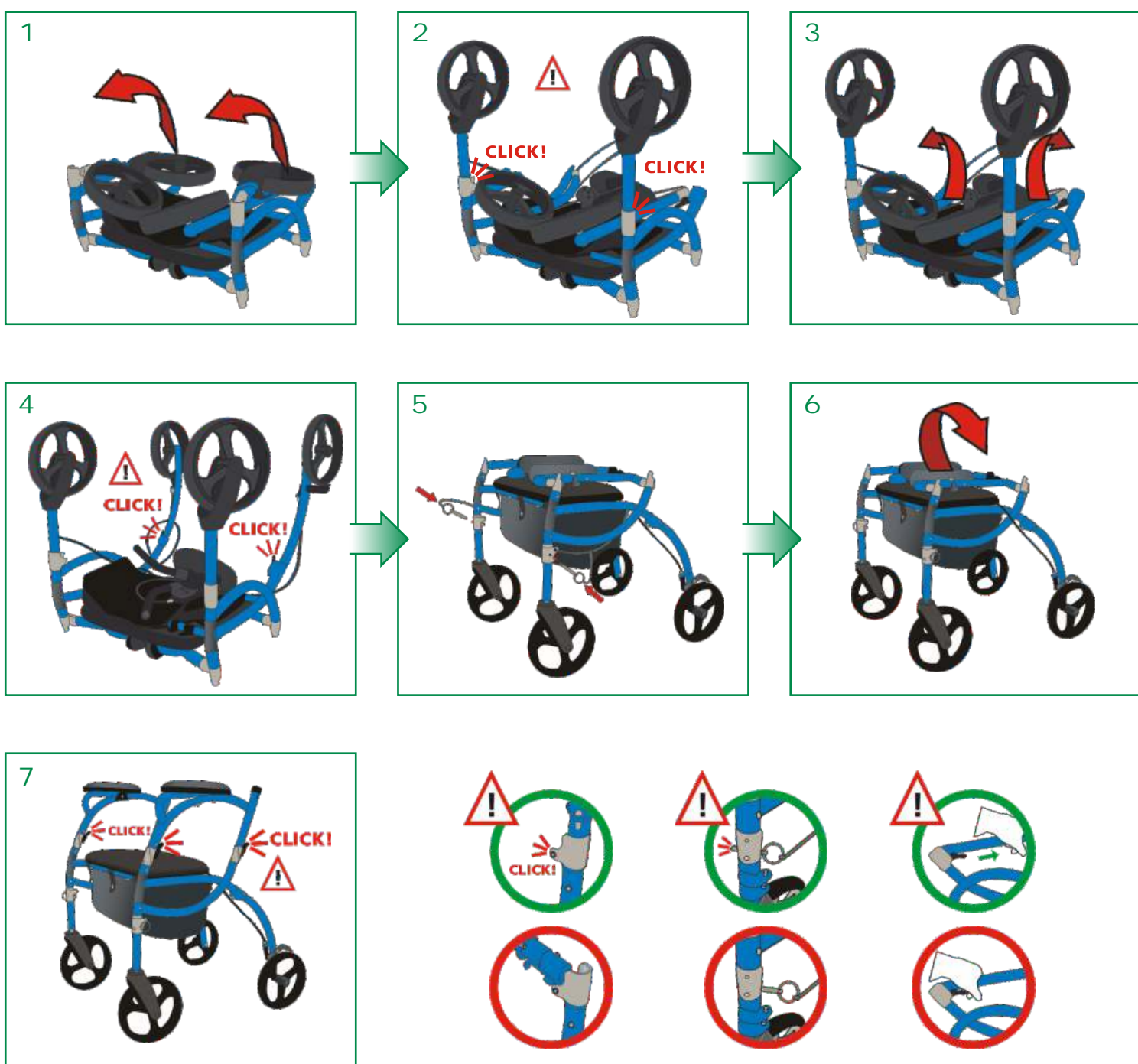
2 UNFOLDING YOUR NAVIGATOR
DÉPLIEZ VOTRE NAVIGATOR • DESPLIEGUE DE SU NAVEGADOR


EN
 Lay the Navigator™ down on its Armrests (B) so that all four wheels are facing up. Unfold the legs completely so that you hear them click into place, then stand the unit upright on all four wheels. **IMPORTANT:** Insert a Safety Pin (J) into each Front Leg by pushing it into the opening in the hinge. Raise each Armrest (B) up until you hear them click into place.

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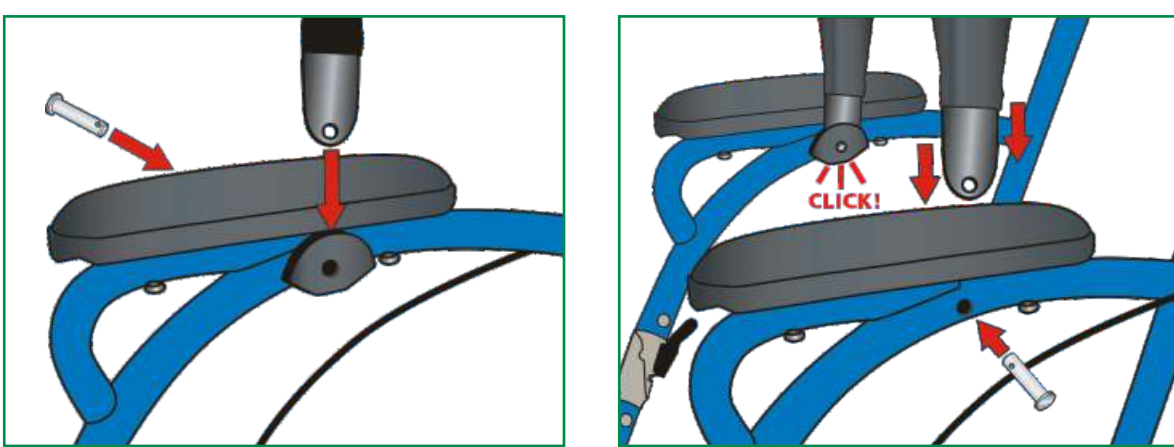3 INSTALL BACKREST • INSTALLEZ L'APPUI-DOS • INSTALACIÓN DEL RESPALDO


EN
 Slide one end of the backrest (L) into the Backrest Fan Bracket (C) located along the inside of the armrests. Insert Backrest Pin (C) into the opening on the outside of the Navigator™ frame until head is flush and repeat on the other side.

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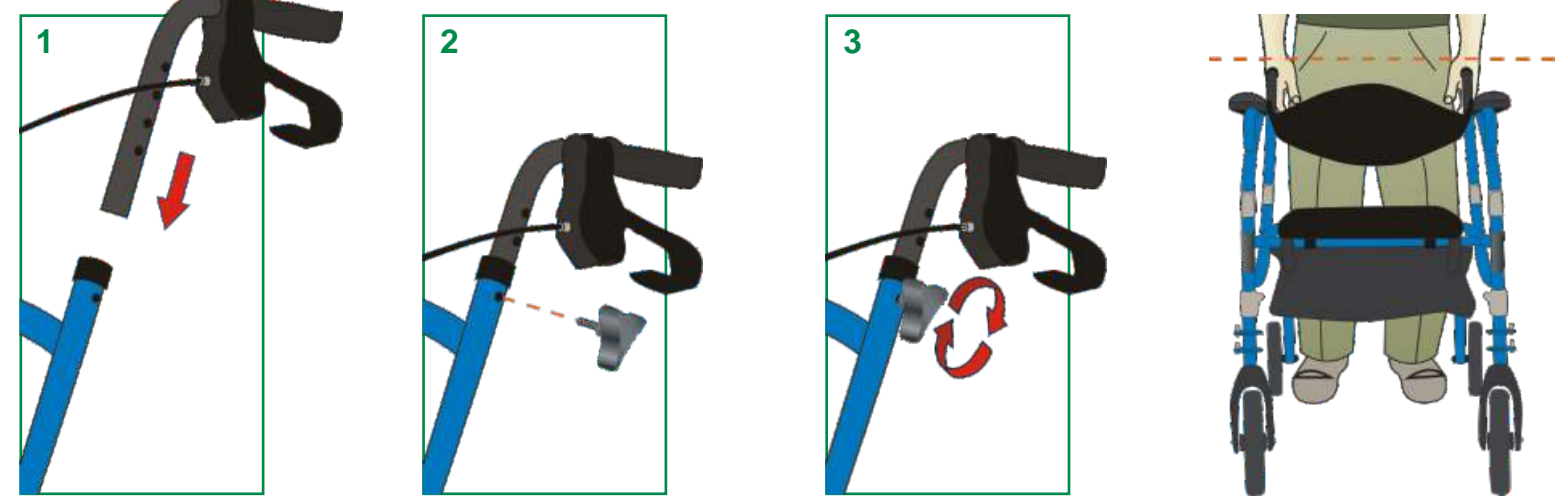
EN
 Insert the Handlebars (A) so that the Handgrips (M) align with the Rear Wheels (K). Brake Cables (P) should be on the outside of the Handlebars and Frame. To find your correct height adjustment level, stand upright behind the Navigator™, shoulders relaxed, arms hanging at your side, and eyes looking straight ahead. In this position, adjust the Handgrips (M) of the Navigator™ to a height even with your wrists. Secure the Handlebars at the closest hole to the desired height. Insert and tighten the triangular Hand Screws (O) until secure. **HAND TIGHTEN ONLY.**

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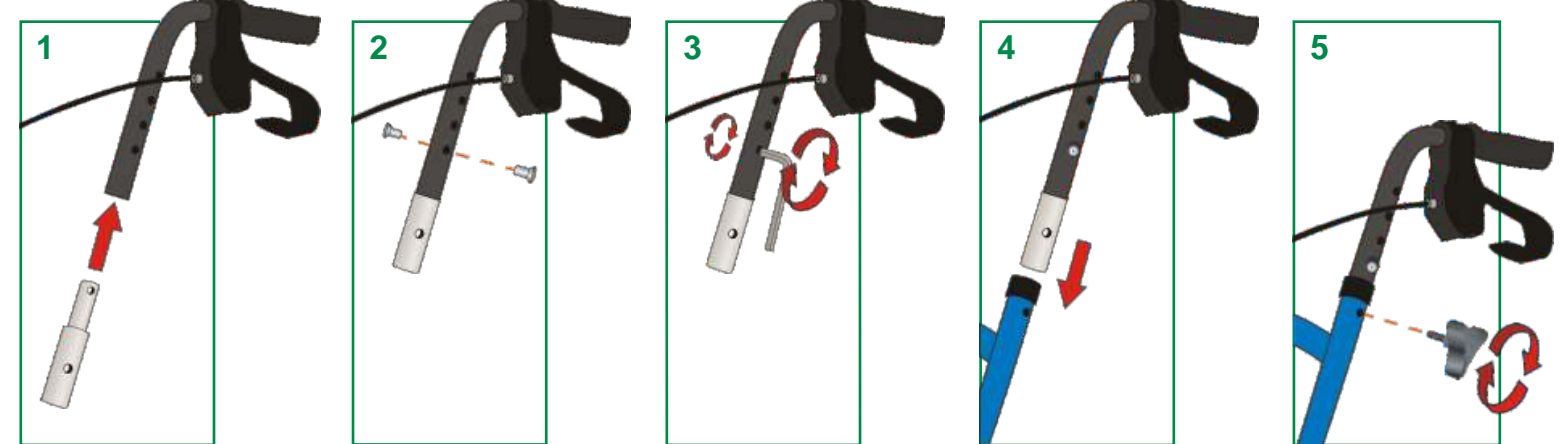
EN
 Insert the Handle Extender into the bottom opening of each handle. Using the included set of Allen Keys, tighten the Handle Extender Screws into the Handlebars until secure. Reinstall the Handlebars and secure at the closest hole to the desired height using the Triangular Hand Screws (O).

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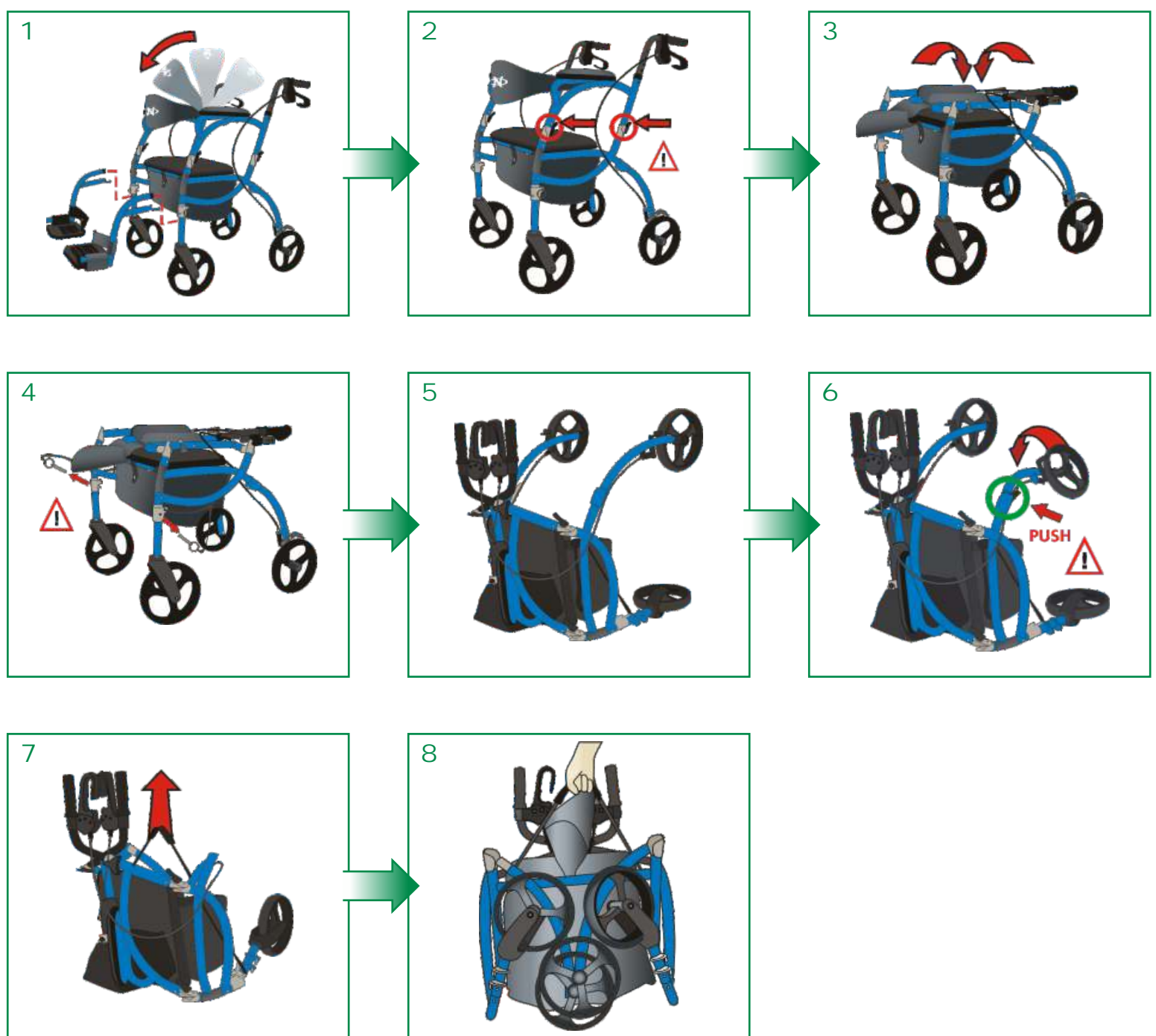5 FOLDING YOUR NAVIGATOR™
POUR PLIER VOTRE NAVIGATOR™ • CIERRE DE SU NAVEGADOR

EN
 Ensure Backrest (L) is flipped forward as shown. If installed, remove and store the Footrests in the Bag (R) under the Seat. Fold down the Armrests (B) by depressing, at the same time, the black Paddle Releases (D) located on the front and back tube of each Armrest. Remove the Safety Pins (J) from the Front Leg hinges by pulling the Safety Pin from each Front Leg hinge, then lay the unit down on the Front Legs (fig. 5).

Fold the Rear Legs by pressing the black Paddle Releases on each leg and fold inward. To fold the Front Legs, place one hand on the Handlebars and lift up the carrying strap located at the rear of the Seat.

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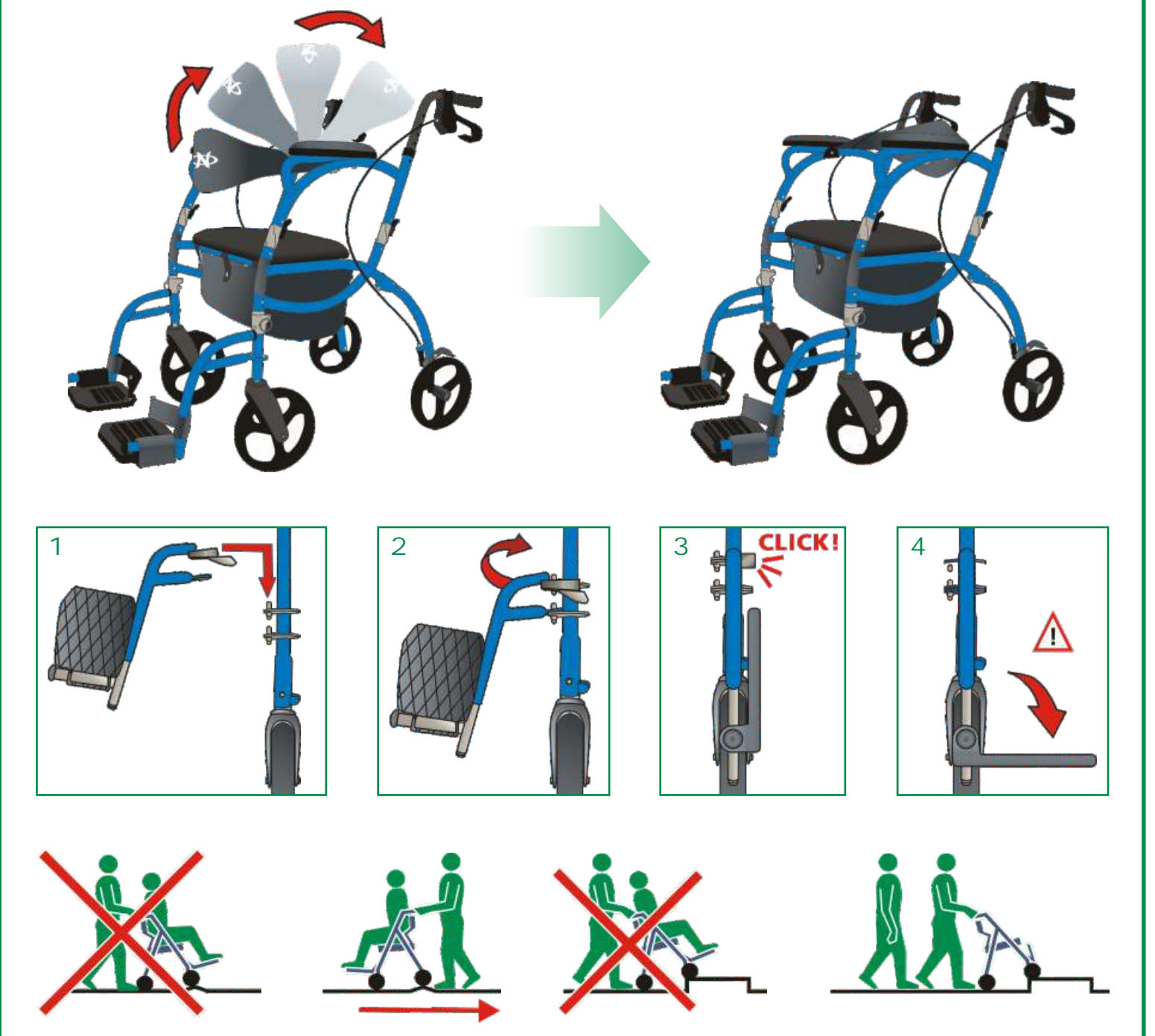
EN
 Flip the Backrest (L) toward the Handlebars (A) so that it is positioned to the back of the unit. Included are a left and right Footrest (G). Align the Footrest holes with the Alignment Pins (F) on the Front Legs of the frame and ensure both pins are engaged, then swing the Footrests towards the center of the unit. To enter or exit the Seat (E), the Footrest Plates (G) must be folded up and the Brakes must be locked. Always fasten the Seat Belt (O). To remove the Footrests, release the lock, swing outward and lift off of the Alignment Pins.

CAUTION WHEN SEATED
 Leaning out of the Navigator™ without proper assistance could cause tipping. To reduce the risk of an accident NEVER reach or lean when you are seated in your Navigator™. Instead, have someone present to help you.

CAUTION WHEN PUSHING
 When pushing someone in the Navigator™, do not roll across small uneven surfaces or use on inclines, doing so can cause the Navigator to suddenly tip forward! To proceed safely, turn the Navigator™ backward. To safely cross curbs, uneven surfaces or use on large inclines, the person sitting on the Navigator™ must get out of the seat.

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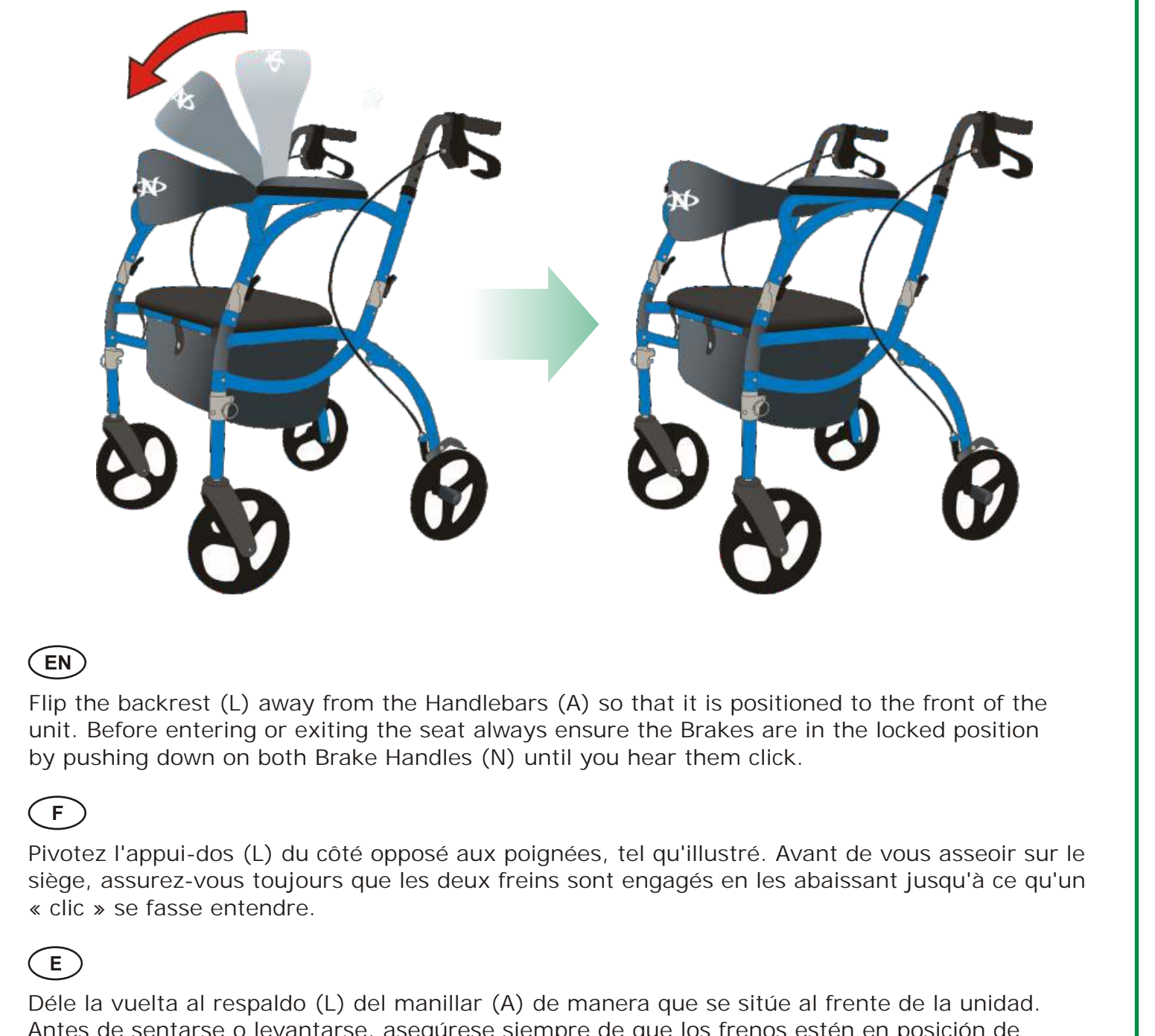
EN
 Flip the backrest (L) away from the Handlebars (A) so that it is positioned to the front of the unit. Before entering or exiting the seat always ensure the Brakes are in the locked position by pushing down on both Brake Handles (N) until you hear them click.

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EN
 Over time the Brake Cables on your Navigator™ will stretch, much like those on a bicycle, and will require adjusting. With the brakes NOT applied, hold the Adjusting Barrel (S) in place and rotate the Adjusting Nut (T) clockwise to increase braking force. To decrease braking force, rotate counter-clockwise. **IMPORTANT** – Before using the Navigator™ again, ensure the brakes are working properly. If the Rear Wheels of the Navigator™ move while the brakes are locked in position, DO NOT use the Navigator™. If you are unsure of how to adjust your brakes or feel you require further assistance please see your authorized Hugo® dealer or call our toll-free number (1-866-518-6099) for help.

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