



AquaSense®

Making your bathroom a safer place



Low-Profile Bathtub Safety Rail

Installation Instructions

Model: 785-360

Installation

1. Before installing your safety rail, make sure your bathtub is dry and dirt free. Locate the flattest section of the bathtub wall. Avoid curved areas.
2. Pull adjustment handle (A) out, to use as a lever. Turn lever counter-clockwise until clamping mechanism (B) is wide enough to fit bathtub wall.
3. Place safety rail onto bathtub wall with adjustment handle facing the outside of the tub, and clamping mechanism facing the inside.
4. Turn adjustment handle, clockwise, to tighten. With safety rail firmly in place, fold down handle until it locks into place.
5. Test mounting of unit by pulling rail forward, back and side to side. If rail moves, replace at desired position and repeat steps.

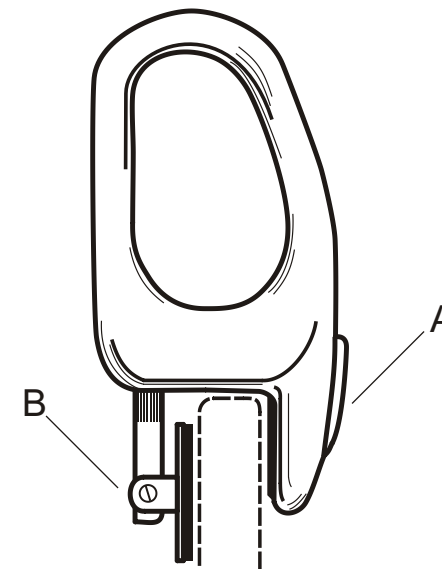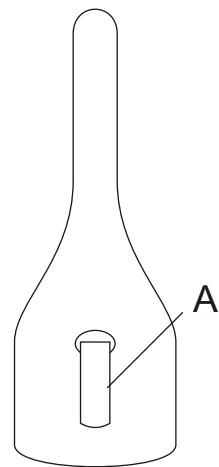

Important

Unit may require periodic retightening. Check before use and tighten if necessary.

Caution

Use extreme care when installing rail on a fiberglass tub. Do not over tighten as the clamp may crack the tub.