


700-622 AMG WHEELCHAIR 18" WIDE DETACHABLE FULL ARMS AND SWING-AWAY FOOTRESTS

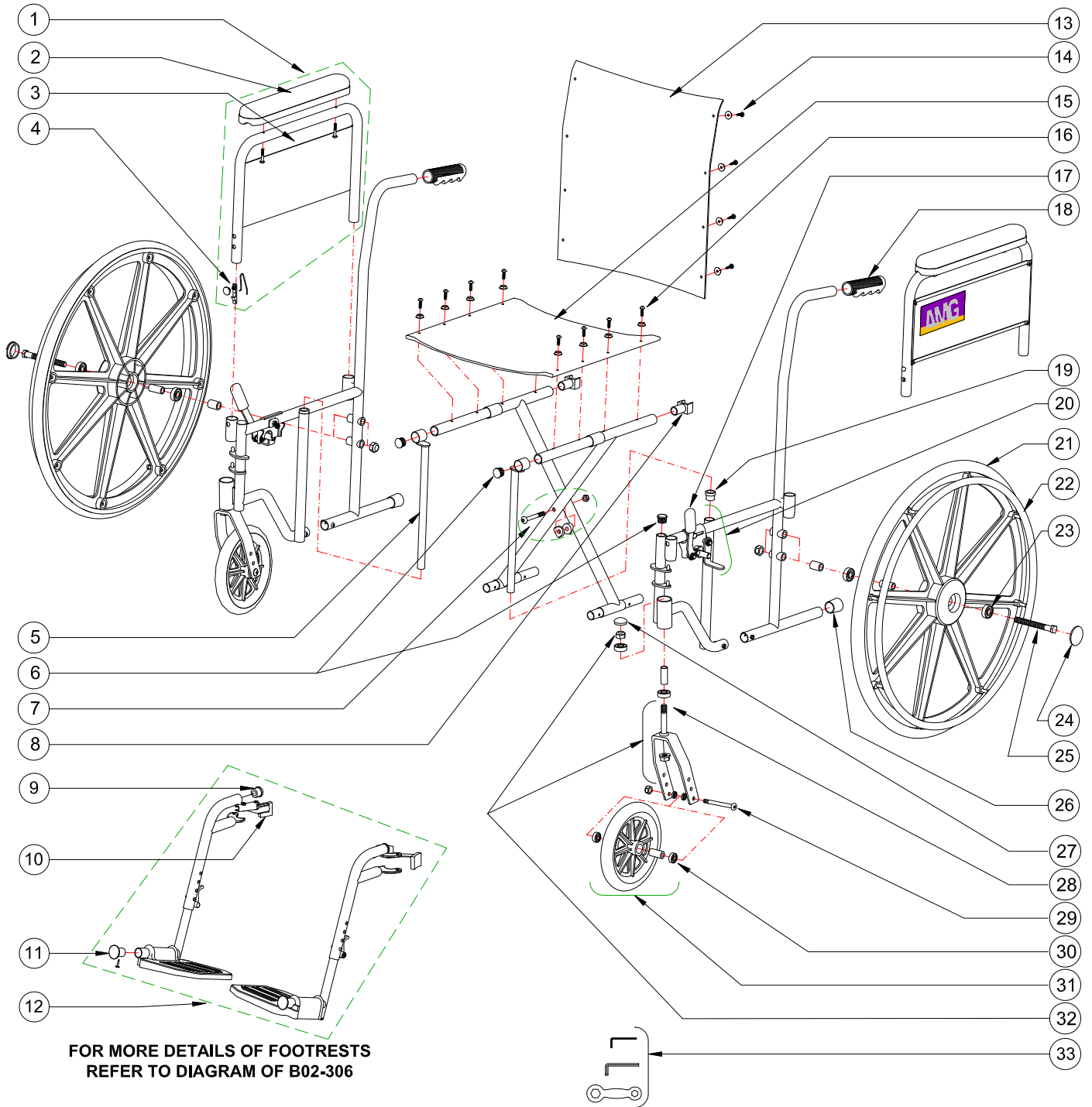

700-622
AMG WHEELCHAIR 18" WIDE
DETACHABLE FULL ARMS AND SWING-AWAY FOOTRESTS

Item #	AMG Part #	Description	Qty Per Unit	Sold by	UOM
1.	701-071	Full Armrest, Left (Black Vinyl Arm Pad)	1	1	EA
	701-072	Full Armrest, Right (Black Vinyl Arm Pad)	1	1	EA
2.	A02-462	Full Arm Pad, Black Vinyl	1	1	PR
3.	A02-122	Hex Socket Screw (M5x30mm) – 4 ea/set	1	1	ST
4.	B02-291	Release Mechanism for Arm (Spring, Button & Latch)	2	1	EA
5.	A02-660	Sliding Tube Attachment, Chrome Plated	2	1	EA
6.	A02-425	End Plug for Seat Rail, Black Plastic	2	1	EA
7.	B02-820	Hardware for Crossbar Pivot (Bolt M8x75mm, 2 Curved Washers & Hex Nut)	1	1	ST
8.	A02-475	Rear Seat Guide, Black Plastic	2	1	EA
9.	A02-430	Bumper for Footrest	2	1	EA
10.	B02-340 C	Release Mechanism Assembly (Long Handle, 2 Bolts, 2 Hex Nuts & Spring), Left & Right – Chrome Plated	1	1	PR
11.	B02-342	Rubber Footplate Plug with Self-threading Screw	2	1	ST
12.	B02-306	Swing-Away Footrests, Tool-free Adjustable (Left & Right)	1	1	PR
13.	701-010	Back Upholstery 18"x18", Black Vinyl, with Hardware (8 Screws & 8 Washers)	1	1	EA
14.	B02-816	Hardware for Back Upholstery (Hex Self-threading Screw and Plastic Washer)	8	1	ST
15.	701-015	Seat Upholstery for 18"x16", Black Vinyl, with Hardware (8 Screws & 8 Washers)	1	1	EA
16.	B02-817	Hardware for Seat Upholstery (Hex Socket Screw M5x18mm & Plastic Washer)	8	1	ST
17.	A02-403	Brake Handle Grip, Black	1	1	PR
18.	A02-405	Hand Grip, Black	1	1	PR
19.	A02-231	Anti-Rattle Insert, Black Plastic	2	1	EA
20.	701-085	Brake Assembly for Detachable Arms W/C, Left	1	1	EA
	701-087	Brake Assembly for Detachable Arms W/C, Right	1	1	EA
21.	701-061	24" Rear Mag Wheel (Grey Solid Tire Wheel with 8 Spokes, Chrome Plated Handrim, 2 Bearings & Bushing)	2	1	EA
22.	701-070	Handrim with Hardware (Chrome Plated Handrim, 16 Washers & 8 Hex Nuts)	2	1	EA
23.	A02-311	6301 Bearing ($\phi 37 \times \phi 12 \times 12$ mm)	4	1	EA
24.	A02-449	Hub Cap, Black	2	1	EA
25.	B02-846	Hardware for 24" Wheel, Detachable Arms W/C (Bolt M12x132mm, Spacer & Hex Nut)	2	1	ST
26.	A02-429	End Plug for Rear Frame, Black Plastic	2	1	EA
27.	A02-447	Front Fork Cover, Black Plastic	2	1	EA
28.	A02-301	6001 Bearing ($\phi 28 \times \phi 12 \times 8$ mm)	4	1	EA
29.	B02-831	Hardware for 8" Front Wheel (Bolt M8x78mm, 2 Spacers & Hex Nut)	2	1	ST
30.	A02-302	608 Bearing ($\phi 22 \times \phi 8 \times 8$ mm)	4	1	EA
31.	B02-131GY	8" Solid Wheel Assembly, Grey (Grey Solid Wheel with 8 Spokes, 2 Bearings & Bushing)	2	1	EA
32.	B02-143	Aluminum Fork for 8" Caster, Anodized Black (Fork, Stem, Hex Nut & Flange Nut)	2	1	EA
33.	A02-703	Tool Package (Allen Key # 4 mm & 6mm & Wrench for 13mm & 19mm)	1	1	ST