


# 700-844 AIRGO ULTRA-LIGHT WEIGHT TRANSPORT CHAIR 19" WIDE

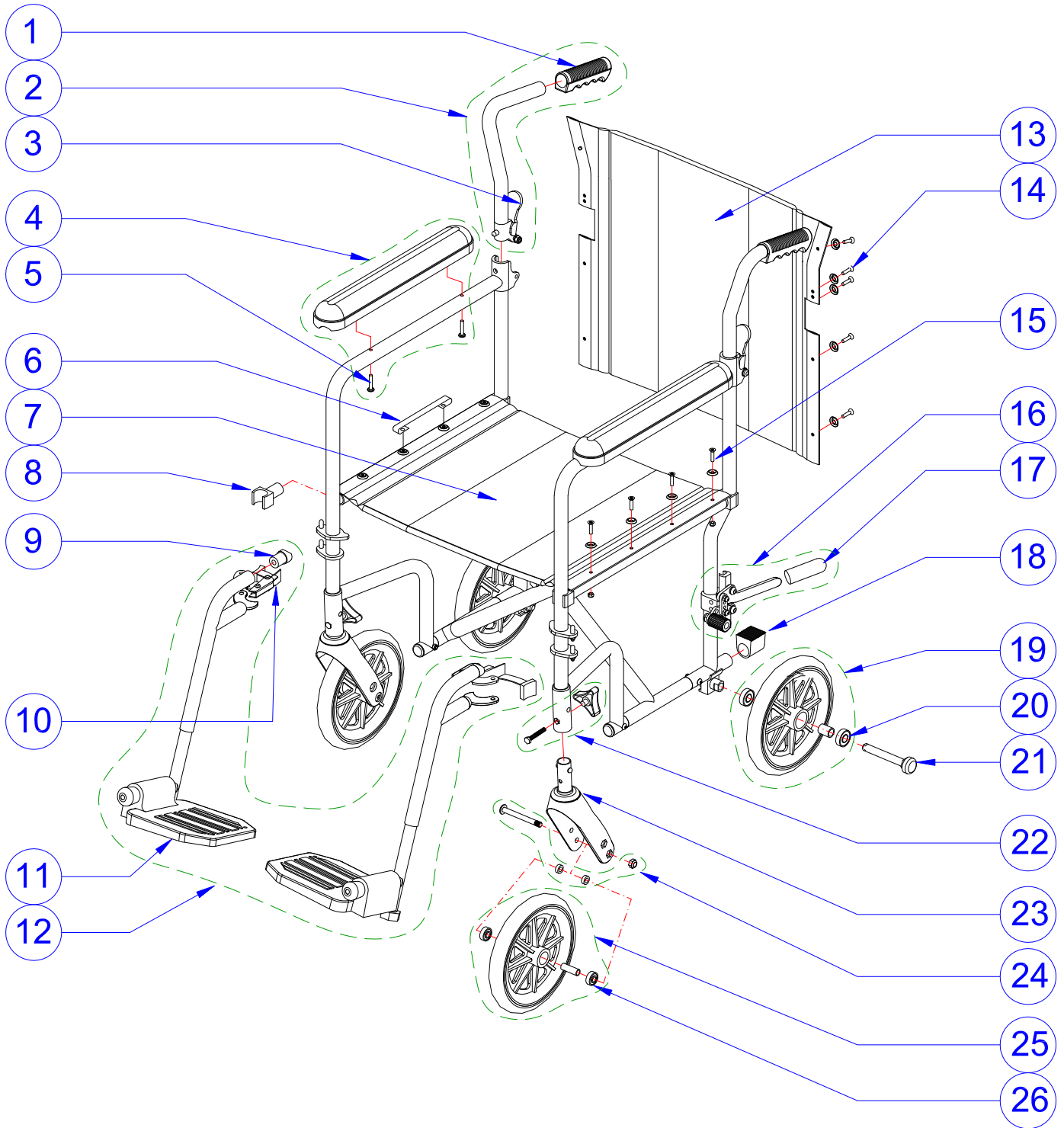

**700-844**  
**AIRGO ULTRA-LIGHT WEIGHT TRANSPORT CHAIR 19" WIDE**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>A07-401</b>	Hand Grip, Black (2 ea/set)	1	1	ST
2.	<b>B07-301L</b>	Folding Black Handle, Left (Left Black, Handle Bar, Hand Grip & Folding Mechanism)	1	1	EA
	<b>B07-301R</b>	Folding Black Handle, Right (Right Black Handle Bar, Hand Grip & Folding Mechanism)	1	1	EA
3.	<b>A07-421</b>	Release Lever to Fold Handle, Black (2 ea/set)	1	1	ST
4.	<b>B07-251</b>	Full Armrest, Black Nylon with Hardware (2 Arm Pads 360x57mm, 4 Round Philips Screws M5x25mm)	1	1	ST
5.	<b>B07-821</b>	Hardware for Armrest (4 Round Philips Screws M5x25mm)	1	1	ST
6.	<b>A07-601</b>	Seat Strap, Black Nylon (2 ea /set)	1	1	ST
7.	<b>B07-421</b>	Seat Upholstery 19"x15.5", Black Nylon, with Hardware (2 Supporting Plates 13x360mm, 4 Round Philips Screws M5x32mm, 4 Hex Locknuts M5, 4 Philips Self-threading Screws M5x17mm & 8 Black Saddle Washers)	1	1	EA
8.	<b>A07-411</b>	Seat Guide, Black (4 ea/set)	1	1	ST
9.	<b>A07-431</b>	Bumper for Footrest, Black (2 ea/set)	1	1	ST
10.	<b>B07-391</b>	Release Handle Mechanism with Hardware, left and right (2 Long Handle L+R, 4 Philips Bolts: 2 M5x20mm + 2 M5x25mm, 2 Hex Locknuts M5, 6 Spacers: 2 long, 2 short and 2 special, and 2 Springs)	1	1	PR
11.	<b>A07-451</b>	Foot Plate, Black (2 ea/set)	1	1	ST
12.	<b>B07-351</b>	Detachable Swing-Away Footrests, Left & Right	1	1	PR
13.	<b>B07-401</b>	Back Upholstery 19"x16", Black Nylon, with Hardware (2 Supporting Plates 13x195mm, 10 Self-threading Screws M5x17mm & 10 Black Saddle Washers)	1	1	EA
14.	<b>B07-801</b>	Hardware for Back Upholstery (10 Philips Self-threading Screw M5x17mm & 10 Black Saddle Washer)	1	1	ST
15.	<b>B07-811</b>	Hardware for Seat Upholstery (4 Round Philips Screws M5x32mm, 4 Locknuts M5, 4 Philips Self-threading Screws M5x17mm & 8 Black Saddle Washers)	1	1	ST
16.	<b>B07-201</b>	Wheel Lock Assembly for Airgo Transport Chair, Left & Right	1	1	PR
17.	<b>A07-402</b>	Wheel Lock Grip, Black	1	1	ST
18.	<b>A07-435</b>	Plug for Rear Frame, Black	1	1	ST
19.	<b>B07-126</b>	8" (20.3 cm) Solid Rear Wheel Assembly, Black (Black Tire Wheel with 8 Spokes, 2 Bearings 6001Z & Bushing)	2	1	EA
20.	<b>A07-301</b>	6001Z Bearing (28 diameter x 12 diameter x 8 mm)	4	1	EA
21.	<b>B07-861</b>	Quick-Release Bolt System for Wheel	2	1	EA
22.	<b>B07-831</b>	Front Fork Secure Knob (Triangular Knob, Hex Bolt M6x35mm & Curved Washer)	2	1	ST
23.	<b>B07-151</b>	Front Fork Assembly (Plastic Fork with 2 Holes, 2 Bearings 6001, Bushing, Locknut M12, Plastic Cover, Stem M12, Push-Buttons)	2	1	EA
24.	<b>B07-851</b>	Hardware for 8" Front Wheel (Hex Socket Bolt M8x78mm, 2 Spacers & 1 Hex Locknut M8)	2	1	ST
25.	<b>B07-101</b>	8" Solid Front Wheel Assembly, Black (Black Tire Wheel with 8 Spokes, 2 Bearings 608 & Bushing)	2	1	EA
26.	<b>A07-305</b>	608 Bearing (22 mm diameter x 8 mm diameter x 8 mm)	4	1	EA