


# 700-855 AMG TRANSPORT CHAIR 19" WIDE

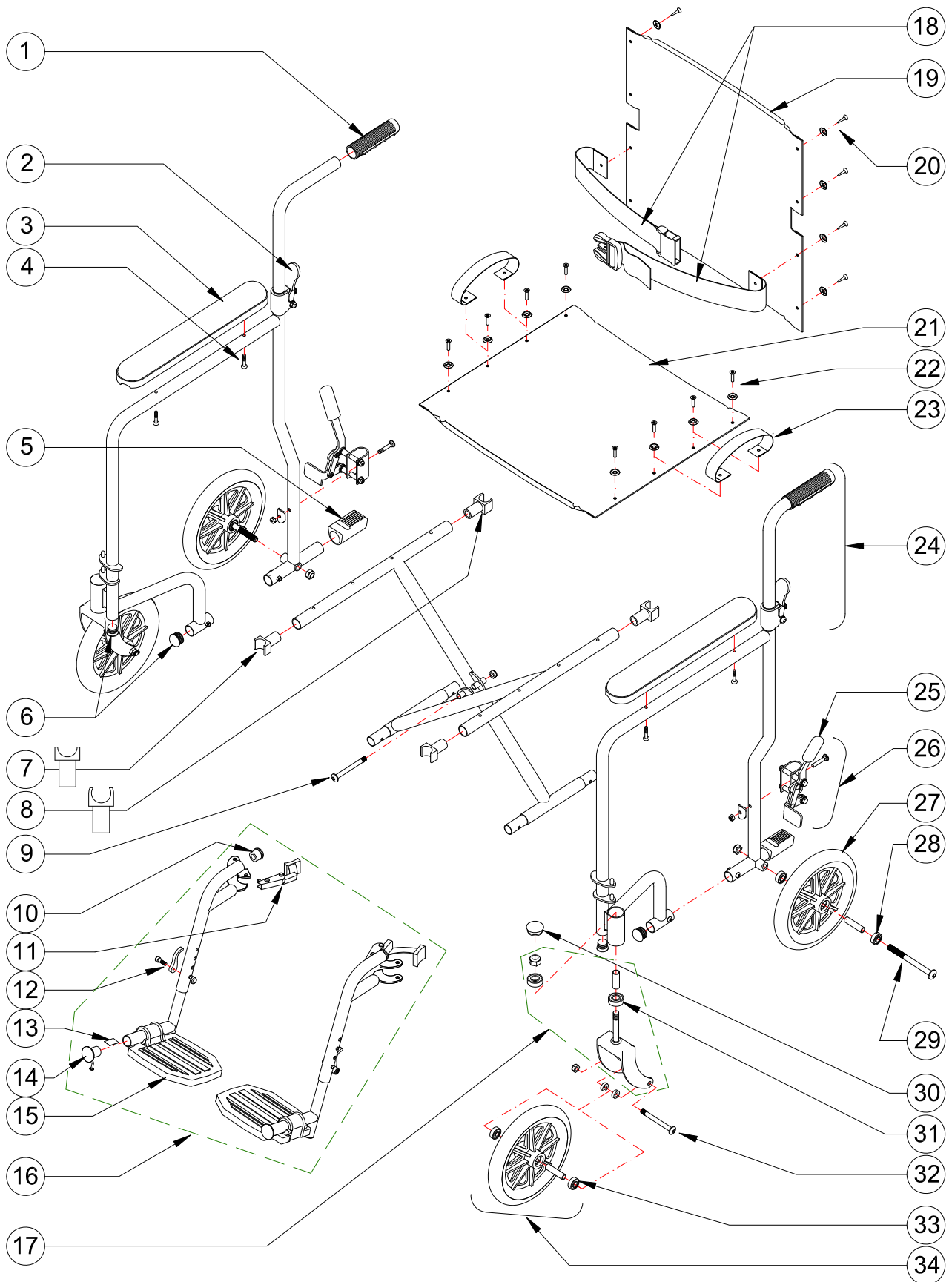

**700-855**  
**AMG TRANSPORT CHAIR 19" WIDE**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>A02-405</b>	Hand Grip, Black	1	1	PR
2.	<b>A02-411</b>	Release Lever to Fold Handle, Black	2	1	EA
3.	<b>A02-462</b>	Full Vinyl Arm Pad, Black	1	1	<b>PR</b>
4.	<b>A02-122</b>	Hex Socket Screw (M5x30mm) – 4 ea/set	1	1	ST
5.	<b>A02-421</b>	Step Tube Plug, Black	2	1	EA
6.	<b>A02-422</b>	Plug for Frame, Black	2	1	EA
7.	<b>A02-474</b>	Front Seat Guide, Black Plastic	2	1	EA
8.	<b>A02-475</b>	Rear Seat Guide, Black Plastic	2	1	EA
9.	<b>B02-821</b>	Hardware for Crossbar Pivot (Hex Bolt M8x75 & Hex Nut)	1	1	ST
10.	<b>A02-430</b>	Bumper for Footrest, Black Rubber	2	1	EA
11.	<b>B02-340C</b>	Footrest Release Mechanism Assembly (Long Handle, 2 Hex Bolts, 2 Hex Nuts & Spring), Left & Right – Chrome Plated	1	1	PR
12.	<b>B02-492</b>	Tightening Lever for Footrest (Lever & Hex Socket Bolt M6x18mm)	2	1	ST
13.	<b>A02-261</b>	Footplate Leaf Spring, Stainless Steel	2	1	EA
14.	<b>B02-342</b>	Rubber Footplate Plug with Self-threading Screw	2	1	ST
15.	<b>A02-542</b>	Foot Plate, Aluminum	2	1	EA
16.	<b>B02-306</b>	Swing-Away Footrests, Tool-free Adjustable (Left & Right)	1	1	PR
17.	<b>B02-141</b>	Front Fork Assembly (Chrome Plated Fork, 2 Bearings, Spacer & Hex Nut)	2	1	EA
18.	<b>700-864</b>	Safety Belt for Transport Chair	1	1	EA
19.	<b>700-851</b>	Back Upholstery 19"x18", Black Nylon, with Hardware (8 Screws & 8 Washers)	1	1	EA
20.	<b>B02-818</b>	Hardware for Back Upholstery (Self-threading Screw & Washer)	8	1	ST
21.	<b>700-853</b>	Seat Upholstery 19"x16", Black Nylon, with Hardware (8 Screws & 8 Washers)	1	1	EA
22.	<b>B02-819</b>	Hardware for Seat Upholstery (Hex Socket Screw M5x18mm & Washer)	8	1	ST
23.	<b>A02-476</b>	Seat Strap	1	1	PR
24.	<b>B02-272</b>	Handle (Short) with Quick Folding Mechanism (Short Handle Bar, Handle Grip & Folding Mechanism Assembly)	2	1	EA
25.	<b>A02-403</b>	Brake Handle Grip, Black	1	1	PR
26.	<b>B02-250</b>	Brake Assembly for Transport Chair, Left & Right	1	1	PR
27.	<b>B02-133GY</b>	8" Solid Rear Wheel Assembly, Grey (Grey Wheel with 8 Spokes, 2 Bearings & Bushing)	2	1	EA
28.	<b>A02-304</b>	608 Bearing (φ23xφ9.5x9mm)	4	1	EA
29.	<b>B02-833</b>	Hardware for Rear Wheel for Transport Chair (Hex Socket Bolt M3/8"x110mm & Hex Nut)	2	1	ST
30.	<b>A02-447</b>	Front Fork Cover, Black	2	1	EA
31.	<b>A02-301</b>	6001 Bearing (φ28xφ12x8mm)	4	1	EA
32.	<b>B02-831</b>	Hardware for Front Wheel (Hex Socket Bolt M8x78mm, 2 Spacers & Hex Nut)	2	1	ST
33.	<b>B02-131GY</b>	8" Solid Front Wheel Assembly, Grey ( Grey Wheel with 8 Spokes, 2 Bearings & Bushing)	2	1	EA
34.	<b>A02-302</b>	608 Bearing (φ22xφ8x8mm)	4	1	EA
Not shown	<b>A02-702</b>	Tool Package (Allen Key #4 mm & #6mm)	1	1	ST