


700-869 (MIDNIGHT-BLUE) HUGO TRANSPORT CHAIR 19" WIDE

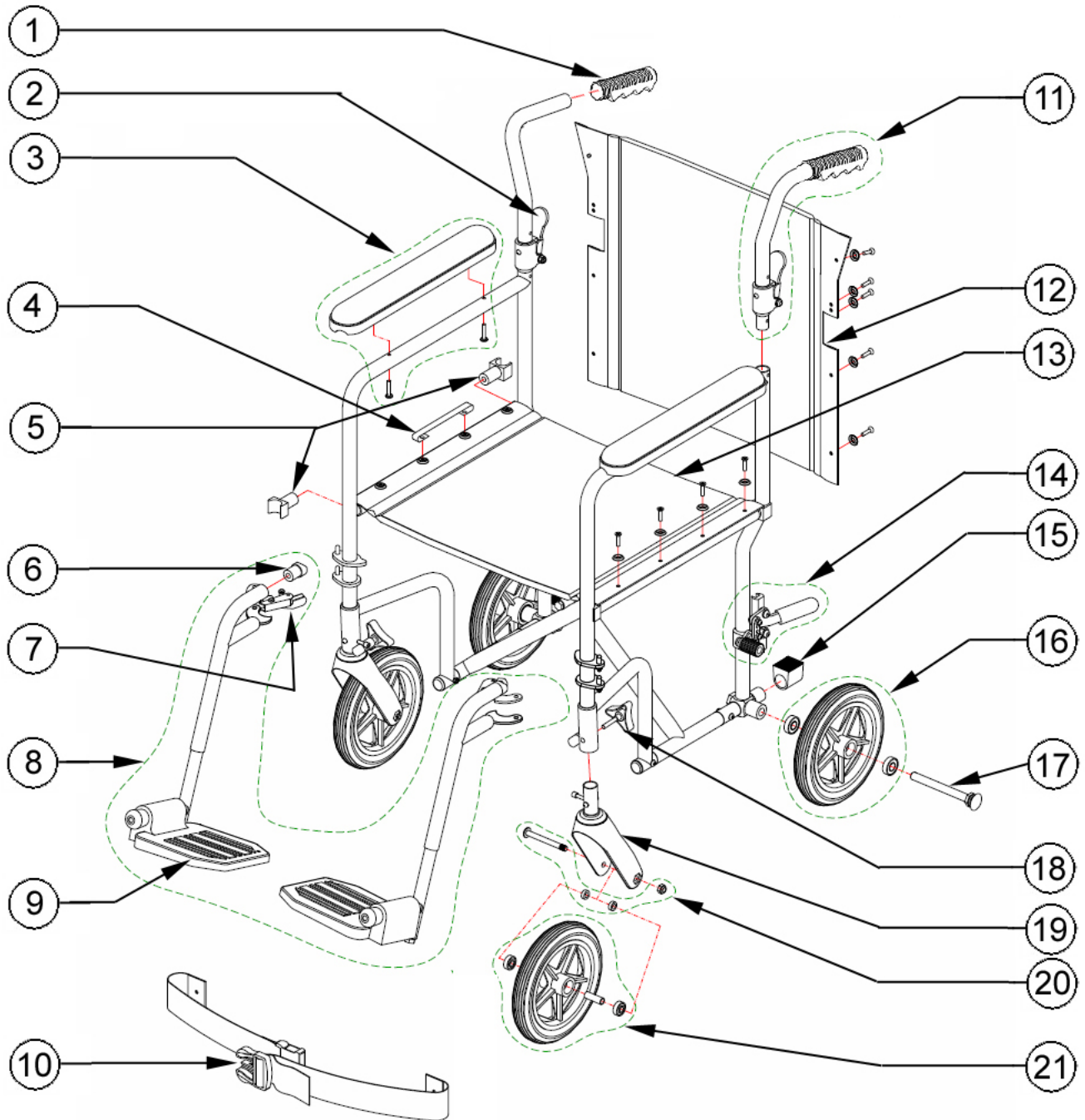

**700-869 (MIDNIGHT-BLUE)
HUGO TRANSPORT CHAIR 19" WIDE**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	A07-401	Hand Grip, Black (2 ea/set)	1	1	ST
2.	A07-421	Release Lever to Fold Handle, Black (2 ea/set)	1	1	ST
3.	B07-251	Full Armrest, Black Nylon with Hardware (2 Arm Pads 360x57mm, 4 Round Philips Screws M5x25mm)	1	1	ST
4.	A07-601	Seat Strap, Black Nylon (2 ea /set)	1	1	ST
5.	A07-411	Seat Guide (Front & Back), Black (4 ea/set)	1	1	ST
6.	A07-431	Bumper for Footrest, Black (2 ea/set)	1	1	ST
7.	B07-391	Release Handle Mechanism with Hardware, Left & Right (2 Long Handle L+ R, 4 Philips Bolts: 2 M5x20mm + 2 M5x25mm, 2 Hex Locknuts M5, 6 Spacers: 2 Long + 2 Short + 2 Special and 2 Springs)	1	1	PR
8.	B02-351	Detachable Swing-Away Footrests, Blue	1	1	PR
9.	A07-451	Foot Plate, Black (2 ea/set)	1	1	ST
10.	700-864	Safety Belt, Black	1	1	EA
11.	B07-303	Folding Blue Handles (Left & Right Blue Handle Bar, Hand Grips & Folding Mechanism)	1	1	PR
12.	B07-402	Hugo Logo Back Upholstery 19"x16", Black Nylon, with Hardware (2 Supporting Plates 13x195mm, 10 Self-threading Screws M5x17mm & 10 Black Saddle Washers)	1	1	EA
13.	B02-431	Seat Upholstery 19"x15.5", Black Nylon, with Hardware	1	1	EA
14.	B02-211	Wheel Lock Assembly for Transport Chair, Left & Right	1	1	PR
15.	A07-435	Plug for Rear Frame, Black	1	1	ST
16.	B07-126	8" Solid Rear Wheel Assembly, Black (Black Tire Wheel with 8 Spokes, 2 Bearings 6001Z & Bushing)	2	1	EA
17.	B07-861	Quick-Release Bolt System for Wheel	2	1	EA
18.	B07-831	Front Fork Secure Knob (Triangular Knob, Hex Bolt M6x35mm & Curved Washer)	2	1	ST
19.	B07-151	Front Fork Assembly (Plastic Fork with 2 Holes, 2 Bearings 6001, Bushing, Locknut M12, Plastic Cover, Stem M12, Push-Buttons)	2	1	EA
20.	B07-851	Hardware for 8" Front Wheel (Hex Socket Bolt M8x78mm, 2 Spacers & Hex Locknut M8)	2	1	ST
21.	B07-101	8" Solid Front Wheel Assembly, Black (Black Tire Wheel with 8 Spokes, 2 Bearings 608 & Bushing)	2	1	EA