


700-940 (PACIFIC BLUE)
Airgo® Lightweight Rollator
8"x1.4" WHEELS, AIRGO HANDLES, VINYL SEAT AND CURVED BACKREST

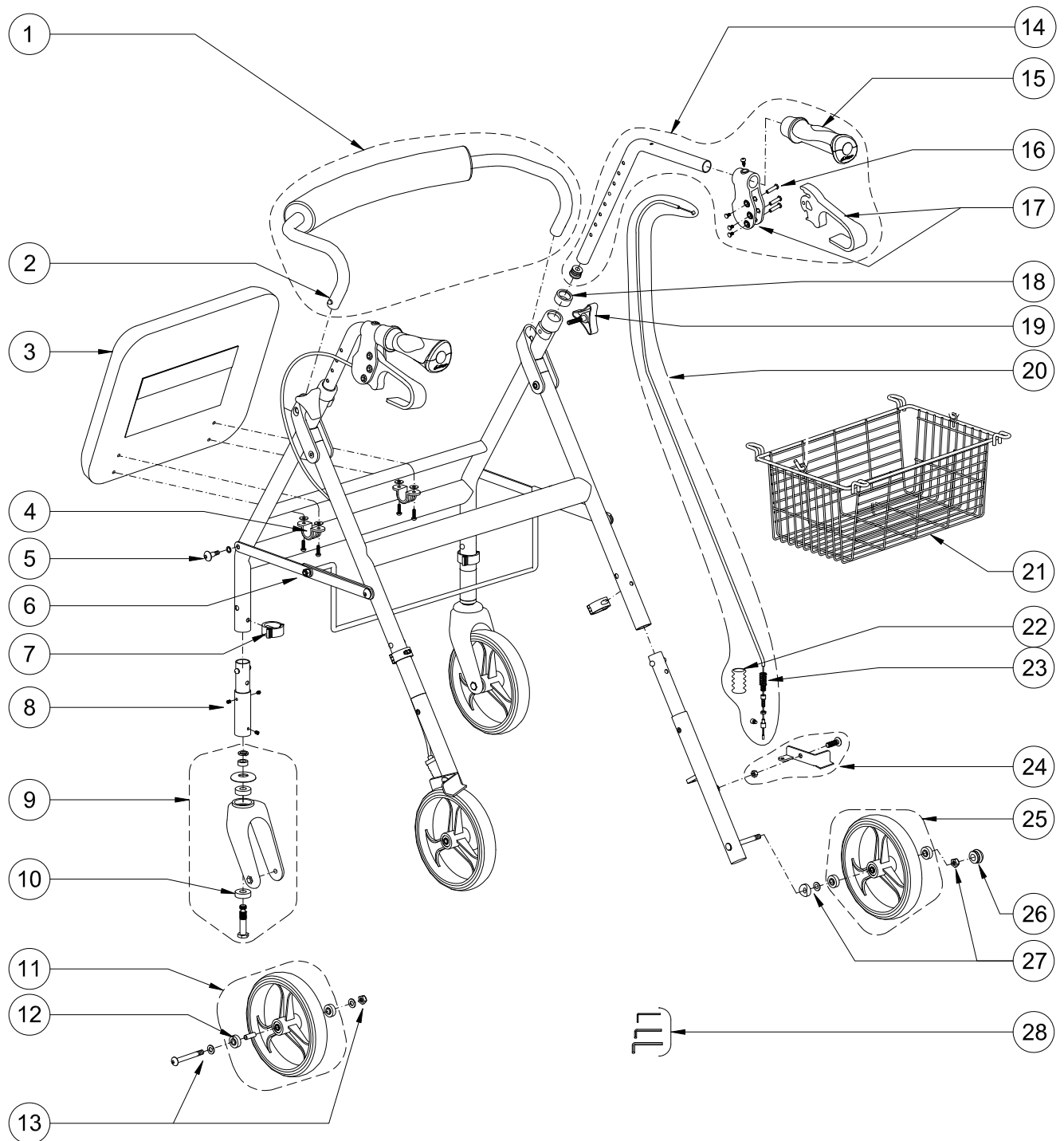

700-940 (PACIFIC BLUE)
Airgo® Lightweight Rollator
8"x1.4" WHEELS, AIRGO HANDLES, VINYL SEAT AND CURVED BACKREST

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	B01-204	Curved Backrest Assembly (Curved Black Tube & 19mm, Foam, Airgo Nylon Cover, 2 Push-Buttons & 2 Springs)	1	1	EA
2.	B01-811	Push-Button & Spring (2 each / set)	1	1	ST
3.	B01-271	Vinyl Seat Cushion (12" x 12") with Hardware (2 Brackets, 2 Screws M5x15 & 4 Washers)	1	1	EA
4.	B01-816	Hardware for Seat (2 Brackets, 4 Screws M5x15mm & 8 Washers)	1	1	ST
5.	B01-822	Hardware for Folding Mechanism (4 Screws, 4 Steel Washers & 4 white Washers)	1	1	ST
6.	B01-086	Folding Mechanism Complete	1	1	ST
7.	A01-241	E-Clip, Black – 4 ea / set	1	1	ST
8.	B01-011PB	Removable Front Connection Tubing with Hardware, Pacific-Blue, L&R	1	1	PR
9.	B01-122	Front Fork Assy (Lock Washer, Spacer, Plastic Cover, 2 Bearings, Plastic Fork, Bolt M12 & Locking Screw)	2	1	EA
10.	A01-301	6001 Bearing	4	1	EA
11.	B01-146	8"x1.4" Hugo Front Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
12.	A01-302	608 Bearing	8	1	EA
13.	B01-832	Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Washers & Nut)	2	1	ST
14.	B01-025L	Left Airgo Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Blue Airgo Handle Grip & End Plug)	1	1	EA
	B01-025R	Right Airgo Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Blue Airgo Handle Grip & End Plug)	1	1	EA
15.	B01-401BL	Blue Airgo Handle Grip, Left & Right	1	1	PR
16.	B01-851	Hardware for Brake Handle (3 Threaded Standoffs, 3 Screws M4x10mm & 1 Self-threading Screw)	2	1	ST
17.	B01-156	Brake Handle Assembly (Brake Handle & Housing, Adjustment Screw, Self-threading Screw, 3 Threaded Standoffs & 3 Screws M4x10mm), L&R	1	1	PR
18.	A01-206	Anti-Rattle, Plastic	2	1	EA
19.	A01-415	Triangular Knob, M6x30mm	2	1	EA
20.	B01-157	Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper)	2	1	ST
21.	A01-615	Wire Basket (385x195x145mm), Black	1	1	EA
22.	A01-256	Brake Spring	2	1	EA
23.	A01-419	Spring Sleeve, Black	2	1	EA
24.	B01-159L	Bottom Brake Assembly, Left (Bolt M6x22mm, Left Brake Pad & Nut)	1	1	ST
	B01-159R	Bottom Brake Assembly, Right (Bolt M6x22mm, Right Brake Pad & Nut)	1	1	ST
25.	B01-147	8"x1.4" Hugo Rear Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
26.	A01-453	Hub Cap for Wheel	2	1	EA
27.	B01-845	Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)	2	1	ST
28.	A01-701	Tool Package (Allen Keys #4mm, #5mm & #6mm)	1	1	PK