


**700-941 (CRANBERRY)**  
**AIRGO ADVENTURE-SIX, ROLLATOR 4.0**  
**DUAL-HEIGHT, 6" WHEELS, AIRGO-HANDLES, VINYL SEAT AND CURVED BACKREST**

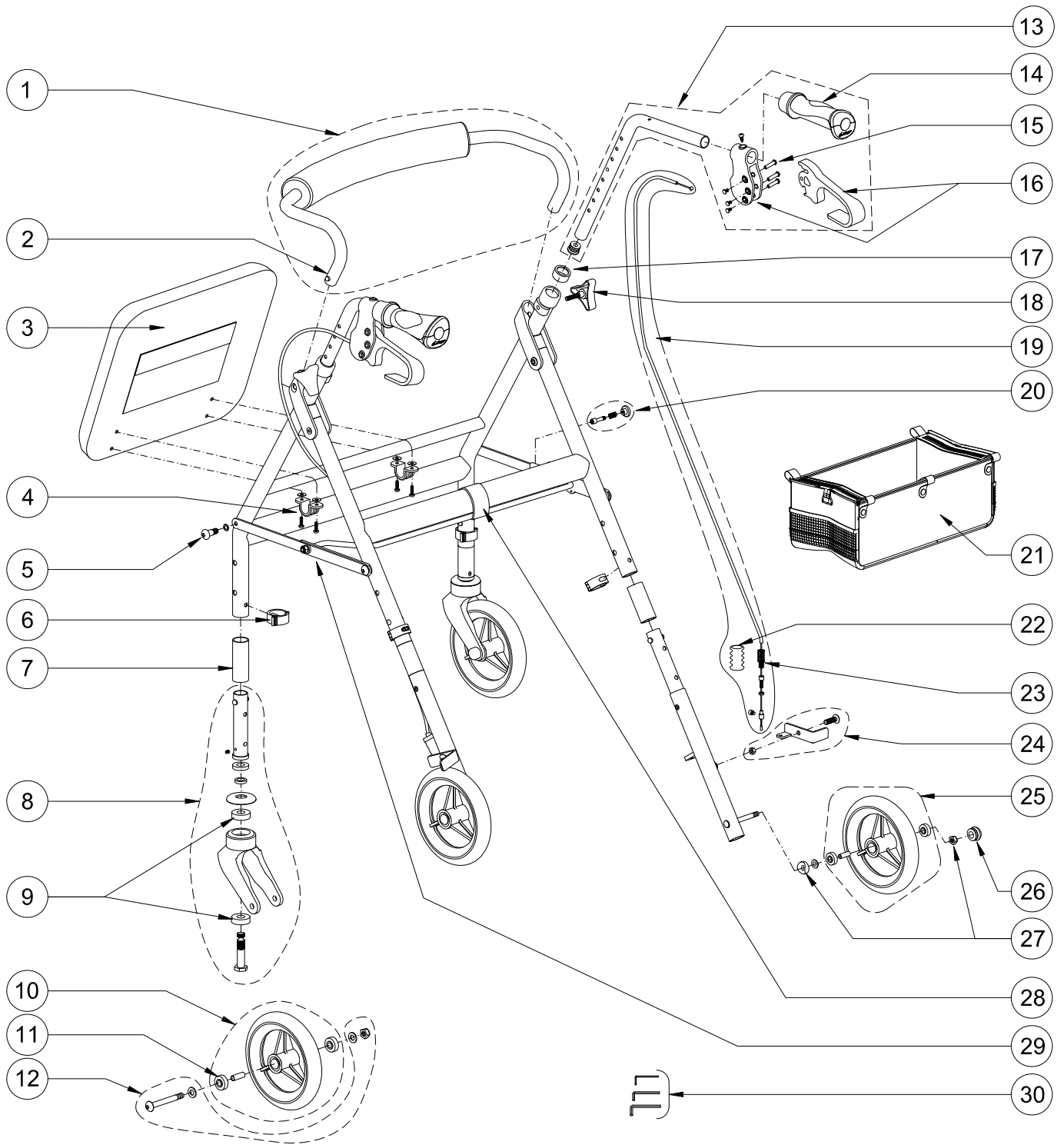

**700-941 (CRANBERRY)**  
**AIRGO ADVENTURE-SIX, ROLLATOR 4.0**  
**DUAL-HEIGHT, 6" WHEELS, AIRGO-HANDLES, VINYL SEAT AND CURVED BACKREST**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>B01-204</b>	Curved Backrest Assembly (Curved Black Tube & 19mm, Foam, Airgo Nylon Cover, 2 Push-Buttons & 2 Springs)	1	1	EA
2.	<b>B01-811</b>	Push-Button & Spring (2 each / set)	1	1	ST
3.	<b>B01-270</b>	Vinyl Seat Cushion (14" x 12") with Hardware (2 Brackets, 2 Screws M5x15 & 4 Washers)	1	1	EA
4.	<b>B01-816</b>	Hardware for Seat (2 Brackets, 4 Screws M5x15mm & 4 Washers)	1	1	ST
5.	<b>B01-822</b>	Hardware for Folding Mechanism (4 Screws, 4 Steel Washers & 4 white Washers)	1	1	ST
6.	<b>A01-241</b>	E-Clip, Black – 4 ea / set	1	1	ST
7.	<b>A01-511CR</b>	3" Extension Tubing, Cranberry (4 each / set)	1	1	ST
8.	<b>B01-126</b>	Front Fork Assy for Airgo 6" (Fork, Aluminum Housing, 2 Spacers, Plastic Cover, 2 Bearings, Plastic Fork, Bolt M12 & Retaining Screw)	2	1	EA
9.	<b>A01-301</b>	6001 Bearing	4	1	EA
10.	<b>B01-138</b>	6" Front Wheel, Black (3 Spokes Hub Wheel, Bushing & 2 Bearings)	2	1	EA
11.	<b>A01-302</b>	608 Bearing	8	1	EA
12.	<b>B01-831</b>	Hardware for 6" Front Wheel (Bolt M8x62mm, 2 Washers & Nut)	2	1	ST
13.	<b>B01-026L</b>	Left Airgo Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Cranberry Airgo Handle Grip & End Plug)	1	1	EA
	<b>B01-026R</b>	Right Airgo Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Cranberry Airgo Handle Grip & End Plug)	1	1	EA
14.	<b>B01-401CR</b>	Cranberry Airgo Handle Grip, Left & Right	1	1	PR
15.	<b>B01-851</b>	Hardware for Brake Handle (3 Threaded Standoffs, 3 Screws M4x10mm & 1 Self-threading Screw)	2	1	ST
16.	<b>B01-156</b>	Brake Handle Assembly (Brake Handle & Housing, Adjustment Screw, Self-threading Screw, 3 Threaded Standoffs & 3 Screws M4x10mm), L&R	1	1	PR
17.	<b>A01-206</b>	Anti-Rattle, Plastic	2	1	EA
18.	<b>A01-415</b>	Triangular Knob, M6x30mm	2	1	EA
19.	<b>B01-157</b>	Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper)	2	1	ST
20.	<b>B01-090</b>	Locking Subassembly for Folding Mechanism (Red Button, Spring & Pin)	1	1	ST
21.	<b>A01-491</b>	Soft Basket for "Airgo Adventure", Black	1	1	EA
22.	<b>A01-256</b>	Brake Spring	2	1	EA
23.	<b>A01-419</b>	Spring Sleeve, Black	2	1	EA
24.	<b>B01-159L</b>	Bottom Brake Assembly, Left (Bolt M6x22mm, Left Brake Pad & Nut)	1	1	ST
	<b>B01-159R</b>	Bottom Brake Assembly, Right (Bolt M6x22mm, Right Brake Pad & Nut)	1	1	ST
25.	<b>B01-139</b>	6" Rear Wheel, Black (3 Spokes Hub Wheel, Bushing & 2 Bearings)	2	1	EA
26.	<b>A01-453</b>	Hub Cap for Wheel	2	1	EA
27.	<b>B01-845</b>	Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)	2	1	ST
28.	<b>A01-433</b>	Velcro Strap for Folding Bar (Black)	1	1	EA
29.	<b>B01-087</b>	Folding Mechanism, Complete with Locking System	1	1	ST

**700-941 (CRANBERRY)**  
**AIRGO ADVENTURE-SIX, ROLLATOR 4.0**  
**DUAL-HEIGHT, 6" WHEELS, AIRGO-HANDLES, VINYL SEAT AND CURVED BACKREST**

30.	<b>A01-701</b>	Tool Package (Allen Keys #4mm, #5mm & #6mm)	1	1	PK
-----	----------------	---	---	---	----