


700-949 (IRIDESCENT BLUE)
AIRGO COMFORT-PLUS XWD ROLLATOR 4.0
DUAL-HEIGHT, 8" COMFORT-GRIP WHEELS, ERGO SEAT AND AIRGO BACKREST

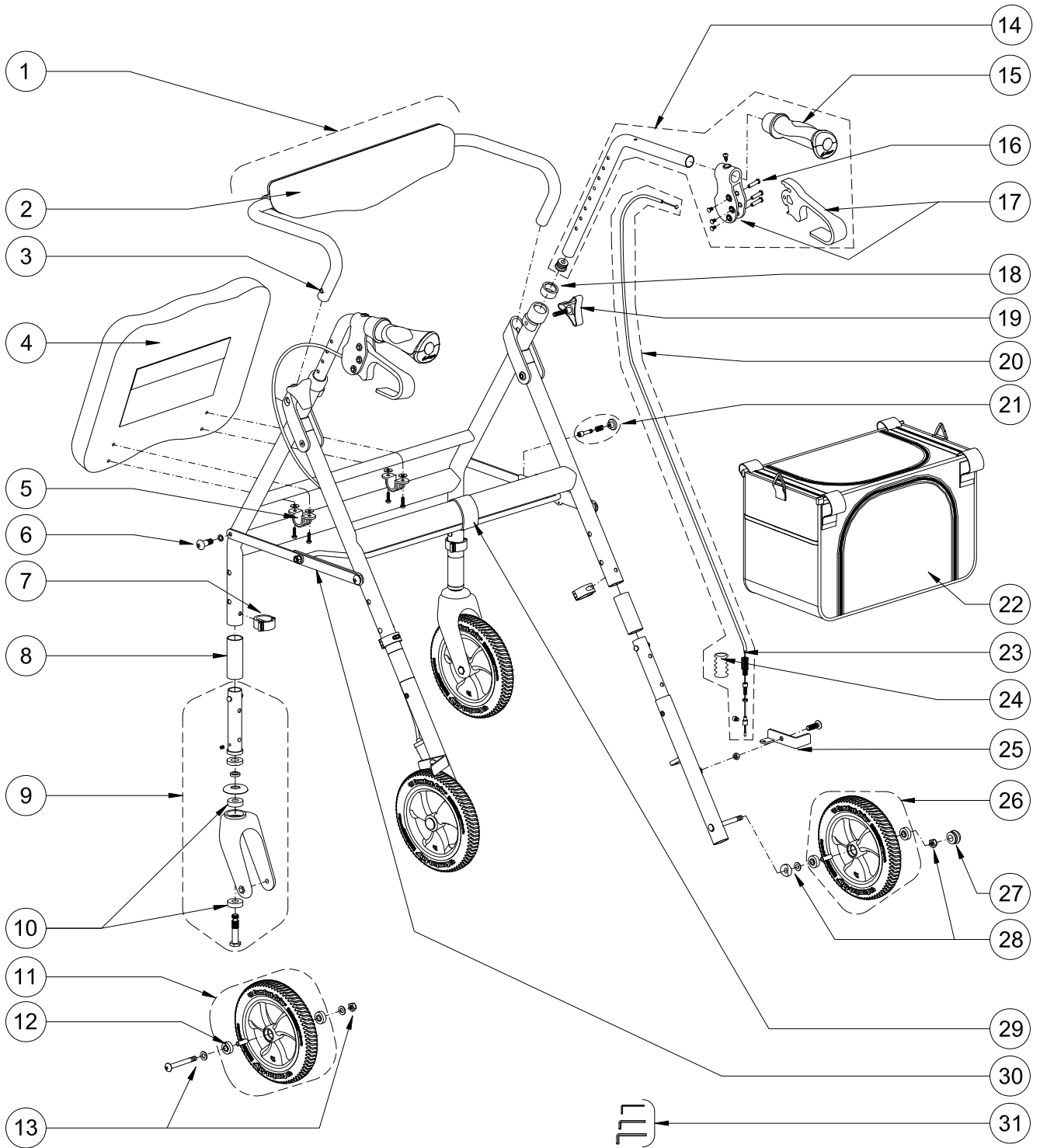

700-949 (IRIDESCENT BLUE)
AIRGO COMFORT-PLUS XWD ROLLATOR 4.0
DUAL-HEIGHT, 8" COMFORT-GRIP WHEELS, ERGO SEAT AND AIRGO BACKREST

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	B01-221	Airgo Backrest XWD Assembly (Black Tube & 19mm, Airgo Back Pad, Bracket, 6 Flat Head Screws M5x16mm, 2 Push-Buttons & 2 Springs)	1	1	EA
2.	B01-251	Airgo Back Pad with Bracket & Hardware (6 Flat Head Screws M5x16mm)	1	1	EA
3.	B01-811	Push-Button & Spring (2 each / set)	1	1	ST
4.	A01-471	Ergo Seat XWD, Visco-Elastic Foam, Two Tones Grey-Blue	1	1	EA
5.	B01-816	Hardware for Seat (2 Brackets, 4 Screws M5x15mm & 4 Washers)	1	1	ST
6.	B01-822	Hardware for Folding Mechanism (4 Screws, 4 Steel Washers & 4 white Washers)	1	1	ST
7.	A01-241	E-Clip, Black – 4 ea / set	1	1	ST
8.	A01-511 IB	3" Extension Tubing, Iridescent-Blue (4 each / set)	1	1	ST
9.	B01-125	Front Fork Assy (Fork, Aluminum Housing, 2 Spacers, Plastic Cover, 2 Bearings, Plastic Fork, Bolt M12 & Retaining Screw)	2	1	EA
10.	A01-301	6001 Bearing	4	1	EA
11.	B01-141	8" Comfort-Plus Front Wheel (PU Tire 8"x1.25" with Full Hub, Bushing & 2 Bearings)	2	1	EA
12.	A01-302	608 Bearing	8	1	EA
13.	B01-832	Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Washers & Nut)	2	1	ST
14.	B01-030L	Left Airgo Handle Bar for XW-Roll. with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Blue Airgo-Handle Grip & End Plug)	1	1	EA
	B01-030R	Right Airgo Handle Bar XW-Roll. with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Blue Airgo-Handle Grip & End Plug)	1	1	EA
15.	B01-401 BL	Blue Airgo Handle Grip, Left & Right	1	1	PR
16.	B01-851	Hardware for Brake Handle (3 Threaded Standoffs, 3 Screws M4x10mm & 1 Self-threading Screw)	2	1	ST
17.	B01-156	Brake Handle Assembly (Brake Handle & Housing, Adjustment Screw, Self-threading Screw, 3 Threaded Standoffs & 3 Screws M4x10mm), L&R	1	1	PR
18.	A01-206	Anti-Rattle, Plastic	2	1	EA
19.	A01-415	Triangular Knob, M6x30mm	2	1	EA
20.	B01-157	Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper)	2	1	ST
21.	B01-090	Locking Subassembly for Folding Mechanism (Red Button, Spring & Pin)	1	1	ST
22.	A01-492	Soft Basket for "Comfort-Plus", Extra-Large, Black	1	1	EA
23.	A01-256	Brake Spring	2	1	EA
24.	A01-419	Spring Sleeve, Black	2	1	EA
25.	B01-159L	Bottom Brake Assembly, Left (Bolt M6x22mm, Left Brake Pad & Nut)	1	1	ST
	B01-159R	Bottom Brake Assembly, Right (Bolt M6x22mm, Right Brake Pad & Nut)	1	1	ST
26.	B01-142	8" Comfort-Plus Rear Wheel (PU Tire 8"x1.25" with Full Hub, Bushing & 2 Bearings)	2	1	EA
27.	A01-453	Hub Cap for Wheel	2	1	EA
28.	B01-845	Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)	2	1	ST
29.	A01-433	Velcro Strap for Folding Bar (Black)	1	1	EA

700-949 (IRIDESCENT BLUE)
AIRGO COMFORT-PLUS XWD ROLLATOR 4.0
DUAL-HEIGHT, 8" COMFORT-GRIP WHEELS, ERGO SEAT AND AIRGO BACKREST

30.	B01-088	Folding Mechanism for XWD, Complete with Locking System	1	1	ST
31.	A01-701	Tool Package (Allen Keys #4mm, #5mm & #6mm)	1	1	PK