


700-959 (PACIFIC BLUE) HUGO ALUMINUM ROLLATOR WITH 8" REMOVABLE WHEELS AND LOOP BRAKES

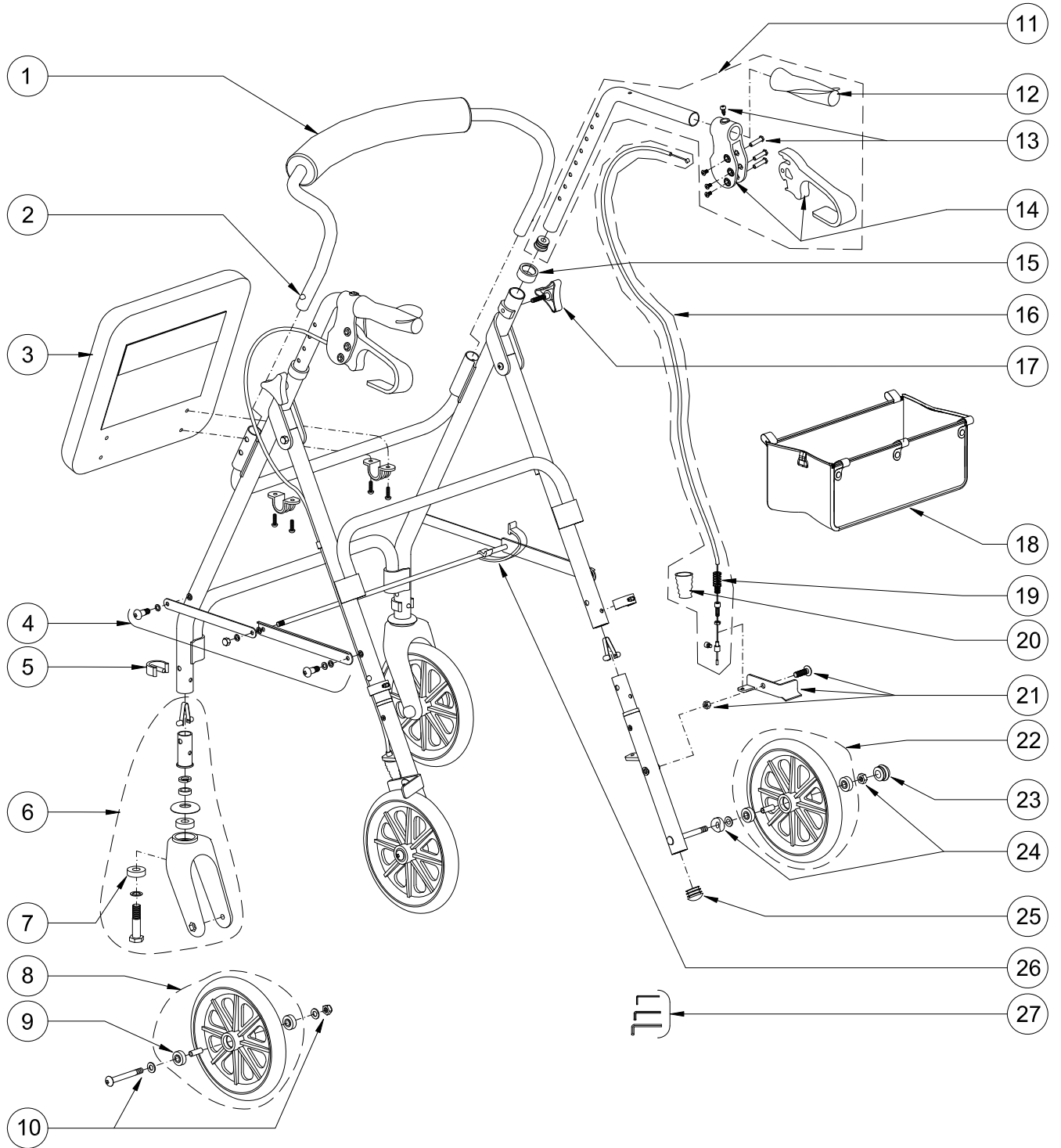

**700-959 (PACIFIC BLUE)
HUGO ALUMINUM ROLLATOR
WITH 8" REMOVABLE WHEELS AND LOOP BRAKES**

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	B01-209BK	Curved Backrest, Black	1	1	EA
2.	B01-811	2 Push Button & 2 Springs	1	1	ST
3.	B01-270	Hugo Seat Cushion, Black , with Hardware (2 Brackets, 2 Screws M5x15 & 4 Washers)	1	1	EA
4.	B01-075	Hugo Folding Mechanism (2 Flat Arms, 2 "L" Arms, 4 Screws, 4 Plastic Washers, 4 Washers, 1 Bar, 4 Brass Washers & 2 Nuts)	1	1	ST
5.	A01-241	E-Clip, Black (for 1" Tubing)	4	1	EA
6.	B01-123	Detachable Front Fork Assy (Hex Bolt M12x50mm, Washer, Spacer, Lock-Washer, Plastic Cover, Buttons & Spring, Plastic Fork, 2 Bearings & Aluminum Housing)	2	1	EA
7.	A01-301	6001 Bearing	4	1	EA
8.	B01-133	8" Front Wheel, Black (PU Tire 8"x1.25" with 8 Spokes, Bushing & 2 Bearings)	2	1	EA
9.	A01-302	608 Bearing	8	1	EA
10.	B01-832	Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Washers & Nut)	2	1	ST
11.	B01-024L	Left Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Ortho-handle Grip & End Plug)	1	1	EA
	B01-024R	Right Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Ortho-handle Grip & End Plug)	1	1	EA
12.	A01-406	Ortho-handle Grip Left & Right	1	1	PR
13.	B01-851	Hardware for Brake Handle (3 Threaded Standoffs, 3 Screws M4x10mm & 1 Self-threading Screw)	2	1	ST
14.	B01-156	Brake Handle Assembly (Brake Handle & Housing, Adjustment Screw, Self-threading Screw, 3 Threaded Standoffs & 3 Screws M4x10mm), L&R	1	1	ST
15.	A01-206	Anti-Rattle, Plastic	2	1	EA
16.	B01-157	Brake Cable & Sleeve Assy (Cable, Sleeve 860mm, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper)	2	1	ST
17.	A01-415	Triangular Knob, M6x30mm	2	1	ST
18.	A01-489	Hugo Soft Basket, Black	1	1	EA
19.	A01-256	Brake Spring	2	1	EA
20.	A01-419	Spring Sleeve, Black	2	1	EA
21.	B01-159L	Bottom Brake Assy, Left (Bolt M6x22mm, Left Brake Pad & Nut)	1	1	ST
	B01-159R	Bottom Brake Assy, Right (Bolt M6x22mm, Right Brake Pad & Nut)	1	1	ST
22.	B01-136	8" Rear Wheel, Black (PU Tire 8"x1.25" with 8 Spokes, Bushing & 2 Bearings)	2	1	EA
23.	A01-453	Hub Cap for Rear Wheel	2	1	EA
24.	B01-845	Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)	2	1	ST
25.	A01-421	End Plug for Rear Frame, Black Plastic	2	1	EA
26.	A01-480	Folding Lock Clip, Red	1	1	EA
27.	A01-701	Tool Package (Allen Keys #4mm, #5mm & #6mm)	1	1	PK