


# 700-959 (PACIFIC BLUE) HUGO ROLLATOR 4.0

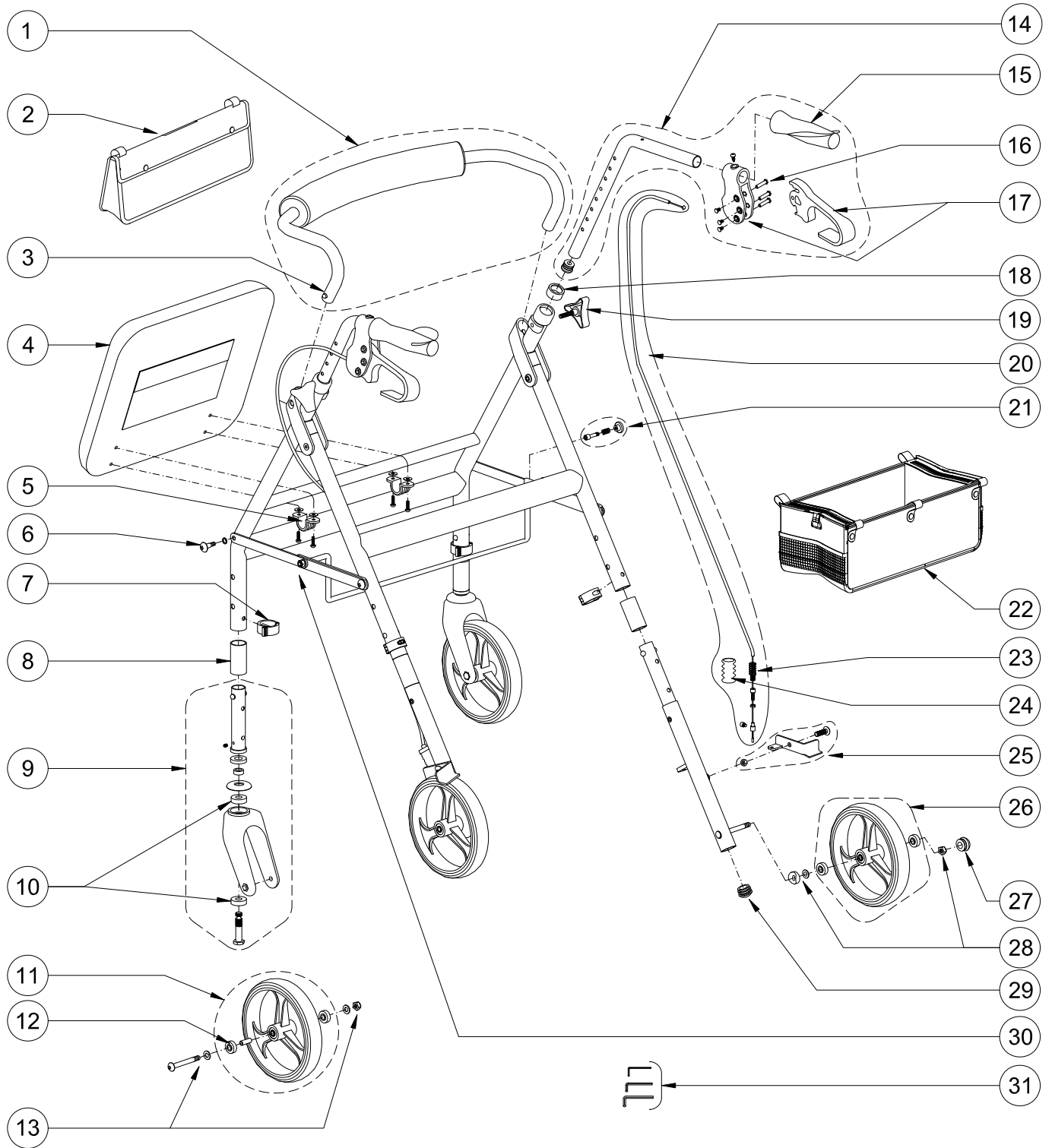

## 700-959 (PACIFIC BLUE) HUGO ROLLATOR 4.0

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>B01-206BK</b>	Curved Backrest Assembly (Curved Black Tube & 19mm, Foam, 2 Push-Buttons & 2 Springs)	1	1	EA
2.	<b>A01-484</b>	Hugo Saddle Bag, Black	1	1	EA
3.	<b>B01-811</b>	2 Push-Button & 2 Springs	1	1	ST
4.	<b>B01-270</b>	Vinyl Seat Cushion (14" x 12") with Hardware (2 Brackets, 2 Screws M5x15 & 4 Washers)	1	1	EA
5.	<b>B01-816</b>	Hardware for Seat (2 Brackets, 4 Screws M 5 x 15 mm, and 8 washers)	1	1	ST
6.	<b>B01-822</b>	Hardware for Folding Mechanism (4 Screws, 4 Steel Washers & 4 white Washers)	1	1	ST
7.	<b>A01-241</b>	E-Clip, Black – 4 ea / set	1	1	ST
8.	<b>A01-510PB</b>	Extension Tubing, Pacific Blue (25.4 diameter x 50mm) – 4 ea / set	1	1	ST
9.	<b>B01-124</b>	Front Fork Assy for Hugo (Fork, Aluminum Housing, 2 Spacers, Plastic Cover, 2 Bearings, Plastic Fork, Bolt M12 & Retaining Screw)	2	1	EA
10.	<b>A01-301</b>	6001 Bearing	4	1	EA
11.	<b>B01-146</b>	8"x1.4" Hugo Front Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
12.	<b>A01-302</b>	608 Bearing	8	1	EA
13.	<b>B01-832</b>	Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Washers & Nut)	2	1	ST
14.	<b>B01-024L</b>	Left Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Ortho-handle Grip & End Plug)	1	1	EA
	<b>B01-024R</b>	Right Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Ortho-handle Grip & End Plug)	1	1	EA
15.	<b>A01-406</b>	Ortho-Handle Grip, Left & Right	1	1	PR
16.	<b>B01-851</b>	Hardware for Brake Handle (3 Threaded Standoffs, 3 Screws M4x10mm & 1 Self-threading Screw)	2	1	ST
17.	<b>B01-156</b>	Brake Handle Assembly (Brake Handle & Housing, Adjustment Screw, Self-threading Screw, 3 Threaded Standoffs & 3 Screws M4x10mm)	2	1	ST
18.	<b>A01-206</b>	Anti-Rattle, Plastic	2	1	EA
19.	<b>A01-415</b>	Triangular Knob, M6x30mm	2	1	EA
20.	<b>B01-157</b>	Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper)	2	1	ST
21.	<b>B01-090</b>	Locking Subassembly for Folding Mechanism	1	1	ST
22.	<b>A01-483</b>	Hugo under Seat Basket (420x225x145mm), Black	1	1	EA
23.	<b>A01-256</b>	Brake Spring	2	1	EA
24.	<b>A01-419</b>	Spring Sleeve, Black	2	1	EA
25.	<b>B01-159</b>	Bottom Brake Assembly, L&R (2 Bolts M6x22mm, Left&Right Brake Pad & 2 Hex Nuts)	1	1	ST
26.	<b>B01-147</b>	8"x1.4" Hugo Rear Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
27.	<b>A01-453</b>	Hub Cap for Wheel	2	1	EA
28.	<b>B01-845</b>	Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)	2	1	ST
29.	<b>A01-421</b>	End Plug for Rear Frame, Black Plastic	2	1	EA
30.	<b>B01-085</b>	Folding Mechanism Complete with Locking System for Hugo	1	1	ST
31.	<b>A01-701</b>	Tool Package (Allen Keys #4mm, #5mm & #6mm)	1	1	PK