


**700-959 (PACIFIC BLUE)
HUGO ROLLATOR
HEIGHT ADJUSTABLE, 8"x1.5" WHEELS, AND CURVED BACKREST**

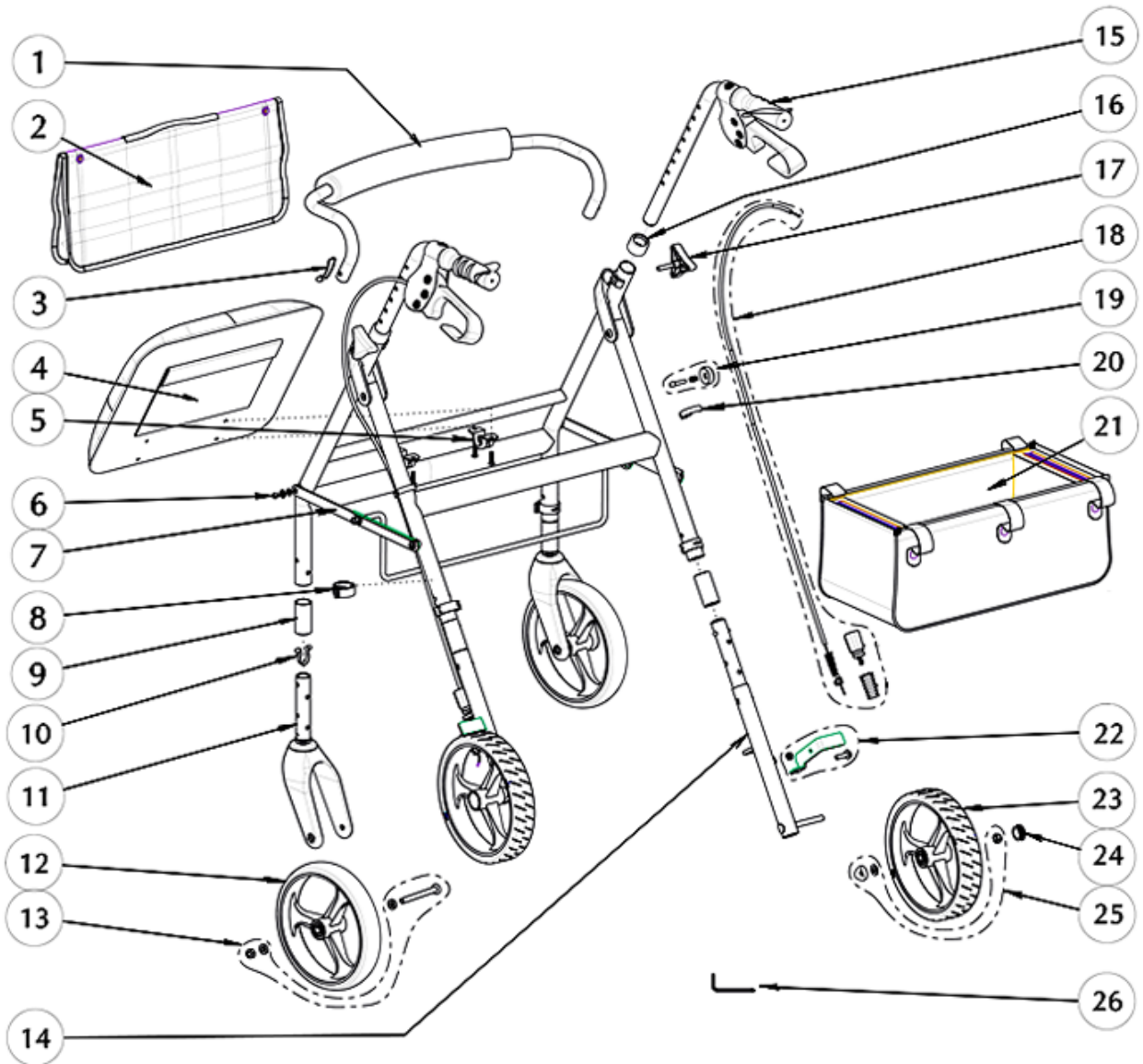

700-959 (PACIFIC BLUE)
HUGO ROLLATOR
HEIGHT ADJUSTABLE, 8"x1.5" WHEELS, AND CURVED BACKREST

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	B01-206BK	Curved Backrest Assembly (Curved Black Tube & 19mm, Foam, 2 Push-Buttons & 2 Springs)	1	1	EA
2.	A01-484	Hugo "Elite" Saddle Bag, Black	1	1	EA
3.	B01-811	Push-Button & Spring	2	1	ST
4.	B01-270	Vinyl Seat Cushion (14" x 12") with Hardware (2 Brackets, 2 Screws M5x15 & 4 Washers)	1	1	EA
5.	B01-816	Hardware for Seat (Bracket, 2 Screws M5x15mm & 2 Washers)	2	1	ST
6.	B01-824	Hardware for Folding Mechanism (4 Screws M5x6mm, 4 Steel Washers & 2 white Washers)	1	1	ST
7.	B01-085N	Folding Mechanism Complete with Locking System for Hugo	1	1	ST
8.	A01-241	E-Clip, Black – 4 ea / set	1	1	ST
9.	A01-510PB	Extension Tubing, Pacific Blue (&25.4 x 50mm) – 4 ea / set	1	1	ST
10.	A01-260	Push-Buttons(dia.5.8mm) with Spring Assembly - 2ea / set	4	1	ST
11.	B01-124N	Front Fork Assy for Hugo (Plastic Fork & Aluminum Housing with Slotted Spring Pin)	2	1	EA
12.	B01-146	8"x1.4" Hugo Front Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
13.	B01-832	Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Washers & Nut)	2	1	ST
14.	C23-00050	Rear Detachable Leg, Hight-adjustable, with M8 Bolt - L&R	1	1	PR
15.	B01-024L	Left Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Ortho-handle Grip & End Plug)	1	1	EA
	B01-024R	Right Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Ortho-handle Grip & End Plug)	1	1	EA
16.	A01-206	Anti-Rattle, Plastic	2	1	EA
17.	A01-415	Triangular Knob, M6x30mm	2	1	EA
18.	B01-181	Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Bottom Brake Adjustment, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper)	1	1	PR
19.	B01-090	Locking Subassembly for Folding Mechanism (Black Button, Spring & Pin)	1	1	ST
20.	A01-243	Cable-Clip, Black – 2 ea / set	1	1	ST
21.	A01-483	Hugo under Seat Basket (420x225x145mm), Black	1	1	EA
22.	B01-159	Bottom Brake Assembly(Bolt M6x22mm, Left Brake Pad & Nut), L&R	1	1	PR
23.	B01-147X	8"x1.5" Hugo Rear Wide Wheel (PU Flat Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
24.	A01-453	Hub Cap for Wheel	2	1	EA
25.	B01-845	Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)	2	1	ST
26.	A01-705	#4mm Allen Key	1	1	EA