


700-979 Hugo® Sidekick™ Side-Folding Rolling Walker with a Seat Blueberry

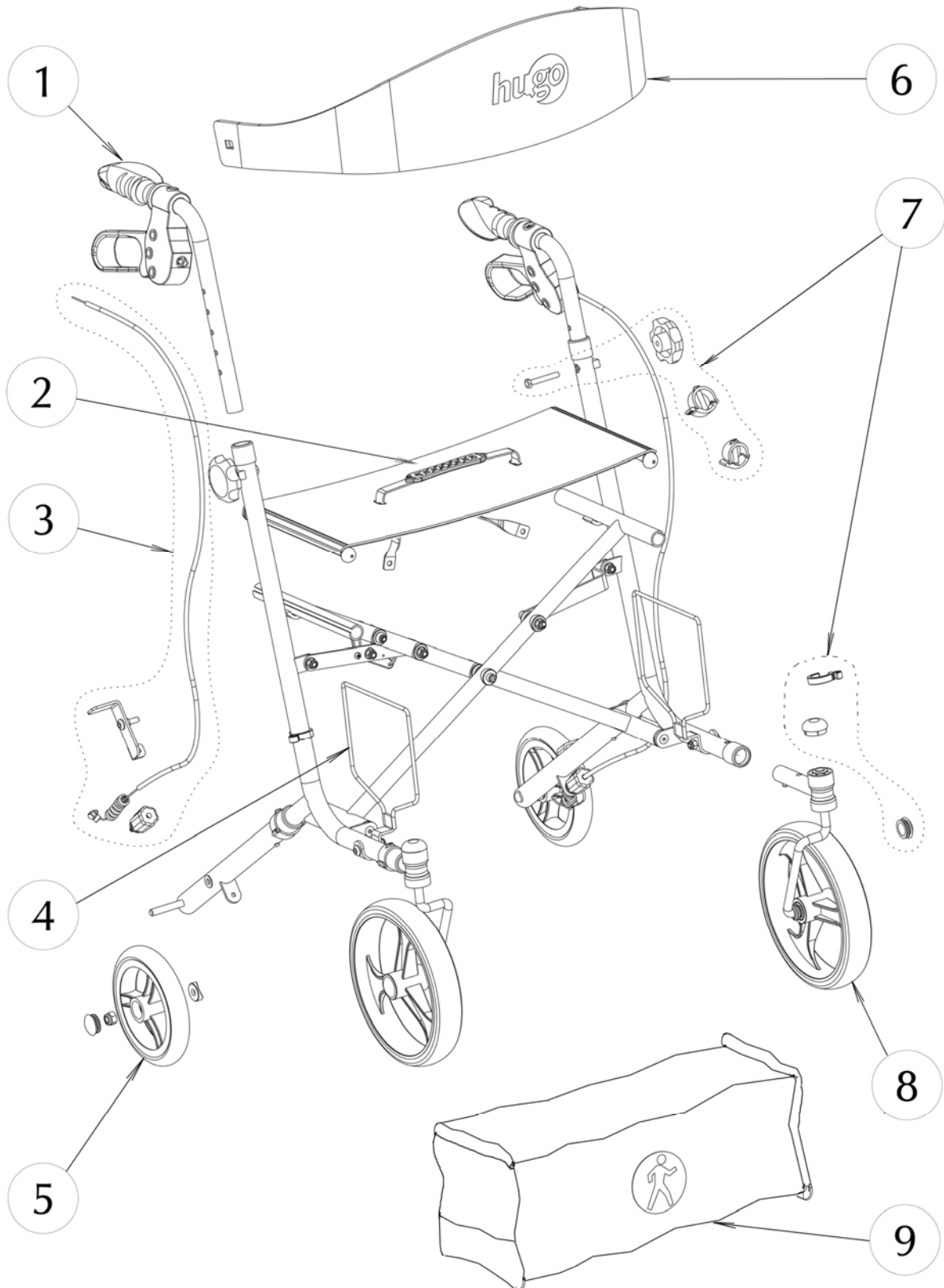

700-979
Hugo® Sidekick™ Side-Folding Rolling Walker with a Seat
Blueberry

Item #	AMG Part #	Item Description	Qty total	Qty sold
1.	C23-00016	Handle Bars with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left / Right Ortho-handle Grip & End Plug) - L & R	1 PR	1 PR
2.	C23-00017	Vinyl Seat 18" x 9" and Seat Strap with Hardware (2 steel bars & 4 Cap Screws)	1 ST	1 ST
3.	C23-00018	Brake Cable and Pad Assy (Cable, Sleeve, Brake Pad L&R, Cable Adjustment Mechanism, Spring with Spring Sleeve, Cable Stopper with Set Screw & Fishtail Stopper) – L & R	1 PR	1 PR
4.	C23-00019	Mounting Brackets with Hardware - L & R	1 PR	1 PR
5.	C23-00020	6" Rear Wheel (3 Spokes) with Hardware (Curved Washer & Locknut M8) – 2 ea / set	1 ST	1 ST
6.	C23-00021	Backrest for Sidekick	1 EA	1 EA
7.	C23-00022	Set of 2 E-Clips dia=22mm, 2 E-Clips Dia=19mm, 2 Tightening Knobs & 2 Hex Bolts M6x32mm, 2 Cable Clips, 4 Wheel Caps & 2 Fork Caps	1 ST	1 ST
8.	C23-00023	Front Leg (Blueberry) with Fork, Wheel and Hardware – L & R	1 PR	1 PR
9.	C23-00025	Hugo-Sidekick under Seat Soft Bag, Black	1 EA	1 EA