


### 793-941 (PACIFIC BLUE) HUGO NAVIGATOR

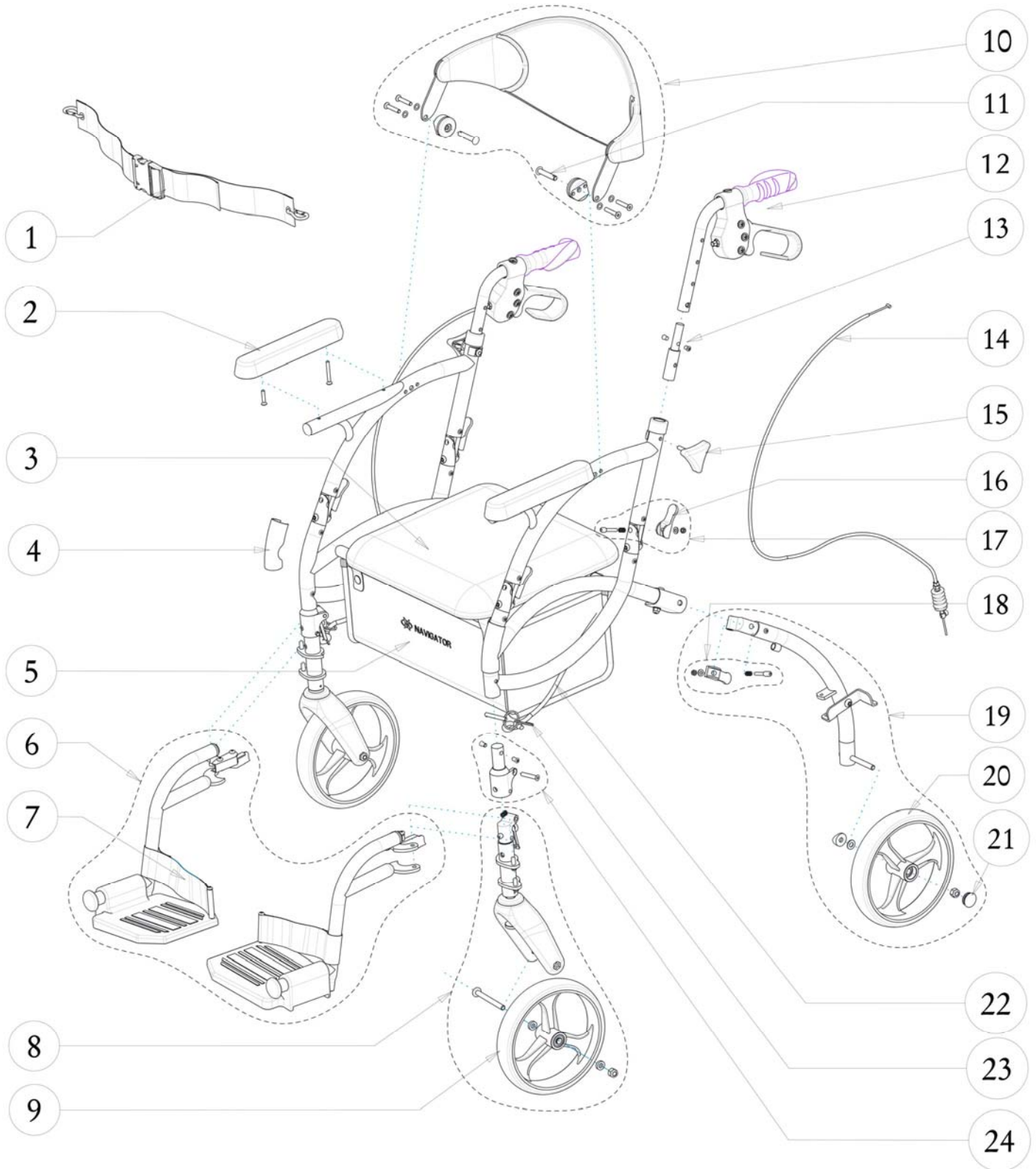

## 793-941 (PACIFIC BLUE) HUGO NAVIGATOR

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>A01-831</b>	Seatbelt with end-clips, Black	1	1	EA
2.	<b>B01-951</b>	Armrests with Hardware, L&R (2 Black Arm Pads, 4 Flat Philips Screws: 2 M5x25mm + 2 M5x40mm)	1	1	PR
3.	<b>B01-931</b>	Black Vinyl Seat 370 x 340 mm with Hardware (2 Brackets, 4 Round Philips Screws M5x15mm and 4 Washers)	1	1	EA
4.	<b>A01-861</b>	Protect-Front Leg, Black Plastic - 2 ea / set	1	1	ST
5.	<b>A01-821</b>	Under-Seat Bag w/Hugo Logo, Black	1	1	EA
6.	<b>B01-980BL</b>	Detachable Swing-Away Footrests, Pacific-Blue, L&R	1	1	PR
7.	<b>B01-991</b>	Heel Loops - 2ea / set	1	1	ST
8.	<b>B01-910BL</b>	Front Leg Complete with Fork & Wheel, Pacific-Blue, L&R	1	1	PR
9.	<b>B01-146</b>	8"x1.4" Hugo Front Wheel (Solid Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
10.	<b>B01-902</b>	Black Backrest with Hardware (2 Brackets, 2 Holding Pins, 4 Screws & 4 Washers)	1	1	EA
11.	<b>A01-801</b>	Backrest Holding Pin - 2 ea / set	1	1	ST
12.	<b>B01-961</b>	Black Handle Bar with Brake Grip Assy, L&R	1	1	PR
13.	<b>B01-966</b>	Handle Extension Tubes and Hardware (2 Tubes & 4 Set Screws M6)	1	1	ST
14.	<b>B01-941</b>	Brake Cable & Sleeve Assembly (Cable, Sleeve, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper), L&R	1	1	PR
15.	<b>A01-415</b>	Triangular Knob, M6x30mm	2	1	EA
16.	<b>A01-401</b>	Plastic Lever for Folding Mechanism, Black - 2 ea / set	3	1	ST
17.	<b>B01-925</b>	Armrest Release Folding Mechanism (4 Plastic Lever, 4 Pins, 4 Springs, 4 Washers & 4 Locknuts)	1	1	ST
18.	<b>B01-926</b>	Rear Leg Release Folding Mechanism (2 Plastic Lever, 2 Pins, 2 Springs & 2 Special Nuts)	1	1	ST
19.	<b>B01-750BL</b>	Rear Leg Complete with Wheel, Pacific-Blue, L&R	1	1	PR
20.	<b>B01-147</b>	8"x1.4" Hugo Rear Wheel (Solid Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
21.	<b>A01-453</b>	Hub Cap for Wheel	2	1	EA
22.	<b>B01-970</b>	Folding Strap with Clips and Pins	1	1	EA
23.	<b>B01-971</b>	Pin, cotterless detent, m6x 27.5mm grip length, w/tether - 2ea / set	1	1	ST
24.	<b>B01-921</b>	Fork Folding Upper Bracket with Hardware, L&R	1	1	PR