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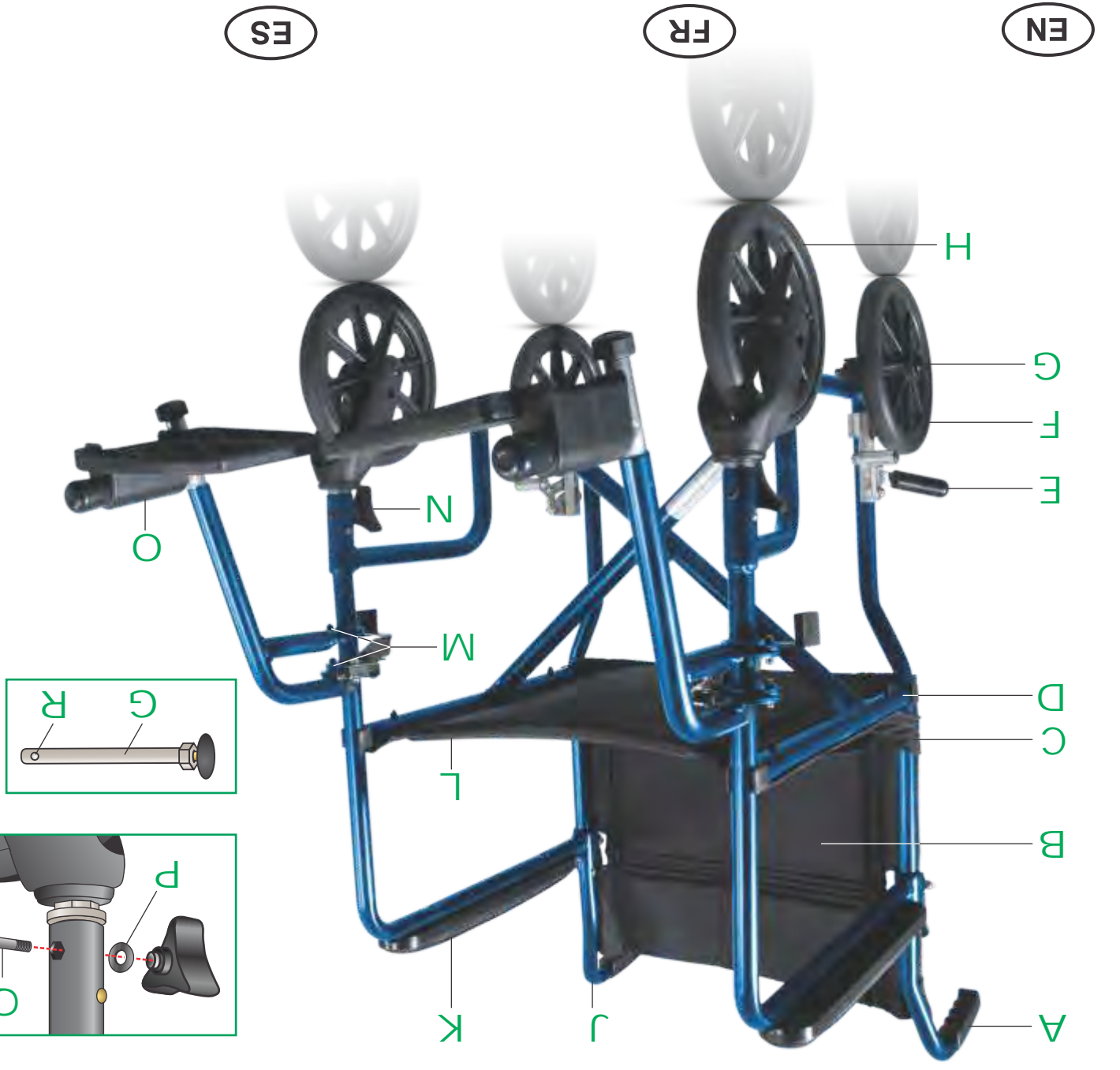
LIMITED WARRANTY • GARANTIE LIMITÉE • GARANTÍA LIMITADA

**EN**  
AMG Medical Inc. warrants the frame of this Hugo® mobility aid to be free from defects in materials and workmanship for three (3) years. All other parts and components are warranted for a period of one (1) year. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty. AMG reserves the right to replace or repair any part that has become defective.  
For warranty service, you have two options:  
1. Return your transport chair to the Vendor where the unit was bought.  
2. In Canada & the U.S. call AMG's Consumer Info Line at 1-800-363-2381 for a return authorization number.  
Please ensure you provide us with the following information when you call: Date of Purchase and Serial Number (on bar underneath the seat).  
Our Customer Service Team will provide you with a return authorization number as well as all necessary shipping instructions. Note that with your return you will be required to include a copy of the original bill of sale.

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TranSport  
Chair / Fauteuil / Silla


SAFETY PRECAUTIONS • CONSIGNES DE SÉCURITÉ  
PRECAUCIONES DE SEGURIDAD

**EN**  
**READ THESE SAFETY INSTRUCTIONS PRIOR TO USING YOUR HUGO® TRANSPORT CHAIR.**  
You and your attendant should understand all instructions in the manual before operating.  
1. Consult your physician, nurse or physical therapist before use, to determine how the Hugo® TranSport Chair can suit your particular needs.  
2. IMPORTANT – Before using, make sure that all parts are secure and that the frame of the chair is fully opened.  
3. The Hugo® TranSport Chair is NOT intended to maneuver independently, an attendant is required.  
4. IMPORTANT – Before entering or leaving the chair, engage the Wheel Locks against the tires on the Rear Wheels.  
5. Before entering or exiting the chair always ensure that 1) the chair is on a stable, level surface, 2) that the wheel locks are fully engaged, and 3) that the Footrest Plates are folded up.  
6. Do not step on the Footrest Plates while entering or leaving the chair. Do not adjust Footrest Plates any closer than 2 1/2 inches from the ground.  
7. Do not reach or lean while you are seated as you may shift the center of balance of your chair and cause it to tip. Please ask someone to help you instead. NEVER reach down between your legs to pick up an object.  
8. Always use caution when going down hills. Do not use the Wheel Locks as a brake while rolling the chair. Avoid changing direction when going down a hill.  
9. Do not operate your TranSport chair in sand or over rough terrain. This may damage the wheels, axles or loosen fasteners.  
CAUTION – DO NOT exceed the following weight limits, or serious injury could result: Handles: 200 lbs.; Seat: 250 lbs.

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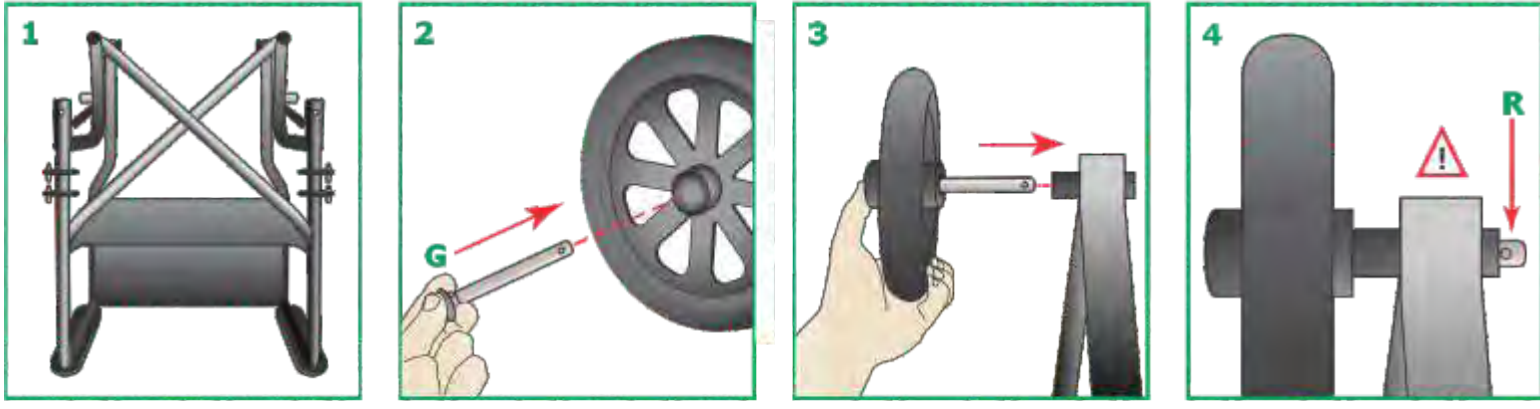EN

Leave the Backrest folded and open the seat of the chair. Carefully turn it upside-down. (1)  
Push the button on the end of the quick release axle (G) as you insert it into the recessed side of the Rear Wheel until flush with the wheel hub. (2)  
Insert Axle into the hole of the outside back leg until it locks into place. (3)  
To avoid injury, ensure the Quick Release Axle and pop button (R) have fully protruded through the back hole of the leg. (4)

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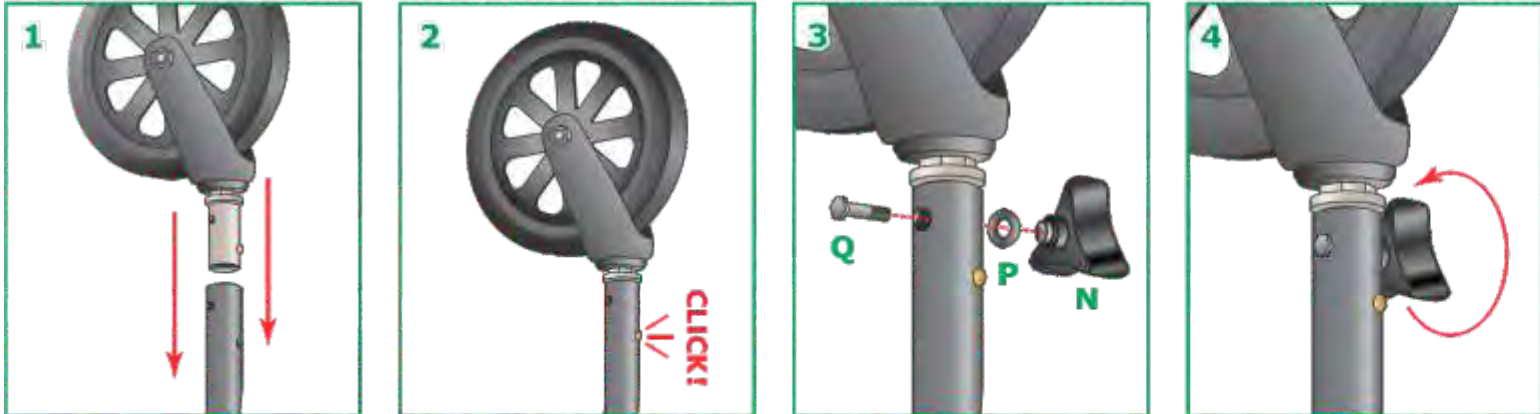
EN

Press the push buttons on either side of one of the front wheel assemblies and insert it into the frame until the push buttons lock into place. (1-2)  
Insert one Hex Screw (Q) into the hex hole, then slide the washer (P) over the protruding end. (3)  
Secure in place using the Triangular Hand Screw (N). Hand tighten only. (4)

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EN

Stand chair upright on all four wheels. Tilt to one side and push down on the side seat frame bars. Firmly push down on the seat frame under the armrests to ensure the seat is fully deployed.

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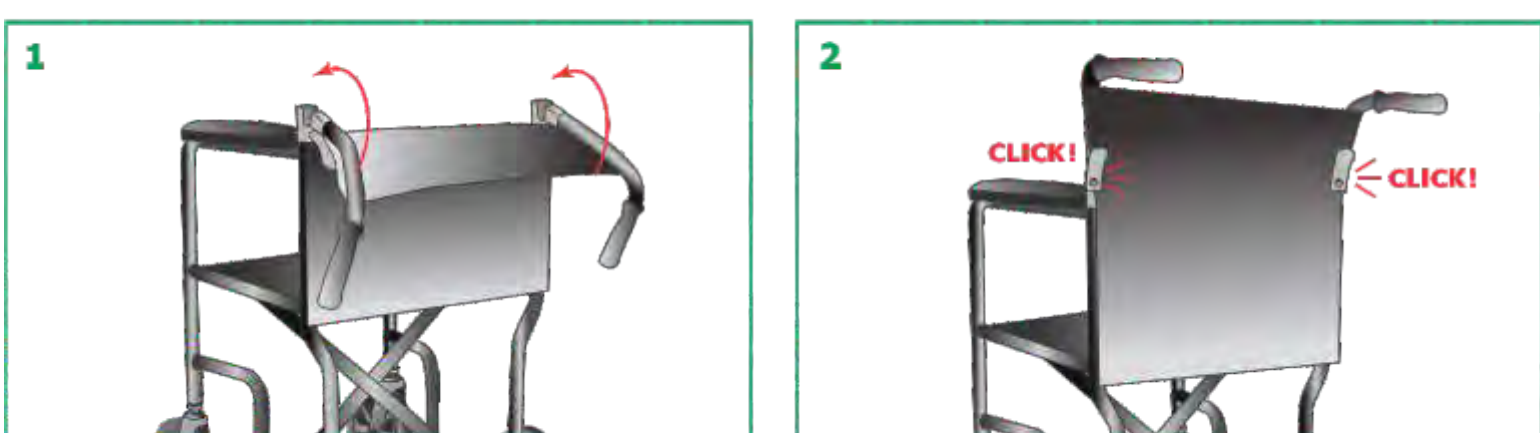
### 6 ENGAGE BACKREST • ENLENCHEZ LE DOSSIER • APERTURA DEL RESPALDO

EN

Grab both handlebars tucked next to frame and pull up lifting the backrest into the upright position (1) until the handlebars lock securely into place (2).

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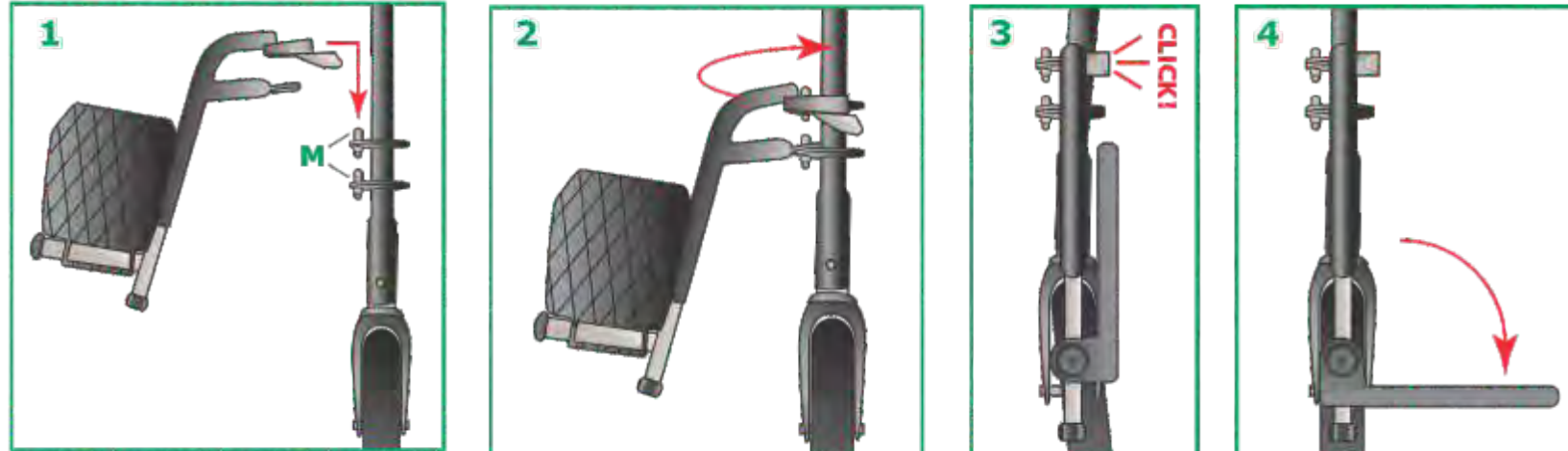
EN

The chair has a right and left footrest. These must be installed correctly to ensure safety. To install footrest, face the front of the chair looking at the seat. Align the footrest holes with Alignment Pins (M) on the front leg of the frame (1). Ensure that both pins are engaged, then swing Footrest (2) toward the center of chair until it locks into place (3). Repeat on opposite side. Do not lower footrest plates until safely in the chair (4). To enter or exit the chair, footrest plates must be folded up.

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### 8 ADJUST FOOTREST • AJUSTEZ LES REPOSE-PIEDS • AJUSTE DEL APOYAPIE

EN

To adjust the length of the footrest, loosen the bolt at the bottom of the telescoping tube (1). Place in desired position, not lower than 2 1/2 inches from the ground (2). Tighten bolt securely (3).

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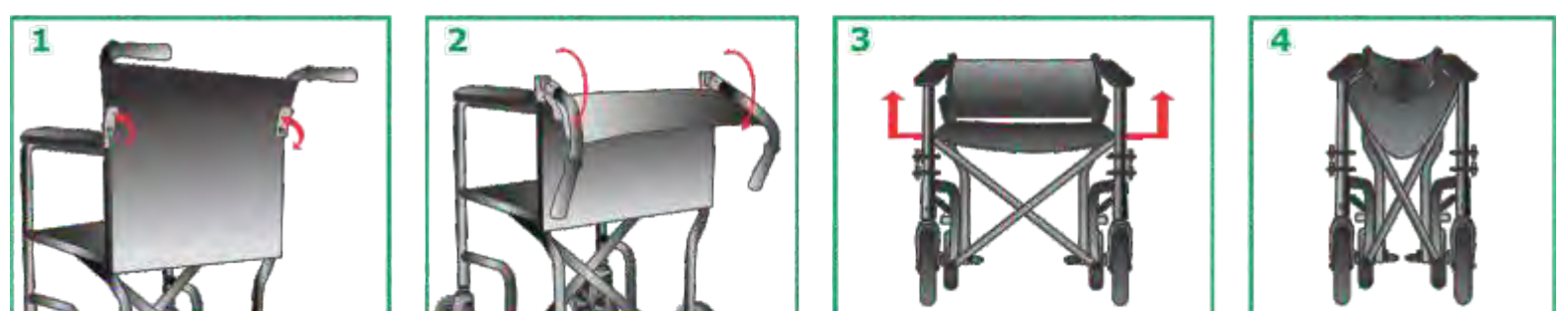
### 9 STORING THE CHAIR • POUR PLIER LE FAUTEUIL • PARA GUARDAR LA SILLA

EN

To close/fold or store chair, release locks on both sides of back of chair (1) and fold down the backrest (2). Stand to one side and pull up on the folding handles (3-4).

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### ! MAINTENANCE • ENTRETIEN • MANTENIMIENTO

EN

- Wipe off frame regularly using a clean soft cloth. Do not use abrasive materials that could scratch the finish of the chair. If the chair is exposed to moisture, dry immediately.
- You can clean the upholstery with a mild soap solution. Rinse immediately and wipe dry.
- Worn or torn upholstery should be replaced promptly as it may not support your body weight.
- Clean tires with a damp cloth. Replace only if they become cracked or severely worn.

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