

MedPro®

The Professional Choice
Le choix des professionnels

Adjustable Back Support

Designed to help reduce the risk of lower back injury by supporting abdominal and back muscles, while encouraging proper lifting techniques.

Instructions

1. Wrap support around waist. Elastic tension straps should be loose.
2. Close belt in front using grip fastener. Adjust for a snug, but comfortable fit. Belt should be worn low on hips.
3. After belt has been adjusted, fasten elastic tension straps.
4. Adjust suspenders.



Washing Instructions

1. Close grip fastener to prevent damage to other fabrics.
2. Machine wash in lukewarm water with mild soap. Do not dry clean.
3. Drip dry. Do not wring out.

1 Year Limited Warranty

A.M.G. Medical Inc. warrants this product to be free from defects in material and workmanship for a period of one (1) year. This warranty is valid for the original purchaser only. Any alterations, abuse, misuse or accidental damage voids this warranty.

For replacement or repair under warranty, please call:
1-800-363-2381 (Monday to Friday, 8:30 am to 5:00 pm EST)

For more information on this, or any other MedPro® product, visit: www.amgmedical.com

A.M.G. Medical Inc.

Montréal, QC H4T 1V5 • **1-800-363-2381**

West Chazy, NY 12992 • **1-888-412-4992**

www.amgmedical.com