
INSTALLATION INSTRUCTIONS

For proper installation, the grab bar should always be aligned with the mounting brackets positioned over wall studs. The screws must go through the wall board or tile and into the wall studs for solid support. Standard wall stud spacing is usually 16 or 24 inches between stud centers.

To locate the wall studs, tap on the wall and listen for a dull sound, or if possible use a commercial wall stud finder. When a wall stud has been found, measure 16 or 24 inches from that point to locate the next wall stud. Shown below are some mounting tips to help in attaching the grab bars securely.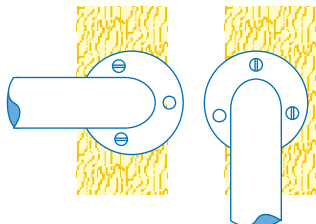

MAINTENANCE

Check grab bars regularly to be sure the attachments are secure. Tighten loose screws making sure they are still solidly secured in the wall stud. To clean grab bars, wipe with a soft cloth using mild soap and water.

SAFETY TIPS

For added safety when using grab bars the following steps are recommended.

1. Always be sure grab bars are correctly and securely installed. Never use grab bars if any looseness in the attachments is noticed.
2. Securely grip grab bars before beginning any bath or toilet activity. Use caution on wet tub or floor surfaces.

CAUTION

The wood screws provided are designed to be attached to the wooden wall studs only. If you are unable to locate a wall stud, or if any of the screws miss the studs the wood screws must not be used.

2 YEAR LIMITED WARRANTY

This product is warranted against defects in materials and workmanship for two (2) years from the date of purchase. This warranty does not cover product abuse or normal wear.

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