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EN

This Profilio™ product is guaranteed to be free from defects in materials and workmanship for five years. This warranty does not extend to non-durable components which are subject to normal wear and replacement. Any alterations, abuse, misuse, or accidental damage voids this warranty. Rental of this product voids warranty. For replacement or repair under warranty, please call:

CANADA & INTL: 1-866-518-6099 (Monday to Friday, 8 am to 5 pm EST)

USA: 1-866-820-2086 (Monday to Friday, 8 am to 5 pm EST)

For more information on this, or any other Profilio™ product, visit: www.amgmedical.com

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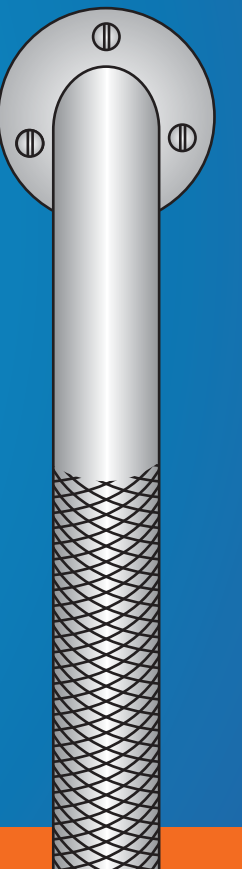
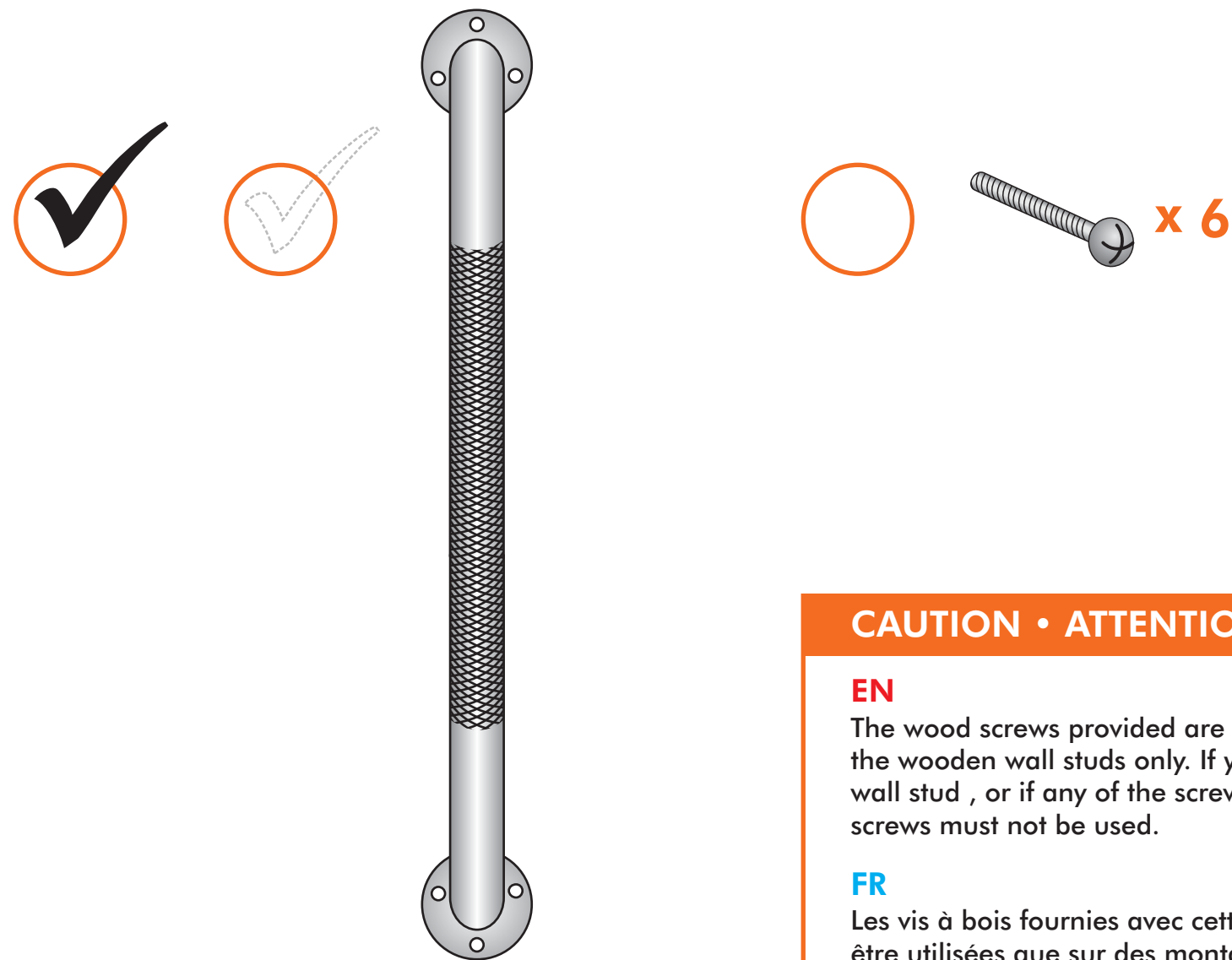
CAUTION • ATTENTION

EN

The wood screws provided are designed to be attached to the wooden wall studs only. If you are unable to locate a wall stud, or if any of the screws miss the studs the wood screws must not be used.

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2 Preparation • Préparation

EN

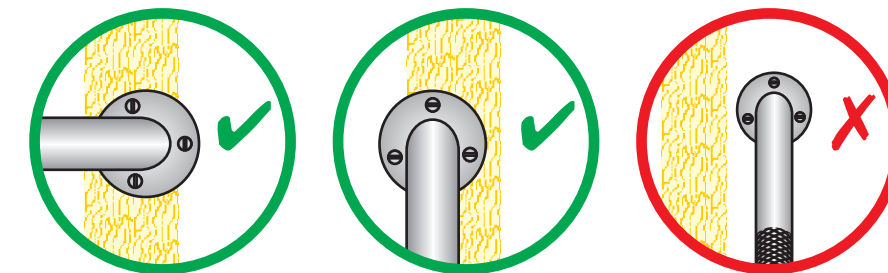
- Position the grab bar where you would like it to be, on the wall.
- Find the wall studs and align the grab bar according to these. Then use a marker to indicate where the holes for the screws should be. Using an appropriate drill bit for the wall material, drill a hole that will enable you to reach the stud with the mounting hardware. Do not drill through the wooden wall stud.

Note: an appropriate drill bit (ceramic, stone, etc.) will ensure that minimal damage will occur, to the wall.

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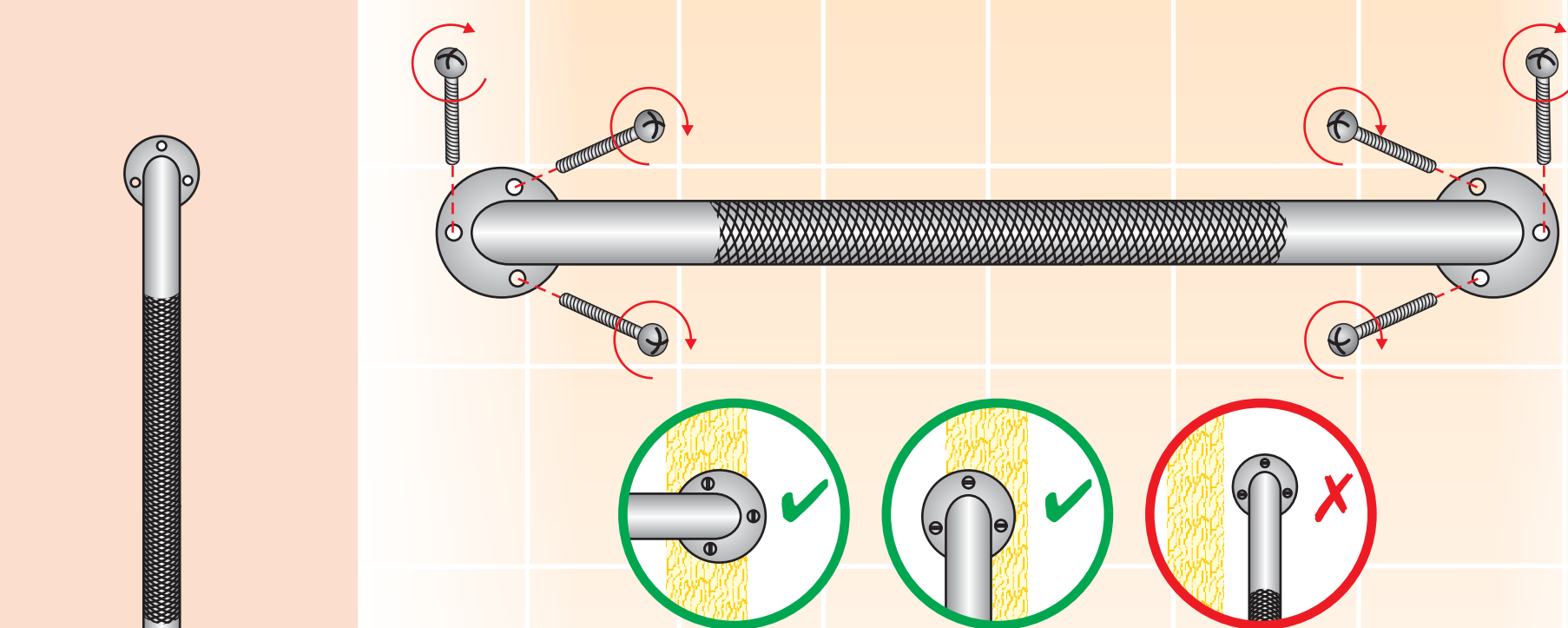
CAUTION • ATTENTION

EN

For proper installation, the grab bar should always be aligned with the mounting brackets positioned over wall studs. **The screws must go through the wall board or tile and into the wall studs for solid support.** Standard wall stud spacing is usually 16 or 24 inches between stud centers.

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EN

For added safety when using grab bars the following steps are recommended.

1. Always be sure grab bars are correctly and securely installed. Never use grab bars if any looseness in the attachments is noticed.
2. Securely grip grab bars before beginning any bath or toilet activity. Use caution on wet tub or floor surfaces.

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MAINTENANCE • ENTRETIEN

EN

Check grab bars regularly to be sure the attachments are secure. Tighten loose screws making sure they are still solidly secured in the wall stud. To clean grab bars, wipe with a soft cloth using mild soap and water.

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