


700-959 (PACIFIC BLUE) HUGO ROLLATOR 5.0

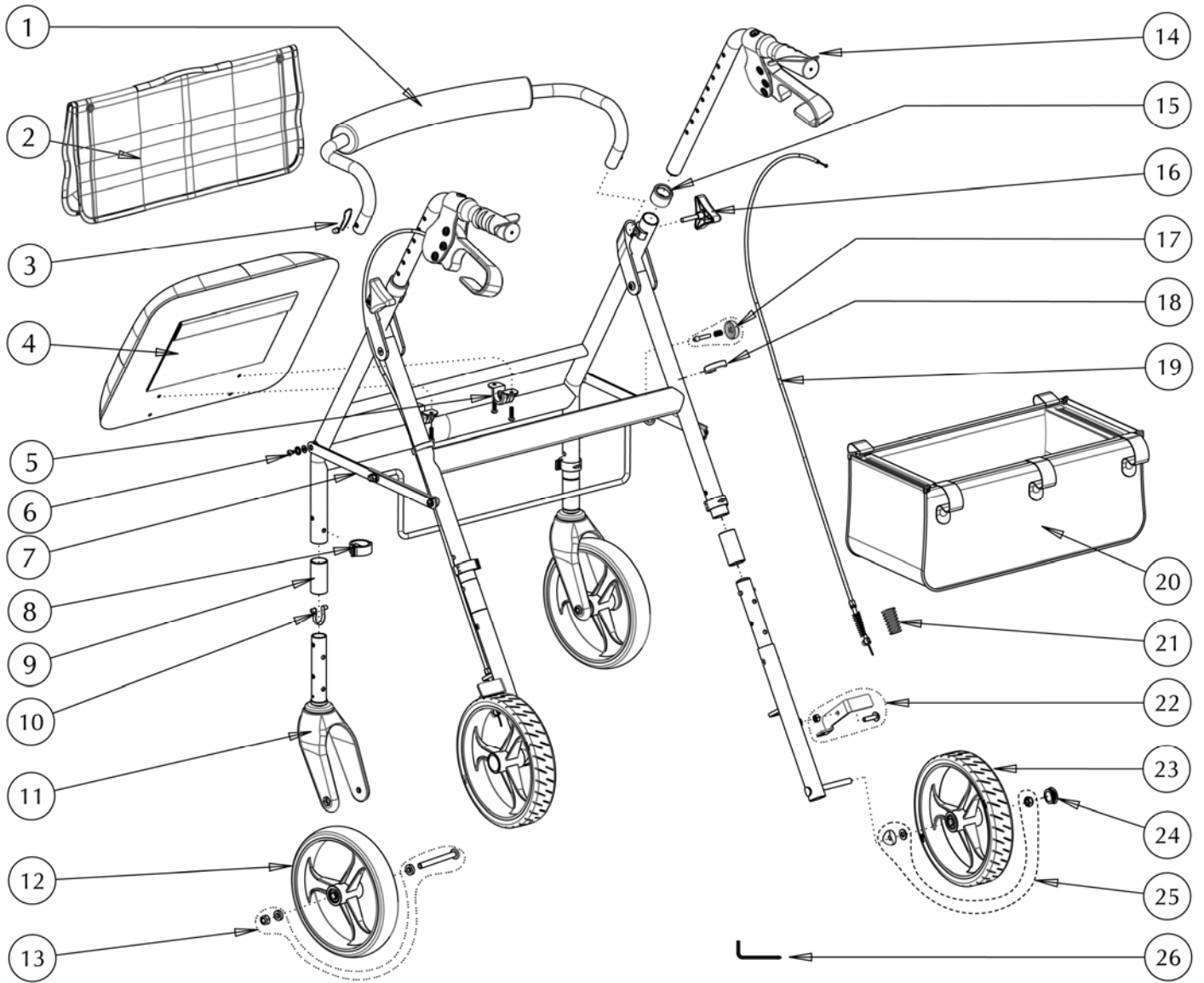

700-959 (PACIFIC BLUE) HUGO ROLLATOR 5.0

| Item # | AMG Part # | Item Description | Qty Per Unit | Sold by | UOM |
|--------|------------------|--|--------------|---------|-----|
| 1. | B01-206BK | Curved Backrest Assembly (Curved Black Tube & 19mm, Foam, 2 Push-Buttons & 2 Springs) | 1 | 1 | EA |
| 2. | A01-484 | Hugo "Elite" Saddle Bag, Black | 1 | 1 | EA |
| 3. | B01-811 | Push-Button & Spring | 2 | 1 | ST |
| 4. | B01-270 | Vinyl Seat Cushion (14" x 12") with Hardware (2 Brackets, 2 Screws M5x15 & 4 Washers) | 1 | 1 | EA |
| 5. | B01-816 | Hardware for Seat (Bracket, 2 Screws M5x15mm & 2 Washers) | 2 | 1 | ST |
| 6. | B01-824 | Hardware for Folding Mechanism (4 Screws M5x6mm, 4 Steel Washers & 2 white Washers) | 1 | 1 | ST |
| 7. | B01-085N | Folding Mechanism Complete with Locking System for Hugo | 1 | 1 | ST |
| 8. | A01-241 | E-Clip, Black – 4 ea / set | 1 | 1 | ST |
| 9. | A01-510PB | Extension Tubing, Pacific Blue (& 25.4 x 50mm) – 4 ea / set | 1 | 1 | ST |
| 10. | A01-260 | Push-Buttons(dia.5.8mm) with Spring Assembly - 2ea / set | 4 | 1 | ST |
| 11. | B01-124N | Front Fork Assy for Hugo (Plastic Fork & Aluminum Housing with Slotted Spring Pin) | 2 | 1 | EA |
| 12. | B01-146 | 8"x1.4" Hugo Front Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings) | 2 | 1 | EA |
| 13. | B01-832 | Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Washers & Nut) | 2 | 1 | ST |
| 14. | B01-024L | Left Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Ortho-handle Grip & End Plug) | 1 | 1 | EA |
| | B01-024R | Right Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Ortho-handle Grip & End Plug) | 1 | 1 | EA |
| 15. | A01-206 | Anti-Rattle, Plastic | 2 | 1 | EA |
| 16. | A01-415 | Triangular Knob, M6x30mm | 2 | 1 | EA |
| 17. | B01-090 | Locking Subassembly for Folding Mechanism (Button, Spring & Pin) | 1 | 1 | ST |
| 18. | A01-243 | Cable-Clip, Black – 2 ea / set | 1 | 1 | ST |
| 19. | B01-157 | Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper) | 2 | 1 | ST |
| 20. | A01-483 | Hugo under Seat Basket (420x225x145mm), Black | 1 | 1 | EA |
| 21. | A01-419 | Spring Sleeve, Black | 2 | 1 | EA |
| 22. | B01-159 | Bottom Brake Assembly, L&R (2 Bolts M6x22mm, Left&Right Brake Pad & 2 Hex Nuts) | 1 | 1 | ST |
| 23. | B01-147X | 8"x1.5" Hugo Rear Wide Wheel (PU Flat Tire, 3 Spokes Hub, Bushing & 2 Bearings) | 2 | 1 | EA |
| 24. | A01-453 | Hub Cap for Wheel | 2 | 1 | EA |
| 25. | B01-845 | Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer) | 2 | 1 | ST |
| 26. | A01-705 | #4mm Allen Key | 1 | 1 | EA |