


700-961 (GARNET RED) HUGO Elite ROLLATOR

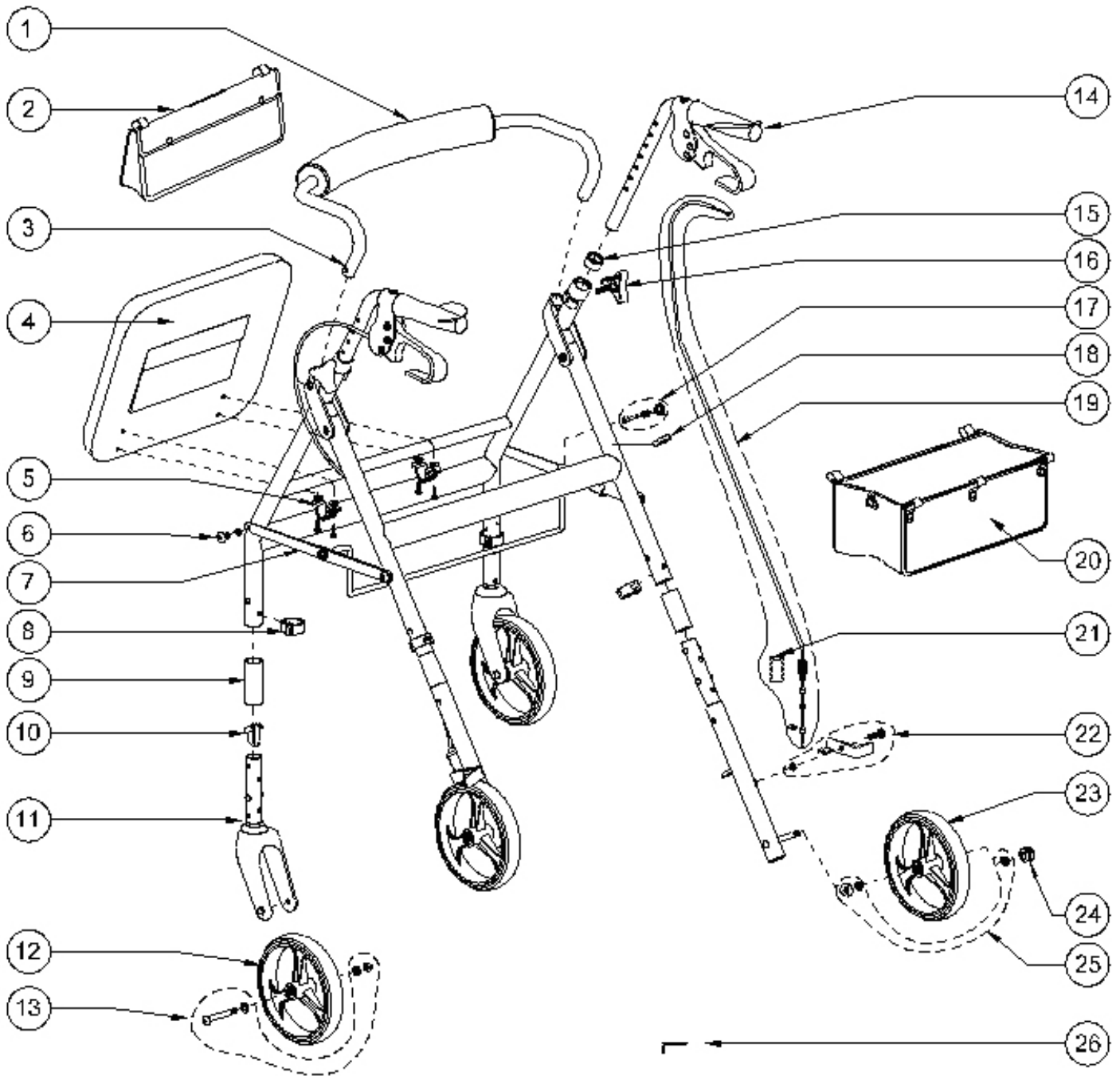

700-961 (GARNET RED) HUGO Elite ROLLATOR

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	B01-206BK	Curved Backrest Assembly (Curved Black Tube & 19mm, Foam, 2 Push-Buttons & 2 Springs)	1	1	EA
2.	A01-484	Hugo "Elite" Saddle Bag, Black	1	1	EA
3.	B01-811	Push-Button & Spring	2	1	ST
4.	B01-270	Vinyl Seat Cushion (14" x 12") with Hardware (2 Brackets, 2 Screws M5x15 & 4 Washers)	1	1	EA
5.	B01-816	Hardware for Seat (Bracket, 2 Screws M5x15mm & 2 Washers)	2	1	ST
6.	B01-822	Hardware for Folding Mechanism (4 Screws, 4 Steel Washers & 4 white Washers)	1	1	ST
7.	B01-085	Folding Mechanism Complete with Locking System for Hugo	1	1	ST
8.	A01-241	E-Clip, Black – 4 ea / set	1	1	ST
9.	A01-510GR	Extension Tubing, Garnet-Red (& 25.4 x 50mm) – 4 ea / set	1	1	ST
10.	A01-260	Push-Buttons(dia.5.8mm) with Spring Assembly - 2ea / set	4	1	ST
11.	B01-124N	Front Fork Assy for Hugo (Plastic Fork & Aluminum Housing with Slotted Spring Pin)	2	1	EA
12.	B01-146	8"x1.4" Hugo Front Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
13.	B01-832	Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Washers & Nut)	2	1	ST
14.	B01-024L	Left Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Ortho-handle Grip & End Plug)	1	1	EA
	B01-024R	Right Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Ortho-handle Grip & End Plug)	1	1	EA
15.	A01-206	Anti-Rattle, Plastic	2	1	EA
16.	A01-415	Triangular Knob, M6x30mm	2	1	EA
17.	B01-090	Locking Subassembly for Folding Mechanism (Button, Spring & Pin)	1	1	ST
18.	A01-243	Cable-Clip, Black – 2 ea / set	1	1	ST
19.	B01-157	Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper)	2	1	ST
20.	A01-483	Hugo under Seat Basket (420x225x145mm), Black	1	1	EA
21.	A01-419	Spring Sleeve, Black	2	1	EA
22.	B01-159	Bottom Brake Assembly, L&R (2 Bolts M6x22mm, Left&Right Brake Pad & 2 Hex Nuts)	1	1	ST
23.	B01-147	8"x1.4" Hugo Rear Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
24.	A01-453	Hub Cap for Wheel	2	1	EA
25.	B01-845	Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)	2	1	ST
26.	A01-705	Allen Key #4mm	1	1	EA