


### 700-961 (GARNET-RED) HUGO ROLLATOR 5.0

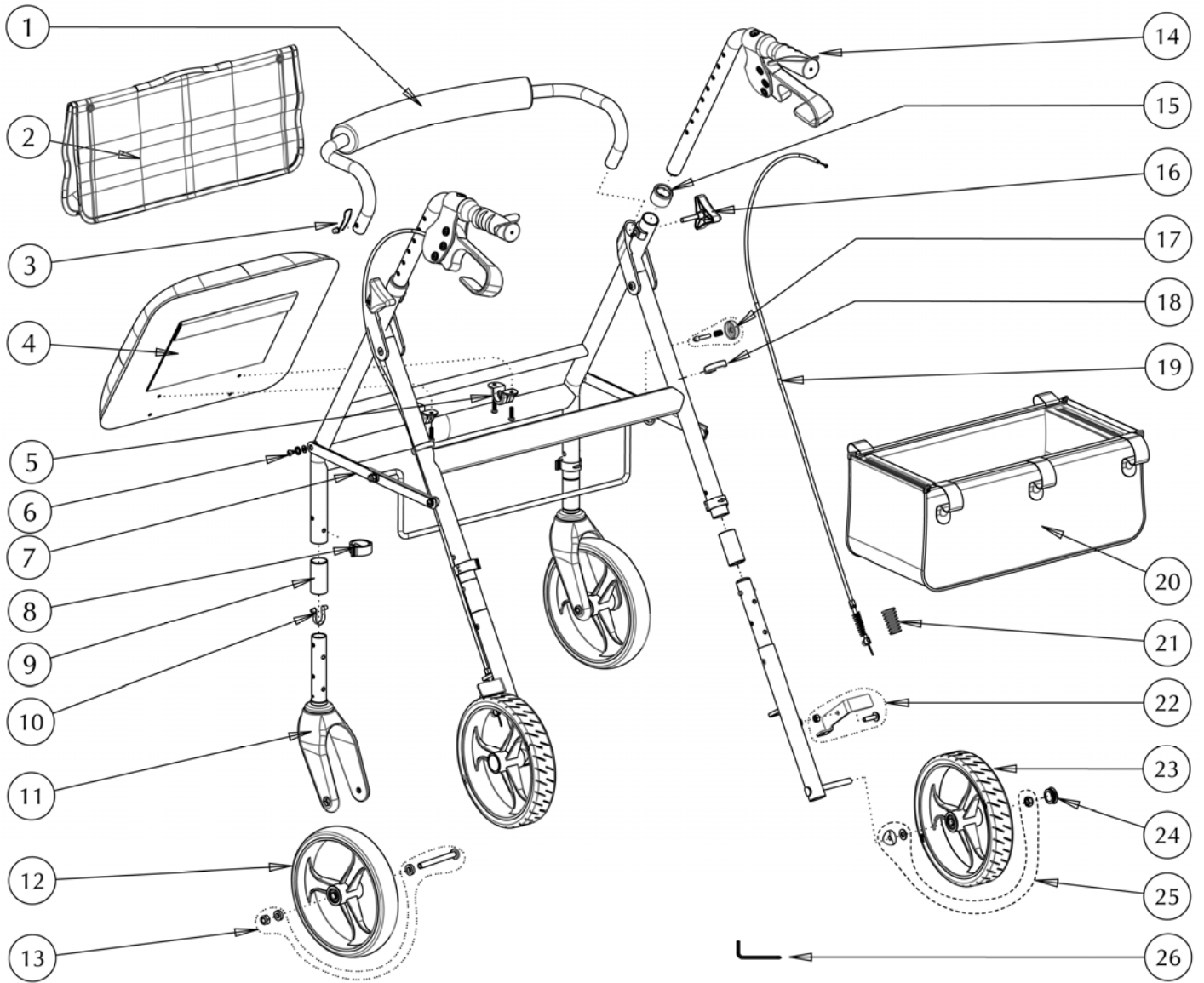

## 700-961 (GARNET-RED) HUGO ROLLATOR 5.0

Item #	AMG Part #	Item Description	Qty Per Unit	Sold by	UOM
1.	<b>B01-206BK</b>	Curved Backrest Assembly (Curved Black Tube & 19mm, Foam, 2 Push-Buttons & 2 Springs)	1	1	EA
2.	<b>A01-484</b>	Hugo "Elite" Saddle Bag, Black	1	1	EA
3.	<b>B01-811</b>	Push-Button & Spring	2	1	ST
4.	<b>B01-270</b>	Vinyl Seat Cushion (14" x 12") with Hardware (2 Brackets, 2 Screws M5x15 & 4 Washers)	1	1	EA
5.	<b>B01-816</b>	Hardware for Seat (Bracket, 2 Screws M5x15mm & 2 Washers)	2	1	ST
6.	<b>B01-824</b>	Hardware for Folding Mechanism (4 Screws M5x6mm, 4 Steel Washers & 2 white Washers)	1	1	ST
7.	<b>B01-085N</b>	Folding Mechanism Complete with Locking System for Hugo	1	1	ST
8.	<b>A01-241</b>	E-Clip, Black – 4 ea / set	1	1	ST
9.	<b>A01-510GR</b>	Extension Tubing, Garnet-Red (& 25.4 x 50mm) – 4 ea / set	1	1	ST
10.	<b>A01-260</b>	Push-Buttons(dia.5.8mm) with Spring Assembly - 2ea / set	4	1	ST
11.	<b>B01-124N</b>	Front Fork Assy for Hugo (Plastic Fork & Aluminum Housing with Slotted Spring Pin)	2	1	EA
12.	<b>B01-146</b>	8"x1.4" Hugo Front Wheel (PU Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
13.	<b>B01-832</b>	Hardware for 8" Front Wheel (Bolt M8x78.5mm, 2 Washers & Nut)	2	1	ST
14.	<b>B01-024L</b>	Left Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Left Ortho-handle Grip & End Plug)	1	1	EA
	<b>B01-024R</b>	Right Handle Bar with Brake Handle Assy (Handle Bar, Brake Handle with Housing & Hardware, Right Ortho-handle Grip & End Plug)	1	1	EA
15.	<b>A01-206</b>	Anti-Rattle, Plastic	2	1	EA
16.	<b>A01-415</b>	Triangular Knob, M6x30mm	2	1	EA
17.	<b>B01-090</b>	Locking Subassembly for Folding Mechanism (Button, Spring & Pin)	1	1	ST
18.	<b>A01-243</b>	Cable-Clip, Black – 2 ea / set	1	1	ST
19.	<b>B01-157</b>	Brake Cable & Sleeve Assembly (Cable, Sleeve 860mm, Adjustment Screw, Spring, Spring Sleeve, Cable Stopper with Screw M5x6mm & Fishtail Stopper)	2	1	ST
20.	<b>A01-483</b>	Hugo under Seat Basket (420x225x145mm), Black	1	1	EA
21.	<b>A01-419</b>	Spring Sleeve, Black	2	1	EA
22.	<b>B01-159</b>	Bottom Brake Assembly, L&R (2 Bolts M6x22mm, Left&Right Brake Pad & 2 Hex Nuts)	1	1	ST
23.	<b>B01-147X</b>	8"x1.5" Hugo Rear Wide Wheel (PU Flat Tire, 3 Spokes Hub, Bushing & 2 Bearings)	2	1	EA
24.	<b>A01-453</b>	Hub Cap for Wheel	2	1	EA
25.	<b>B01-845</b>	Hardware for Rear Wheel (Locknut M8, Curved Washer & Steel Washer)	2	1	ST
26.	<b>A01-705</b>	#4mm Allen Key	1	1	EA